

PRODUCT INSTRUCTION SHEET

REFERENCE

Belgrave Bariatric Commode
(190, 191, 192, 193)

DESCRIPTION

All welded steel frame with cellular moulded polyurethane backrest and over seat, with clip/on-off rota moulded seat and circular 14L potty. Available with fixed or adjustable seat height, and fixed or detachable armrests.

TECHNICAL SPECIFICATIONS

- Seat Height Min (Adjustable): 43cm (17")
- Seat Height Max (Adjustable): 58cm (23")
- Seat Height (Fixed): 43cm (17")
- Seat Width (between arms): 61cm (24")
- Seat Depth: 43cm (17")
- Width: 66cm (26")
- Depth: 54cm (21")
- Seat to top of arms: 18cm (7")
- Seat to top of back: 39cm (15.5")
- Aperture width: 25cm (10")
- Aperture depth: 28cm (11")
- Product weight 13kg (28lb)
- Maximum Weight Capacity: 280kg (44st)



SAFETY WARNING

- Appliances to be used on a level surface only.
- When issuing a shower chair, a risk assessment should be carried out to determine the suitability of the equipment for the user, with emphasis on the risk of entrapment.
- Do not exceed the weight limit or swing on the frames in any direction.
- Ensure that any 'E' clips securing the height adjustable legs or detachable armrests are present and fully inserted.

CLEANING AND CARE

- This product is intended for use in a wet room or shower.
- It is not intended for use outside and should not be immersed in water for long periods of time.
- To clean, use a mild detergent and warm water.
- Ensure the product is dried thoroughly after use or cleaning.

WARRANTY

Your Drive DeVilbiss Healthcare branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment.

This warranty is an expression of our confidence in the materials and workmanship of our products, and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist. This does not affect your statutory rights.