

Inelastic adhesive strip

Relieving tape using Cetylated Fatty Acids (CFAs)

USAGE: Cetilar® Tape is an inhelastic adhesive strip that, thanks to its formulation, is an adjuvant for skin treatment useful for temporary relief of minor aches.

HOW TO USE:

PLIT OUT



Onen the package and take out the strip.

CUT



2. Cut the strip to the required size.

APPI Y



Make sure the skin is clean and dry before applying. Press lightly onto skin and ensure that the edges are sticking to the skin. Keep in situ for 8 hours.

PLIT AWAY



Put the unused product back in its packaging.

Product can be applied repeatedly throughout the day, leaving the skin exposed to air for a short period of time between applications to prevent sensitisation. The adhesive strip is intended for single use and cannot be reused.

INGREDIENTS: Cetyl Myristate, Cetyl Oleate, Cetyl Linoleate, Cetyl Palmitate, Cetyl Stearate, Cetyl Palmitoleate, Cetyl Laurate, acrylate copolymer, menthol.

WARNINGS: Keep out of the reach and sight of children. For external use only. The product should only be applied to intact, clean and dry skin. Avoid contact with mucous membranes, eyes and mouth, if necessary, wash abundantly with water. Remove the adhesive strip during bathing/showering. If apparent signs of irritation or hypersensitivity to the product occur, discontinue use. Do not use in case of known hypersensitivity to the components. Do not ingest. Store in a cool, dry place, between 41°F and 104°F. Do not use the product if the packaging is damaged. Do not use after the expiry date. Do not use Cetilar® Tape with Menthol at the same time as other topical treatments in the area to be treated, without consulting your physician. If you are pregnant or breastfeeding, consult your physician before using the product. For use in children under 12 years of age, consult your physician before using the product.

EXTERNAL USE

Batch and expiry date: see bottom of package.



Distributed in U.S.A. by Country of origin: Italy











Produced by Pharmanutra S.p.A, Via delle Lenze 216/B 56122 (PI) Italy

PHARMANUTRA