

### hemaPEN® | W300S

Research use only

<b>Intended use</b>	<b>Research use only.</b> hemaPEN® is designed to collect and store accurate and precise micro volumes of blood or a bio-sample. Samples are transferred from a collection capillary on to a substrate and stored in a dried blood spot format suitable for shipping by standard postal service for analysis in a central laboratory. hemaPEN RUO W300S is a research only device and does not have regulatory approval for medical use.
<b>Product</b>	The hemaPEN is a hand held device for blood microsampling. As the device name implies, the hemaPEN has a pen-like design and is supplied as a non-sterile, single use device for the collection of small accurate volumes of blood, via four (4) small glass capillaries that protrude slightly from the body of the hemaPEN. After blood collection, the device is engaged into its protective cap and the blood transfers from the capillaries onto four discrete 3.5 mm pre-punched substrate discs. The four replicate blood spots are contained within the body of the device and the blood dries within one hour. The dried samples within the device can then be transported under ambient conditions to the testing laboratory using standard postal services. The most common blood source is anticipated to be derived capillary blood from a finger stick draw however hemaPEN's use is not limited to the collection of capillary blood.
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• Follow instructions for use.</li> <li>• Recommended use by research personnel and healthcare professionals trained on the device.</li> <li>• Single use only.</li> <li>• Contains glass capillaries. Do not use if the capillaries are broken.</li> <li>• Use the hemaPEN device within 30 minutes once removed from the silver polyfoil bag.</li> </ul>
<b>Storage instructions</b>	<ul style="list-style-type: none"> <li>• Store the devices in original packaging at room temperature, in a dry area.</li> <li>• Avoid direct sunlight exposure.</li> <li>• After use, store the hemaPEN device sealed in original packaging. Store or transport product at room temperature.</li> </ul>
Manufactured in accordance to Trajan's internal Quality Management System.	

Technical data	
Part number	498100011
Product features	
Sample type	Whole blood, serum, plasma or other biological fluids
Storage format	Dried
Sample volumetric accuracy*	2.74 ± 0.14 µL
Volumetric precision*	CV < 0.6% (interdevice)
Substrate*	903® 100% cotton fibre supplied by Eastern Business Forms Inc.
Number of samples	4
Number of capillaries	4
Capillary specification	Borosilicate 0.36 mm internal bore diameter
Number of pre-punched discs	4
Disc dimensions	3.5 mm diameter
Shelf life	Six months

\* Based on preliminary data generated by Trajan Scientific Australia Pty. Ltd.

\* 903® is a registered trademark of Eastern Business Forms, Inc.

The hemaPEN is under development and the prototype device is supplied for research or investigational purposes only. This device is not for therapeutic or diagnostic use.

Technical data	
<b>Material and colours</b>	
Product colour	White main body, a clear plastic base and a green indicator stripe
Product material	Modified poly (acrylonitrile-butadiene-styrene) thermoplastic
Capillary material	Borosilicate glass HGB1 3.3 for laboratory glassware ISO3585
Desiccant	Multiform CSF® - Silica gel with high density polypropylene filler and hydroxypropyl methylcellulose with titanium oxide
Clear base	Poly (bisphenol-A-carbonate) thermoplastic
Tape	Polyester film with an acrylic adhesive polymer
Packaging material	Polyethylene terephthalate co polyester blister tray in a polyfoil bag
<b>Packaging</b>	
Units/pack	Provided as a single unit per pack
Pack dimensions	160 x 230 x 50 mm

### Australia and New Zealand

Trajan Scientific Australia Pty Ltd  
 7 Argent Place, Ringwood  
 Victoria 3134, Australia  
 Toll Free: 1800 257 213  
 Tel Melbourne: +61 (0) 3 9874 8577  
 anz@trajanscimed.com

### Americas

Trajan Scientific Americas Inc  
 Wells Branch Tech Center - Austin TX  
 1421 W Wells Branch Pkwy # 108  
 Pflugerville, Texas 78660, USA  
 Toll Free: 800 945 6154  
 Tel: 512 837 7190  
 Fax: 512 836 9159  
 usa@trajanscimed.com

### Europe, Middle East, Africa and India

Trajan Scientific Europe Ltd  
 Crownhill Business Centre  
 14 Vincent Avenue  
 Crownhill, Milton Keynes  
 MK8 0AB, United Kingdom  
 Tel: +44 (0) 1908 568 844  
 Fax: +44 (0) 1908 566 790  
 europe@trajanscimed.com

### Asia

Trajan Scientific Japan Inc  
 RK CUBE 3F, 6-85 Otamachi  
 Naka-ku, Yokohama-shi  
 Kanagawa 231-0011, Japan  
 Tel: +81 (0) 45 222 2885  
 Fax: +81 (0) 45 222 2887  
 japan@trajanscimed.com

For more information about this product visit [www.hemapen.com](http://www.hemapen.com) or contact [techsupport@trajanscimed.com](mailto:techsupport@trajanscimed.com)