EN Bellylabs Pregnancy Test For Dogs

Early detection rapid dog pregnancy test for domestic use. Developed with veterinarians and breeders.

- 96 % accuracy*
- Test no earlier than 28 days post-ovulation**
- Quick, easy, safe, and reliable test for home use

*The Bellylabs pregnancy test was compared to a confirmed medical diagnosis. The study involved 70 performed tests.

**Gestation days may vary if calculated from mating and the time of ovulation is unknown. A progesterone test can be performed to precisely determine ovulation before mating.

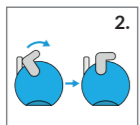
The Bellylabs pregnancy test is intended to determine pregnancy in a bitch as well as to distinguish between pseudopregnancy and a real gestation. The test measures relaxin hormone which is present during gestation. The relaxin hormone is a reliable biomarker for determining dog pregnancy.

Before you start, read the instructions, and watch the "how to" video. QR-code/link www.belly-labs.com


Please make sure that test kit components have reached room temperature.

- Test kit contains:**
- Instruction leaflet
 - 2 safety lancets
 - Sample swab
 - Buffer tube
 - Test cassette (foil packaging)
 - Test package has a cut out to hold the buffer tube

1. Unpack all the test-kit parts when you are ready to perform the test. Place them on a table or a flat surface and have a timer available. To ease the testing process, press the buffer tube in its assigned location on the packaging, and open the top lid.

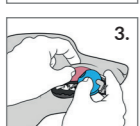
2. Push down the safety tab to activate the lancet.


3. Have a firm hold of your dog. Lift your dog's upper lip and press the lancet device firmly against the dog's inner lip horizontally and press the release button. If you miss or cannot withdraw a sufficient amount of blood, take the spare lancet and repeat this step.

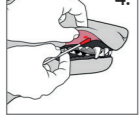
4. When you see a full droplet of blood, take the sample swab and collect the sample on to the swab until it is red throughout and will no longer absorb any more blood.

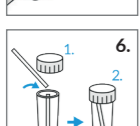
5. Place the sample swab into the liquid and set the timer for 1 minute. Roll the swab in the buffer liquid for 3-5 rounds to help release the sample.

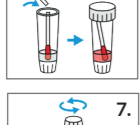6. Break off the swab against the tube's side from the pre-cut break point and close the top lid carefully.

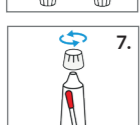
7. After 1 minute, turn the buffer tube upside down and open the smaller lid.


8. Turn the buffer tube upright again, hold it vertically and drop 2 drops of sample into the test cassette's sample hole.

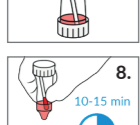9. Leave the test undisturbed on a table or a flat surface and set your timer for 10 minutes.

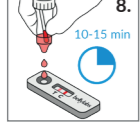10. Read the test result 10-15 minutes after performing the test.

Reading the test:

Positive:

- Both Test (T) and Control lines (C) are visible.

Negative:

- Only Control line (C) is visible.

Test is invalid. Repeat with a new test.

- Only Test line (T) is visible.
- No lines appear.

The test results can be read between 10-15 minutes from the completion of the test procedure. Results read after this time can be unreliable. The strength or intensity of the test line does not indicate the accuracy of the test result. Any forming of a result in the test line area (T) is to be considered as a positive result.

Interpretation of the test:

The test is positive

- The dog is pregnant.

The test is negative

- The dog is not pregnant.
- The dog is experiencing phantom/ pseudopregnancy.
- The test may have been performed too early. Repeat with a new test after one week. It is recommended that the test is performed no earlier than 28 days post-ovulation. The gestation days can vary a few days if they are calculated from the mating day and the time of ovulation is unknown.

Possible reasons for false results

- Something went wrong with the test procedure. Please read the instructions and retest your dog with a new test.
- The relaxin hormone production may still be active for some time after the resorption of all fetuses or complete abortion.
- The false positive result can be caused by certain medical conditions.

It is recommended that a veterinarian is contacted following a positive result. The veterinarian can monitor the pregnancy and determine the number and health of the foetuses via ultrasound or x-ray.

- Make sure to read the instructions before proceeding with the test. For domestic use. For animal use only. For single use only. Keep out of reach of children and pets.
- Temperature: all test materials should be room temperature at the time of testing.
- Storage: store at 2-30 °C / 36-86 °F.
- Do not use if any of the packages are damaged.

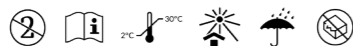
The entire risk due to the performance of this product is assumed by the purchaser. The manufacturer shall not be liable for indirect, special or consequential damages of any kind resulting from the use of this product.

Recycle and dispose of waste according to instructions by your local authorities.

For more information, see FAQ at www.belly-labs.com or contact customer service hello@belly-labs.com

Patent pending
Bellylabs Ltd
Veneentekijäntie 12
FI-00210 Helsinki
Finland
www.belly-labs.com

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