



KITTY KATH FEMALE EXTERNAL CATHETER

INSTRUCTIONS

- “Going” while seated in the race car can be difficult. It can take extra concentration especially if you’re in the driver seat. We recommend testing the device out at home in the shower before testing in the race car.
- For personal fit you can cut/shape the contact area of the Silicone device to fit your body.
- Hair removal on the contact areas is important to get a proper seal and prevent leakages.
- Brush on the Urobond silicone adhesive on the contact area. Cover the entire contact area of the silicone device and put more glue on the back.
- Carefully place the device with the tube facing forward or up.
- After use, clean the device using warm soap and water.
- Use adhesive remover wipes to remove the UroBond from skin and the device.
- Clean the device using warm soap and water.
- We have included three pieces of silicone tubing. We don’t recommend re-using the silicon tubing, additional tubing can be purchased for \$4.95.
- Use only provided adhesive and removers. If the device is damaged by using other chemicals, the warranty will be void.