


# Creating a write cell

A write cell is used to send text to the writing area. If you have chosen 'speak-as-you-type' settings then your text will also be spoken.

Enter Edit mode by clicking the top menu (or use **F11**) and clicking the **Edit** button. Click on a blank cell, and then click **Create Cell**.




Click **Write**, then **Next**.

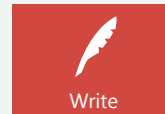

Type your text into the top box and select a picture.

You can click **Find Picture** for more image options.

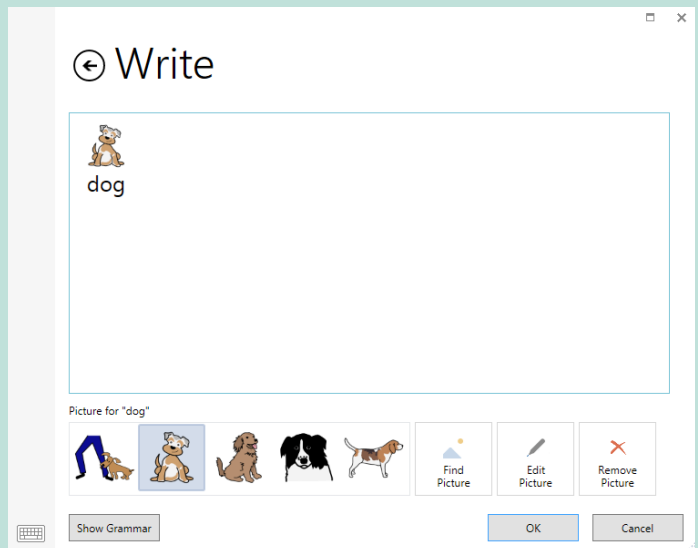

Click **OK**, then **Finish Editing** in the menu bar.

## ★ Tip

You can highlight multiple words to select one symbol for all of them.

# Editing a write cell

To change the text which is sent to the writing area, select the cell and click **Edit** in the Commands box. Changing the text won't change the label.


## Commands

 Write ✕





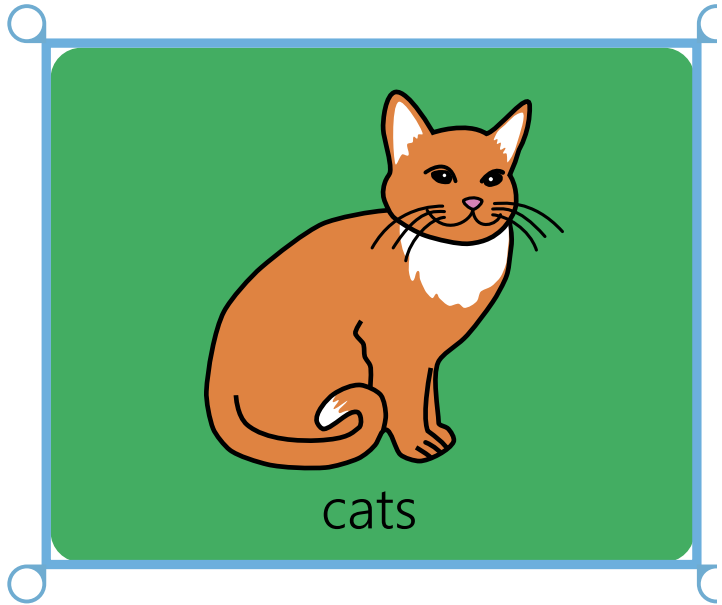

Write some text into the writing area.

   
I like cats

 Edit



To change the cell label, click **Change Label** under the Home menu. Make your changes and press **Enter**.

GRID SET		HOME		STYLE		LAYOUT	
							
Finish Editing	Undo	Copy	Paste	Delete	Change Label		