




We bring speech to life™

Cleaning and Disinfecting Devices and Accessories

You'll need:

- Hospital-grade germicidal (appropriate for the material)

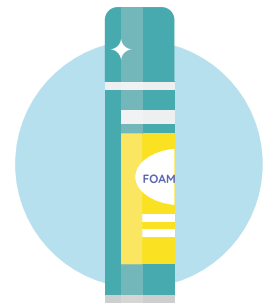

Speech-Generating Devices & Related Accessories Cleaning



Clean all surfaces of Speech-Generating Device, charger, and cords and all parts/surfaces of accessories by wiping with disinfectant wipe.




Allow adequate time to dry, as specified in the instructions of your cleaner. Repeat wiping/drying process with new wipe.



Let air dry.

Carrying Case Cleaning

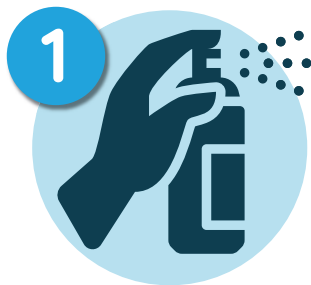

Clean all interior and exterior surfaces of carry case by spraying with disinfectant.



Carrying cases with a hard exterior may be cleaned with disinfecting wipes.



Allow adequate time to dry, as specified in the instructions of your cleaner.



Let air dry.

Customer Care

877-392-2299 Option 3
support@talktometechnologies.com