



We bring speech to life™

GoTalk NOW Plus Quick Reference Guide

for wego™ A devices with GoTalk NOW Plus



Edit a Button Message or Image

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Select a button to edit.
3. To edit a button label: Select **Button Text** on the bottom of the screen > select **Text** > type message > **Done**. Tap anywhere on the screen to minimize Button Text menu when finished. Text can be held and dragged around the screen to move to desired location on the button.
4. To add an image: Select **Add Image** > choose the preferred image type > **type search word if needed** > **Select Image**. To remove an image, hold down on the icon and select **Delete Image**.
*Note: More than one image can be added, and images can be formatted by moving around screen or pinching or expanding them to change size.
5. Select **Done** in lower right corner > **Home** in lower left corner to save changes.
6. Select **Player** mode.

Add a Button Action > Navigate

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Select button to be made into a folder.
3. Select **Action** at the top.
4. Select **Jump to Page**.
5. Select **Change** > select page name to jump to. Tap anywhere on screen to minimize menu.
6. Select **Done** in lower right corner > **Home** in lower left corner to save changes.
7. Select **Player** mode.

Create a Board from Template

*Note: If you have already created a template, start at Step 4.

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Select  icon on lower right. Select the **+** to add a page to the library.
3. Select **My Templates** > **Create New Template** > name the new template > choose **Set** > choose a page > **Done** > **Create**.
4. Tap **Pages**, then **+**. Tap **My Templates** > select the new template > **Add to Communication Book**.
5. Select **Page Name** (e.g. Page 2) at the bottom of the screen > type new page name > **Done**.
6. Select **Pages** icon on lower right > select the page with the button to be made into a folder (e.g. Page 1).
7. Follow steps 2-7 for Add a Button Action > Navigate.

Copy and Paste Buttons

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Press and hold on button to be copied.
3. Select **Copy**.
4. If needed, navigate to the page where button will be placed, using the **Pages** icon.
5. Press and hold on button location for copied button to be placed.
6. Select **Paste** > select **Home** in lower left corner to save changes > select **Player** mode.

Change the Pronunciation of a Word

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Select button to edit pronunciation for.
3. Select **Action** at the top > select **Text-to-Speech**.
5. Type in the word by spelling it how it sounds > select **Preview** to test the pronunciation.
6. Tap anywhere on screen to minimize menu.
7. Select **Done** in lower right corner > **Home** in lower left corner to save changes > select **Player** mode.

Hiding Buttons

1. Enter edit mode by swiping down from the top of the screen with three fingers OR holding  for 8 seconds, then select **Page Editor**.
2. Press and hold on button to be hidden > select **Disable**.
3. Tap anywhere on screen to minimize menu.
4. Select **Done** in lower right corner > **Home** in lower left corner to save changes > Select **Player** mode.

Guided Access

All dedicated devices come with *Guided Access* enabled.

1. Open *Guided Access* menu by quickly triple-clicking the device Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111)**.
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. Lock Menu (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume**.

Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

GoTalk NOW Plus is copyright Attainment Company Inc.

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



talktometechnologies.com

Updated October 2021



We bring speech to life™

Hardware, Buttons & Features



1 Power

Press and hold this button (plus a volume button on some models) to turn the device on or off.

Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2 Volume

Use these buttons to increase or decrease volume level.


3 Home

Tap to return to the home screen. If your wego A model does not have a Home button, swipe up from the bottom of the screen.

4 Charging the wego A

Charge device and TTMT Speaker (SPKR) at night or when not in use. Please only use a TTMT-approved charger with your device.

wego 7A-D, 10A-D & 13A-D

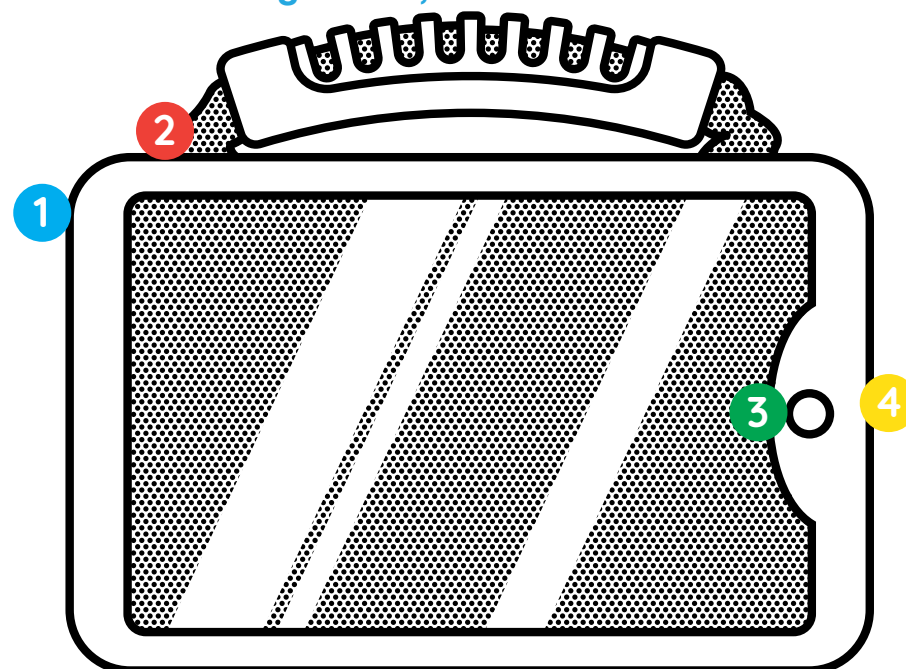

If your wego A model does not have a Home button, swipe up from the bottom of the screen.

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com