



We bring speech to life™

# QuickStep CVI/VI Quick Reference




for zuvo and eyespeak devices with QuickStep CVI/VI



## Change Symbol Color

The default symbol colors of your QuickStep CVI/VI vocabulary set can be easily personalized in any color you choose.

1. Enter *Edit Mode* by selecting (Menu) > **Edit Grid**.
2. Select the button with the image you want to change.
3. On the *Home* tab, select **Edit Picture**.
4. Choose the **Color** icon, then **Adjust Color**.
5. Use the color wizard to choose a new color. Make sure *Saturation* is set to the highest level. *Luminosity* should also be set to a relatively high level. When finished, select **OK**.
6. Use the **Fill** tool to change the color of the symbol.
7. Select **OK** > **Finish Editing** > **Yes** to save changes and exit *Edit Mode*.

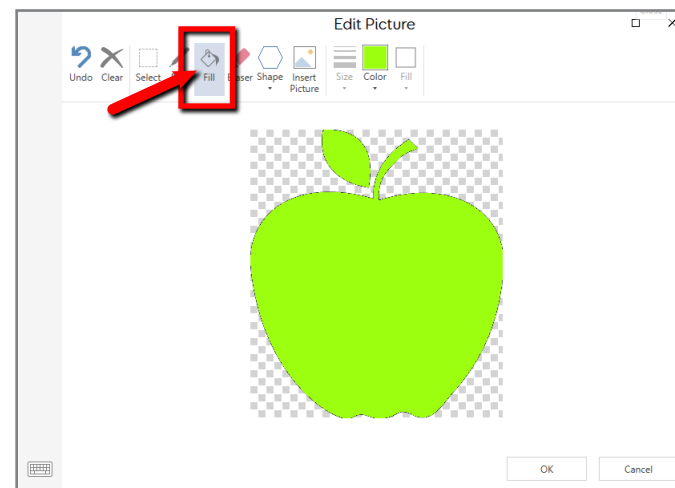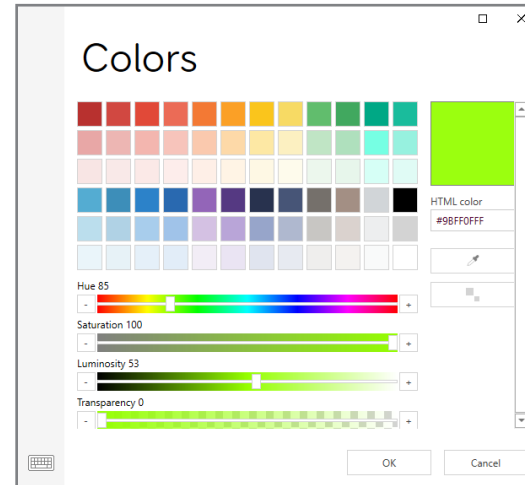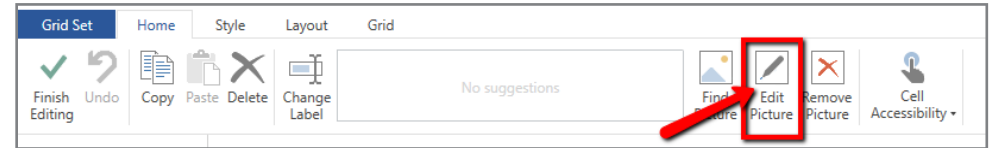

*Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

Need help?

877.392.2299 Option 3

[support@talktometechnologies.com](mailto:support@talktometechnologies.com)

[talktometechnologies.com](http://talktometechnologies.com)