



We bring speech to life™

wego™ A Quick Reference


TTMT Switch Interface - GoTalk NOW PLUS




Setup Instructions

Turn on the TTMT Switch Interface, using the on/off switch located on the side of the interface box. You will see a red power light to indicate the interface is turned on.

Ensure the Switch Interface is connected to the wego via Bluetooth.

1. Exit the communication software. If you are unable to exit the software, you may be in Guided Access.
2. Open the *iOS Settings*  then select **Bluetooth**. Ensure Bluetooth is toggled On. Under *My Devices*, you should see TTMT Switch listed as **Connected**. If not, tap on “TTMT Switch” to connect. You may see a pop-up Bluetooth Pairing Request, if so, press **Pair**.

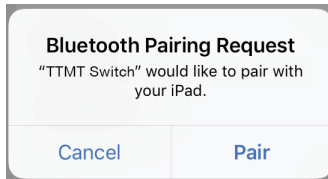

3. Once the Switch Interface is connected via Bluetooth, you can plug in your switches and begin setup. **Connect your Switch or Switches to the TTMT Switch Interface.**

The Switch Interface can be used with one or two switches. For single-switch scanning, plug your switch to Switch Port 1. For two-step scanning, plug in both switches, with your “chooser” in Port 1 and your “mover” switch in Port 2.

Setup with GoTalk NOW PLUS

1. Return to the communication app, **GoTalk NOW PLUS**. Open **Settings** and **Scroll to Scanning**.
2. Select **Scanning Device Type > Space** and **Enter Switch**.
3. Determine Scanning Mode.
 - a. For single-switch scanning, toggle on Auto Scan.
 - b. For two-switch scanning, toggle off Auto Scan.
4. Select Grouping Style (row, column, etc), Speed, Repeat, etc. as preferred.
5. Press **Done** to close Settings and return to Player Mode.

Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com



We bring speech to life™

Hardware, Buttons & Features



TTMT Switch Interface

1 Power

Flip switch into up position to turn on the Bluetooth device and flip switch down to turn it off.

2 Power light indicator

When this light is on and red, the Bluetooth device is on.

3 Charging the TTMT Switch Interface


Plug charger into port to charge the switch interface. Please only use a TTMT-approved charger with your device.

4 Switch Port 1

Plug switch cords into the switch ports.

5 Switch Port 2

Plug switch cords into the switch ports.

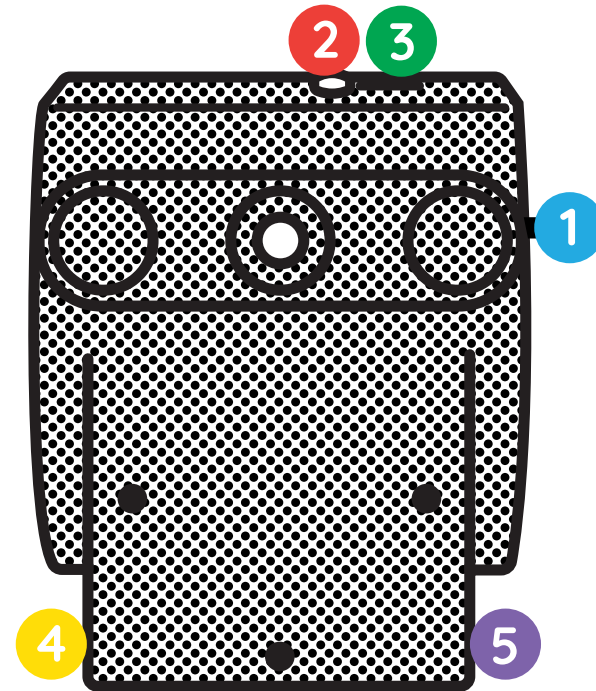

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com