



We bring speech to life™

wego™ A Quick Reference LAMP

Edit a Button Message and Image

1. **Menu > Edit Page > Select button to edit > Edit This Button.**
2. *Button Label:* In text box below *Button Label*, type text to display on button > **Done** (keyboard).
3. *Button Message:* In text below *Button Message*, type message button should speak when pressed > **Done** (keyboard).
4. Select, import, or take new button image:
Choose a symbol: **Select Image > tap magnifying glass Q > type word to search > Search > Select symbol > Save > Done.**
Import photo: **Import Image > Image Library > Select picture from photo library > Use > Save > Done.**
Take a picture: **Import Image > Camera > Press white circle on screen to take a picture > Use photo > Save > Done.**

Add a Button Action > NAVIGATE

1. **Menu > Edit Page > Select button to edit > Edit This Button.**
2. Scroll down to *Button Actions* > **Add > Navigate.**
3. Select page to navigate to > Select No Animation or animation type > **Save > Done.**

Create a New Page from a Template

1. **Menu > Edit Page.**
2. **Menu > Add New Page > under Page Name, type new page name > Save > Done.**
3. Connect the new page to a button by following *Add a Button Action > NAVIGATE* steps 1–3.

Copy and Paste Buttons

1. **Menu > Edit Page > Select button to copy > Copy Button.**
2. Tap location to paste button > **Paste Button.**
3. To copy button style only: Follow above instructions but select **Paste Button Style** on Step 2.

Change the Pronunciation of a Word

1. **Menu > Edit Pronunciations.**
2. Tap the **+** sign.
3. In the *Word* box, type the word you would like to edit (e.g. fajita).
4. In the *Pronounce As* box, type the word's proper pronunciation (e.g. Fuhheeta).
5. Select **Pronounce** to hear pronunciation.
6. **Save > Done.**

Word Finder

1. **Menu > Word Finder > Type word to search > Find.**
2. The path to find the word will display.
3. Select the path and it will guide you through the motor plan to selected button.

Guided Access

All dedicated devices come with Guided Access enabled.

1. To turn off Guided Access: open Guided Access by quickly triple-clicking the Home button on the front of your device (*note: if your wego A model does not have a home button, use the Power button as shown on reverse*).
2. Enter passcode: default password is all 1s (1111 or 111111).
3. Select the path and it will guide you through the motor plan to selected button.
4. To end Guided Access: press **End** (top left).
5. **Lock Menu** (restrict use of the menu bar or specific buttons/pages):
Open Guided Access > Use finger to draw a circle around area/buttons to restrict.

Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



www.talktometechnologies.com



We bring speech to life™

Hardware Buttons and Features

1 Power

Press and hold this button to turn the device on or off.
Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2 Volume


Press the left of volume toggle to increase speaker volume or press the right of the toggle to decrease volume.

3 Home


Tap to return to the home screen. If your wego A model does not have a Home button, use 1 Power.

4 Charging the wego A

Charge device and speaker at night or when not in use.

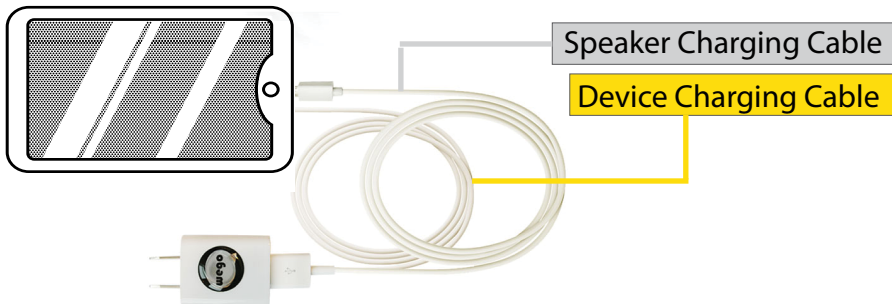

wego 5A

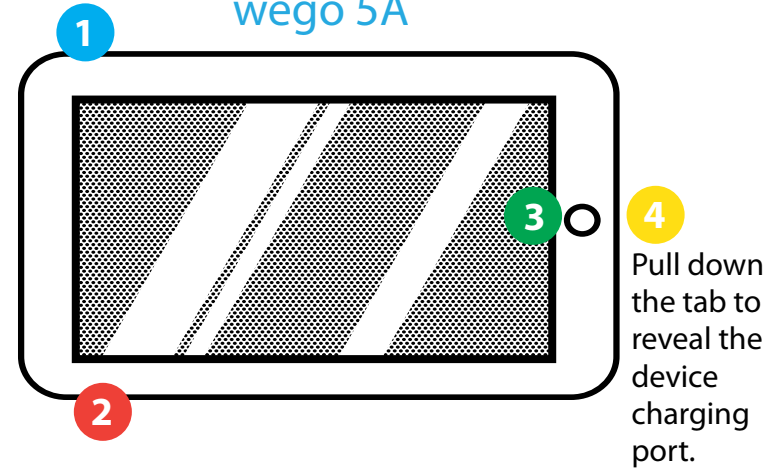

wego 7A, 10A & 13A

