




We bring speech to life™

wego™ A Quick Reference

LAMP



Edit a Button Message and Image

1. **Menu > Edit Page > select button to edit > Edit This Button.**
2. *Button Label:* In text box below *Button Label*, type text to display on button > **Done** (keyboard).
3. *Button Message:* In text below *Button Message*, type message button should speak when pressed > **Done** (keyboard).
4. *Select, import, or take new button image:*
Choose a symbol: **Select Image > tap magnifying glass**  > type word to search > **Search > select symbol > Save > Done.**
Import photo: **Import Image > Image Library > select picture from photo library > Use > Save > Done.**
Take a picture: **Import Image > Camera > press white circle on screen to take a picture > Use Photo > Save > Done.**

Add a Button Action > NAVIGATE

1. **Menu > Edit Page > select button to edit > Edit This Button.**
2. Scroll down to *Button Actions* > **Add > Navigate.**
3. Select page to navigate to > select **No Animation** or animation type > **Save > Done.**

Create a New Page from a Template

1. **Menu > Edit Page.**
2. **Menu > Add New Page > under Page Name**, type new page name > **Save > Done.**
3. Connect the new page to a button by following *Add a Button Action > NAVIGATE* Steps 1–3.

Copy and Paste Buttons

1. **Menu > Edit Page > select button to copy > Copy Button.**
2. Tap location to paste button > **Paste Button.**
3. To copy button style only: Follow above instructions but select **Paste Button Style** on Step 2.

Change the Pronunciation of a Word

1. **Menu > Edit Pronunciations.**
2. Tap the **+** sign.
3. In the *Word* box, type the word you would like to edit (e.g. fajita).
4. In the *Pronounce As* box, type the word's proper pronunciation (e.g. fuhheeta).
5. Select **Pronounce** to hear pronunciation.
6. Save > **Done.**

Word Finder

1. **Menu > Word Finder > Type word to search > Find.**
2. The path to find the word will display.
3. Select the path and it will guide you through the motor plan to selected button.

Guided Access

All dedicated devices come with *Guided Access* enabled.

1. Open *Guided Access* menu by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111).**
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. *Lock Menu* (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume.**

Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com



We bring speech to life™

Hardware, Buttons & Features



1 Power

Press and hold this button (plus a volume button on some models) to turn the device on or off.

Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2 Volume

Use these buttons to increase or decrease volume level.


3 Home

Tap to return to the home screen. If your wego A model does not have a Home button, swipe up from the bottom of the screen.

4 Charging the wego A

Charge device and TTMT Speaker (SPKR) at night or when not in use. Please only use a TTMT-approved charger with your device.

wego 7A-D, 10A-D & 13A-D

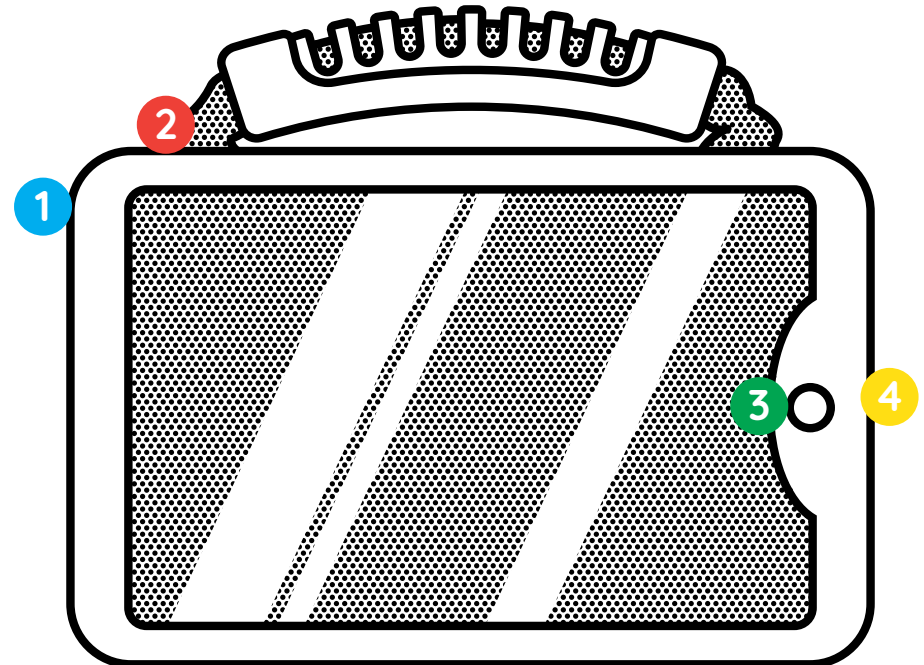

If your wego A model does not have a Home button, swipe up from the bottom of the screen.

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com