




We bring speech to life™

wego™ A Quick Reference Predictable



Quick keys and customization buttons

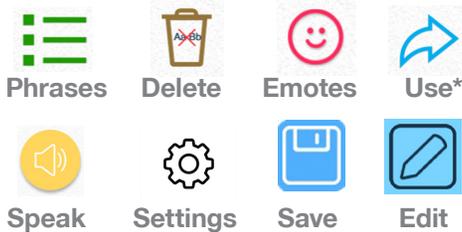

Add a phrase category

1. Select **Phrases** > select **Edit** at bottom of page > swipe left to the last category page > tap **Add category**.
2. Type name of new phrase category at the blue cursor.
3. Select **No Image**, or select one of the following to add an image:
Take from camera > take photo using camera > **Use Photo**.
Choose from gallery > tap to select image from gallery.
Choose from Predictable image library > tap to select symbol.
4. When finished, tap **Save** at the top right of screen.

Save a phrase

1. **Phrases** > select desired category > select **Add a new phrase...** at the bottom of the list of saved phrases.
2. Select **Text** from left navigation menu.
3. On *Text to speak* line: type message to speak when phrase is selected.
4. On *Text to display* line: type text (label) to display in phrase bank.
5. When finished, tap **Save** at the top right of screen.

Quick-save a phrase from the homepage

1. Type phrase using keyboard on homepage > select **Use** > select **Add** from pop-up menu.
2. Select category to save new phrase > tap **Save** at the top right of screen.

Edit a saved phrase

1. **Phrases** > select desired category > select **Edit** at bottom of page > tap phrase to edit > choose **Edit** from pop-up menu.
2. Follow steps 2-5 from **Save a phrase** to finish editing.

Change pronunciation of a word

1. Select **Settings** > select **Speech** from left menu > select **Speech Settings** at the top right of the page > **Custom Words** > select **Custom Pronunciation**.
2. On *Display* text line: type word with correct spelling (i.e. fajita).
3. On *Pronunciation* text line: type word as you want pronounced (i.e. faheeta).
4. Press **Play** next to pronunciation to hear how word will sound.
5. When finished, tap **Save** at the top right of screen.

Guided Access

All dedicated devices come with *Guided Access* enabled.

1. Open *Guided Access* by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111)**.
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. *Lock Menu* (restrict use of specific buttons/categories): open *Guided Access* > use finger to draw a circle around buttons to restrict > **Resume**.

Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

**Some Use key functions are enabled for -PLUS models only. Contact us for details.*

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com



We bring speech to life™

Hardware, Buttons & Features



1 Power

Press and hold this button (plus a volume button on some models) to turn the device on or off.

Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2 Volume



Use the toggle button to increase or decrease volume level.

3 Home

Tap to return to the home screen. If your wego A model does not have a Home button, swipe up from the bottom of the screen.

4 Charging the wego A

Charge device and speaker at night or when not in use. Please only use a TTMT-approved charger with your device.

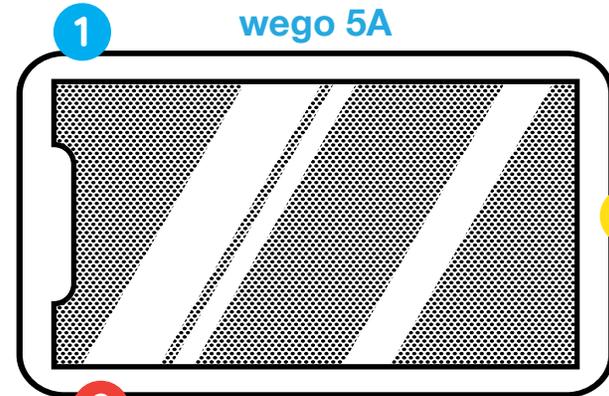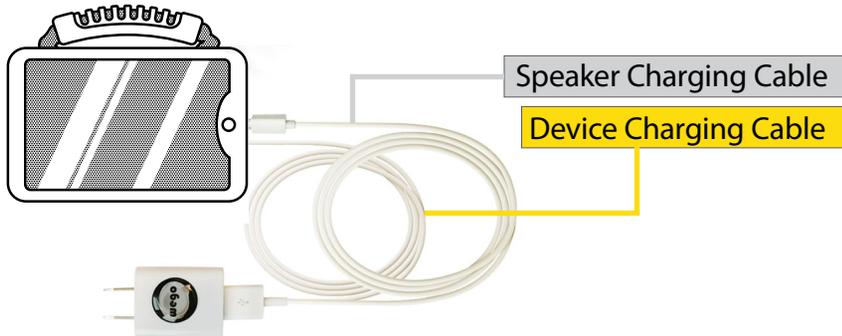


wego 5A

4

Pull down the tab to reveal the device charging port.

wego 7A, 10A & 13A

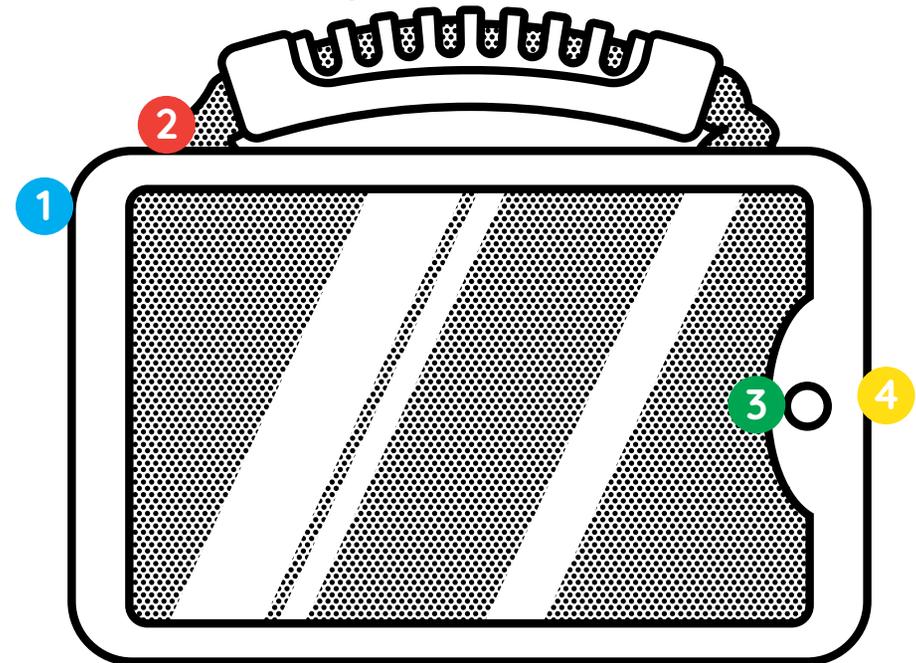

If your wego A model does not have a Home button, swipe up from the bottom of the screen.

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com