




We bring speech to life™

wegoTM A

Proloquo2Go® User Guide

For **wego A-D** Speech-Generating Devices

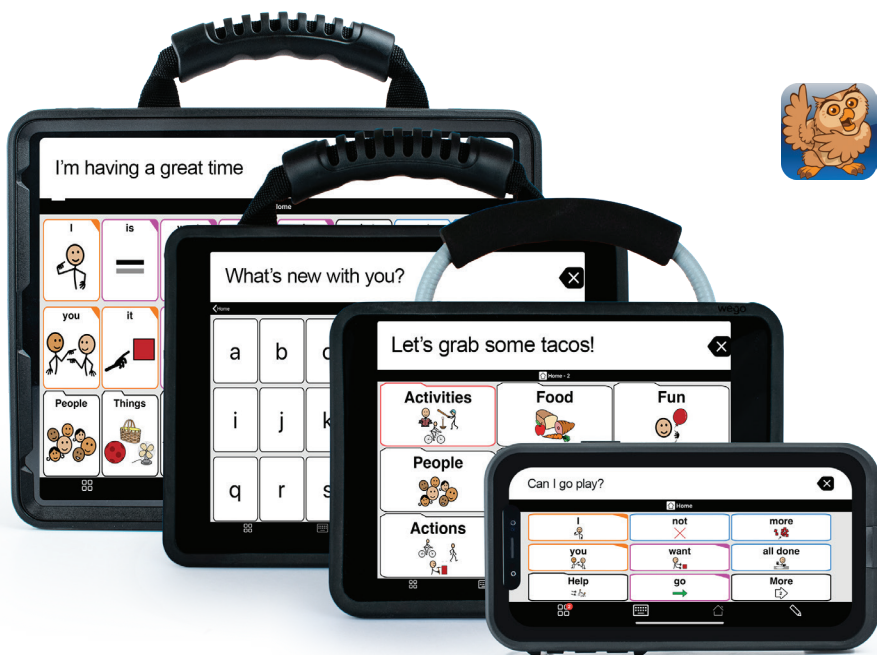

Find more resources online:

talktometechnologies.com (click **Support**)

Table of contents

Hardware, buttons and features	2
Editing mode	4
Workspace overview	5
Options	6
Edit an existing button	7
Pronunciation	8
Create a new regular button	9
Create and link a folder button	10
Typing view	11
Pair the TTMT Speaker (SPKR)	12
User information, passwords and notes	13

Hardware and features

1

Power

Press and hold this button (plus a volume button on some models) to turn the device on or off.

Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2

Volume

Press the volume toggle up or down to adjust volume levels.

3



Home

Tap to return to the home screen. If your wego A model does not have a Home button, swipe up from the bottom of the screen.

4

Charging the device

Pull down the tab to reveal the device charging port. Charge device and TTMT Speaker (SPKR) at night or when not in use. Please only use a TTMT-approved charger with your device.

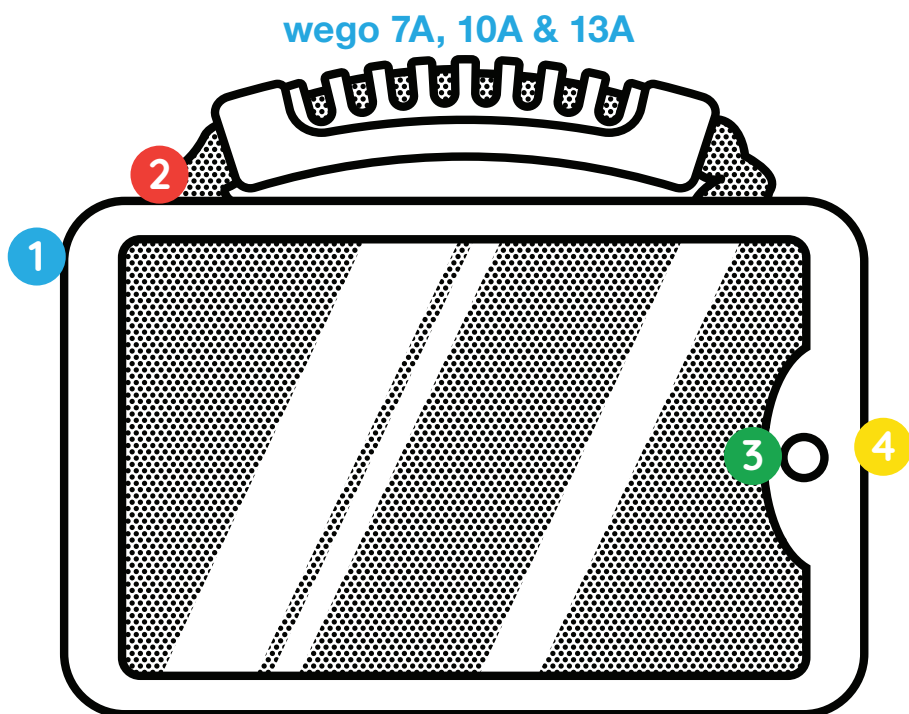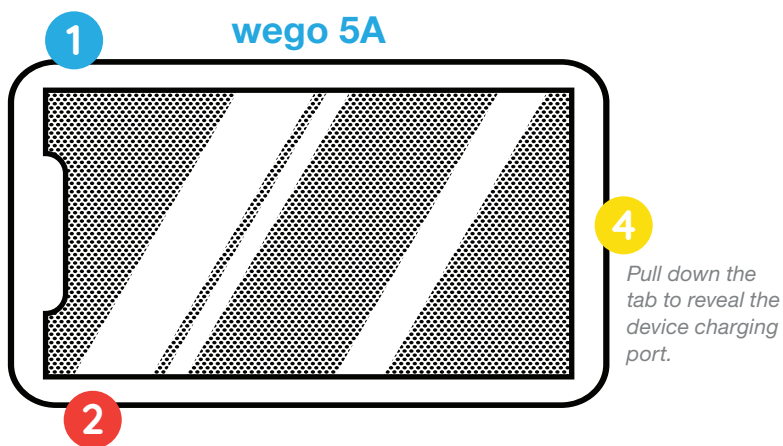

If your wego A model does not have a Home button, swipe up from the bottom of the screen.

Editing mode

Tap pencil icon to enter editing mode. The editing toolbar will appear.




Editing toolbar quick reference




- Select all
- Deselect all
- Copy
- Paste
- Delete
- Move to primary level
- Move to secondary level
- Move to storage
- Sort
- Swap
- Export
- Tool tips

Workspace overview

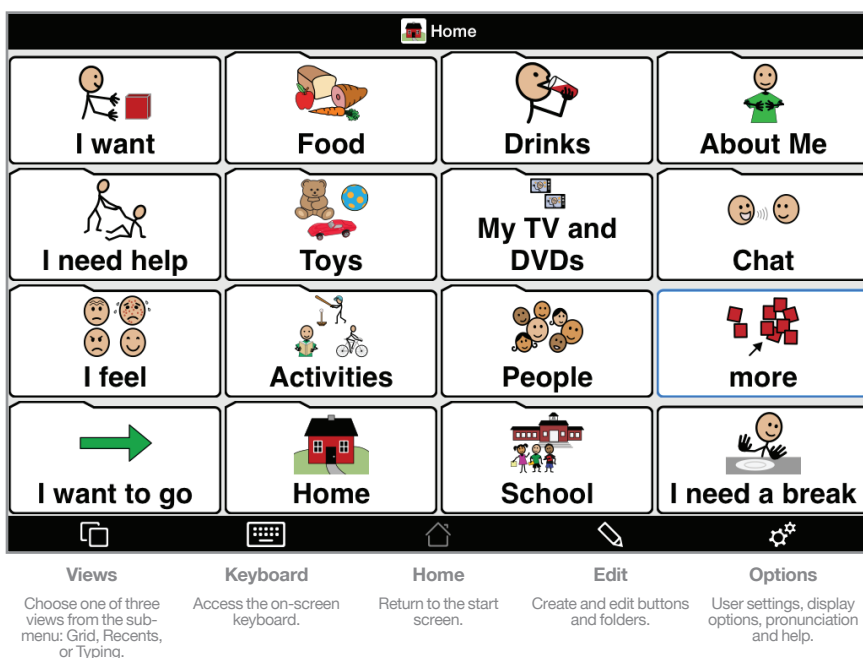

There are three button types available in Proloquo2Go®

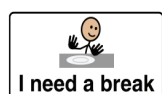

Regular Button

The most common type of button you will create in order to add symbols and messages to the speech window to be spoken aloud when a button is pressed.



Folder Buttons

Link to an additional folder with more button choices. For example, you might choose to link a “School” button to a pre-made folder of “School Supplies”, or create a new folder and create a custom list of “School Supplies” buttons.



Action Buttons

Perform a specific action when pressed. These are helpful for creating quick messages, and for individuals who do not have the ability to perform particular software functions, like clearing the message window.

Options


User Settings & Features

Within your **wego A**, there are several settings that are worth noting. Access to some of these features may be integrated into your particular vocabulary set.


Help menu
Extensive list of resources included on your device.



Options menu
Calibrate access method, speech settings and touch sensitivity.

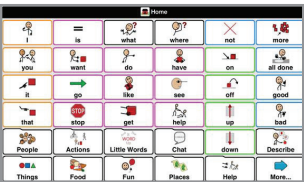

Sample with 6 columns

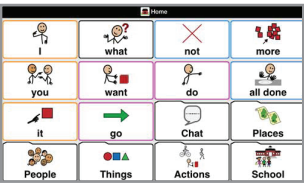



Sample with 4 columns



Appearance sub-menu
Within the options menu, you can adjust additional settings: display style, label position and color, number of columns.

Edit an existing button

In **Edit Mode**, first tap the button you wish to edit, then tap the arrow under the toolbar to display the button properties.

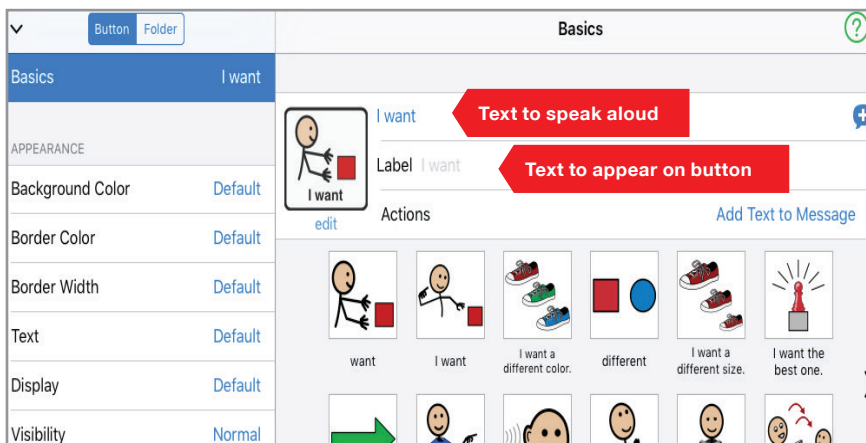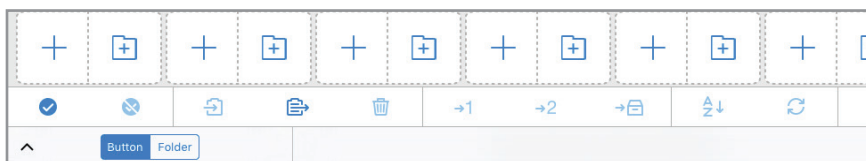

Button Image

Choose from a selection of images, or take a photo with the device camera.

Text to Speak

The message to be spoken aloud when the button is pressed.

Expressive Power™

Apply realistic prerecorded sounds and phrases to buttons. To explore, tap the speech bubble icon with plus symbol.

Label

The visible text label to appear on the button.

Additional Button Properties

Explore the list of additional customizing options for individual buttons, including: Border Color, Background Color, Voice Options, etc.

Pronunciation

Pronunciation can be edited within the options window by tapping the gear icon > **Speech and Language**.




Example: “french fries” vs. “french fryz”

1. Tap the  icon beside the voice.




2. Tap the language you wish to edit.
3. Tap **Edit Pronunciations**.
4. Tap the word from the list you would like to edit.

Note: New pronunciations can be added to the list by tapping the + symbol.

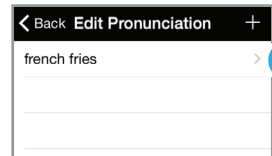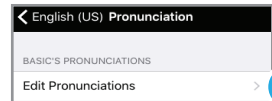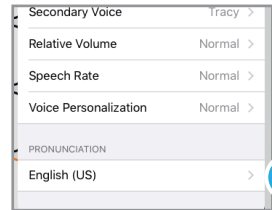


5. Tap **Speak** to test your new pronunciation.

Note: New and edited pronunciations are automatically saved when you return to the word list.

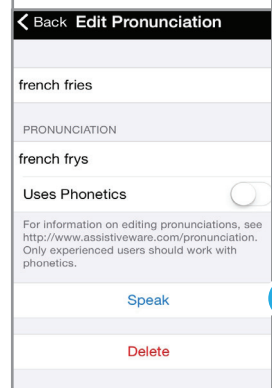


Create a new regular button

New buttons and folders can easily be added in edit mode. Click the pencil icon to enter edit mode.

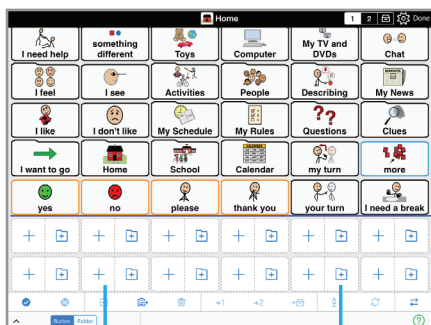


Add a button.

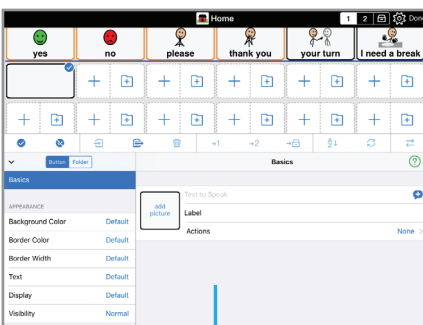

Add a folder.

New button properties window.

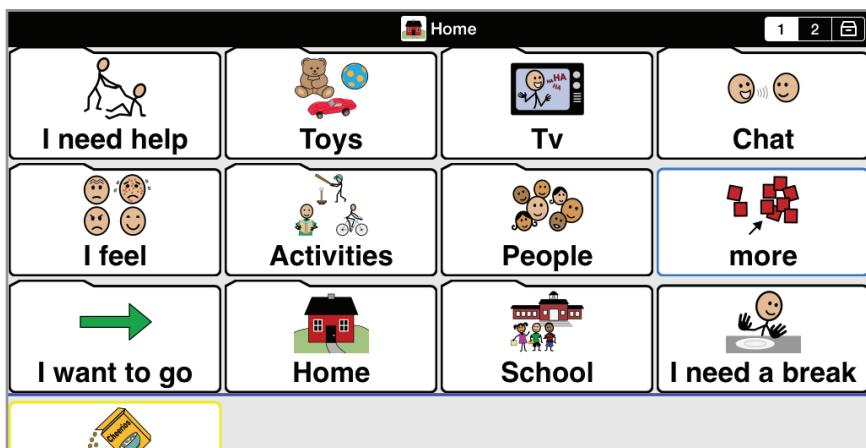


Button placement

Buttons above the blue line will be visible on the screen without scrolling.

Move a button

Tap any button and hold until it begins to pulse. Drag and release it on the page where you would like it to appear.

Create and link a new folder button

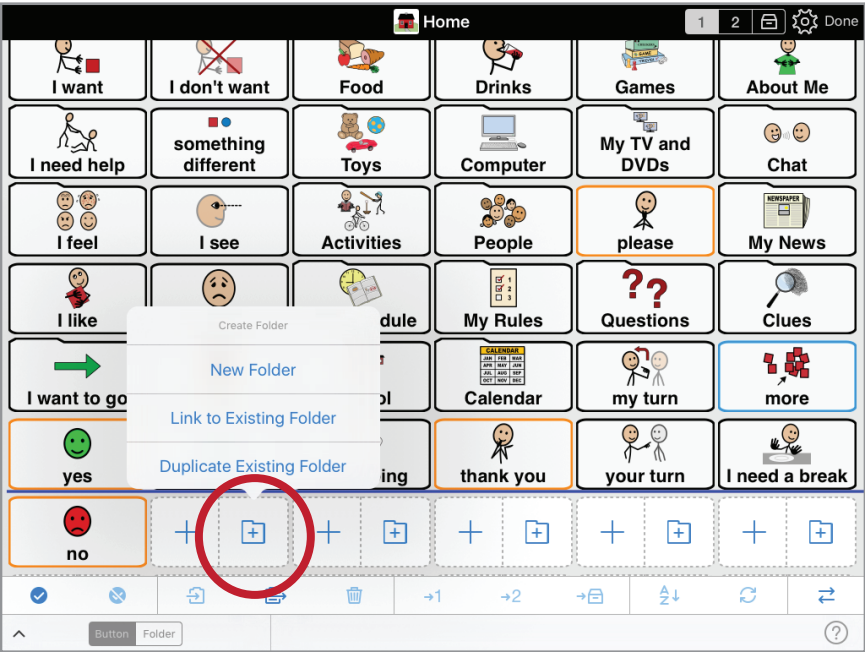

Link to an existing folder of buttons

Add Folder > Link to Existing Folder > choose from the list of folders of buttons displayed.


Link to a new folder

Add Folder > New Folder


Add buttons to a folder

Double tap any folder to view the folder contents. New buttons can be added or edited.

Typing view

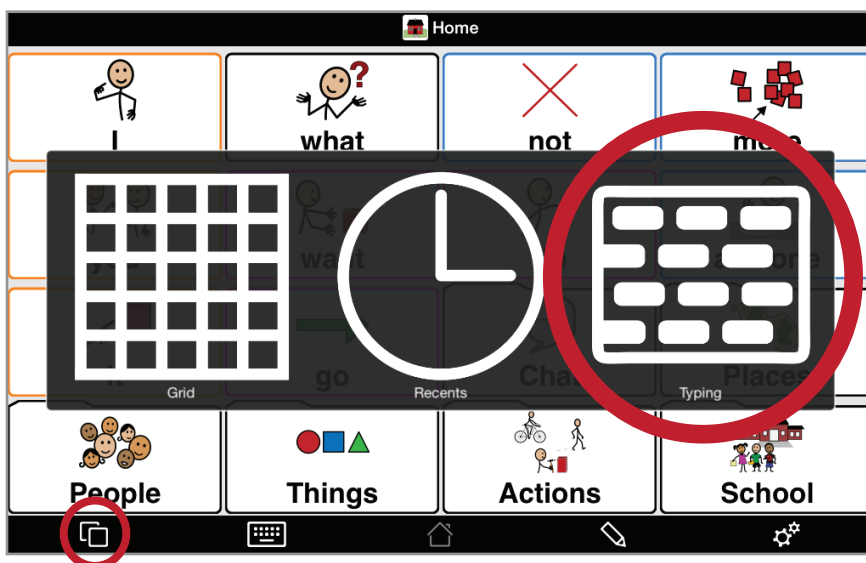

Views > Typing. Type your message using the on screen keyboard. When you are ready to speak your message, tap **Speak**.

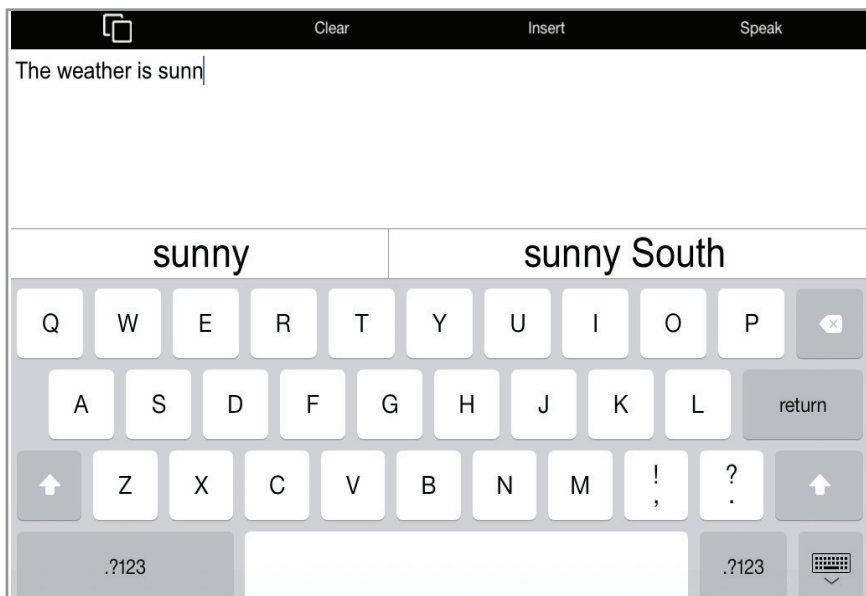

Tip: messages will automatically be saved! Tap the recents icon to see a list of most recent phrases.

Pair the TTMT Speaker (SPKR)

Your wego A's TTMT Speaker (SPKR) is already paired, but on occasion you may need to re-pair it.


- 1. Turn off *Guided Access*: quickly triple-click Home button to open *Guided Access*.

*Note: if your wego A model does not have a home button, use the Power button as shown on page 2. Enter pass code (default pass code is all 1s) > press **End** (top left).*


- 2. Press **Home** button to view device home screen.

Note: if your wego A model does not have a home button, use the Power button as shown on page 2.

- 3. **Settings** > **Bluetooth** > If off, tap the **Bluetooth** button to turn on.

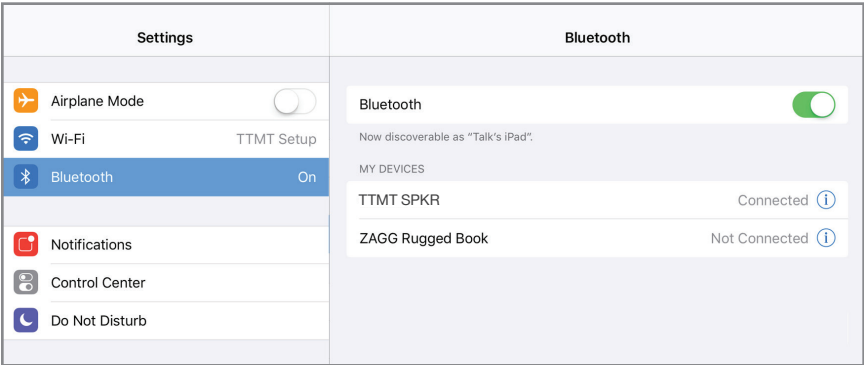

- 4. Under *My Devices*, tap **TTMT SPKR** to pair.

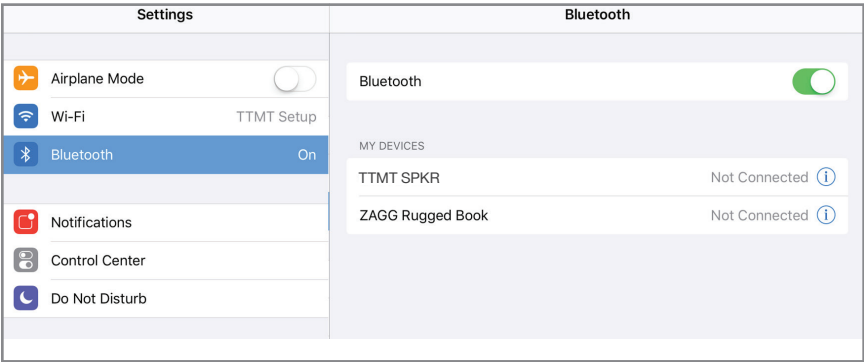

- 5. Press **Home** button to return to device home screen.

Note: if your wego A model does not have a home button, use the Power button as shown on page 2.

- 6. Open vocabulary set.
- 7. Quickly triple-click the **Home** button to resume *Guided Access*.


User information and notes

Apple ID: _____

Password: _____

Guided Access code: _____

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Notes: _____









We bring speech to life™

Need Help?



877.392.2299 Option 3



support@talktometechnologies.com



talktometechnologies.com (click **Support**)



Join in the conversation...

Connect with us for resources, contests and more!