




We bring speech to life™

wego™ A Quick Reference Software Update

Installing updates regularly will help ensure that your vocabulary/page set software runs properly. The following instructions will guide you through the software update process.


Initial Setup: Turn On Access to the App Store

1. Turn off Guided Access: quickly triple-click **Home** button to open Guided Access > enter passcode (default passcode is all 1s) > press **End** (top left).
2. Press **Home** button to view device home page.
3. Connect to Wi-Fi if not already: **Settings** > **Wi-Fi** > connect to your network.
4. **Settings** > **Screen Time** > **Content & Privacy Restrictions** > enter passcode: **9357**.
5. At the top of the screen, toggle **Off** Content & Privacy Restrictions.
6. Press **Home** button and App Store icon will show on device home screen.

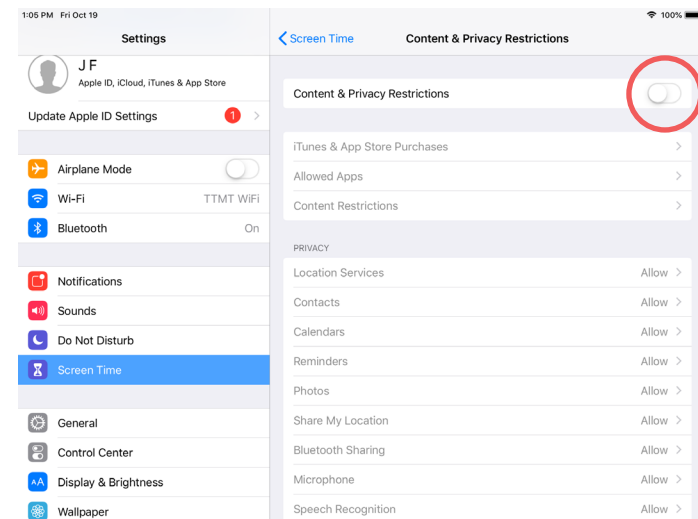

Update Software in the App Store

1. Select the **App Store** icon.
2. Select **Updates** on the bottom menu bar to display applications with available updates.
3. Select **Update** button next to application.
4. If prompted, enter your Apple user ID and password. This is printed in your *User Guide*. If you cannot locate your user ID or password, please contact our Support Department at 877-392-2299 option 3, or support@talktometechnologies.com.

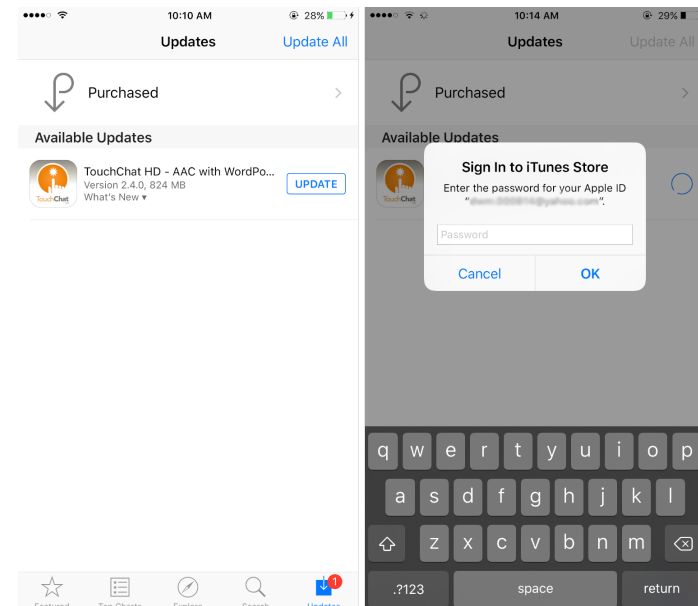

Update Complete: Turn Off Access to the App Store

1. After update is complete, press **Home** button to return to home page.
2. **Settings** > **Screen Time** > **Content & Privacy Restrictions** > enter passcode: **9357**.
3. At the top of the screen, toggle **On** Content & Privacy Restrictions.
4. Press **Home** button to return to home screen.
5. Open vocabulary set.
6. Quickly triple-click **Home** button to return to Guided Access.

Tip: Occasionally a pop-up notification may prompt you to update the IOS software. When time allows, follow the on-screen instructions to update the operating system.

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



www.talktometechnologies.com