




We bring speech to life™

wego™ A Quick Reference

TouchChat

Edit a Button Message and Image

1. **Menu > Edit Page > Select button to edit > Edit This Button.**
2. **Button Label:** In text box below *Button Label*, type text to display on button > **Done** (keyboard).
3. **Button Message:** In text below *Button Message*, type message button should speak when pressed > **Done** (keyboard).
4. Select, import, or take a new button image:
Choose a Symbol: **Select Image > tap magnifying glass  > type word to search > Search > Select symbol > Save > Done.**
Import photo: **Import Image > Image Library > Select picture from photo library > Use > Save > Done.**
Take a picture: **Import Image > Camera > Press white circle on screen to take a picture > Use photo > Save > Done.**

Add a Button Action > NAVIGATE

1. **Menu > Edit Page > Tap button to edit > Edit This Button.**
2. Scroll down to *Button Actions* > **Add > Navigate.**
3. Select page to navigate to > Select No Animation or animation type > **Save > Done.**

Create a New Page from a Template

1. **Menu > Edit Page.**
2. **Menu > Load Another Page > select .template > Done.**
3. **Menu > Import/Copy Pages > Duplicate this Page.**
4. **Menu > Load Another Page > select .template (Copy) > Done.**
5. **Menu > Edit Page Layout > Under Page Name, type new page name > Save > Done.**
6. Connect the new page to a button by following *Add a Button Action > NAVIGATE* steps 1–3.

Copy and Paste Buttons

1. **Menu > Edit Page > Select button to copy > Copy Button.**
2. Tap location to paste button > **Paste Button.**
3. To copy button style only: Follow above instructions but select **Paste Button Style** on Step 2.

Change the Pronunciation of a Word

1. **Menu > Edit Pronunciations.**
2. Tap the **+** sign.
3. In the *Word* text box, type word you would like to edit (e.g. fajita).
4. In the *Pronounce As* text box, type the word's proper pronunciation (e.g. Fuhheeta).
5. Select **Pronounce** to hear pronunciation.
6. **Save > Done.**

Hide Mode

1. **Menu > Edit Page > Menu > Hide Mode.** The task bar will turn green.
2. Select buttons to hide > **Done.**
3. To unhide buttons: **Menu > Edit Page > Menu > Hide Mode > Select buttons to unhide > Done.**

Guided Access

All dedicated devices come with Guided Access enabled.

1. To turn off Guided Access: open Guided Access by quickly triple-clicking the Home button on the front of your device (*note: if your wego A model does not have a home button, use the Power button as shown on reverse*).
2. Enter passcode: default password is all 1s (1111 or 111111).
3. Select the path and it will guide you through the motor plan to selected button.
4. To end Guided Access: press **End** (top left).
5. **Lock Menu** (restrict use of the menu bar or specific buttons/pages):
Open Guided Access > Use finger to draw a circle around area/buttons to restrict.

Important Note: Vocabulary sets are updated frequently. If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



www.talktometechnologies.com



We bring speech to life™

Hardware Buttons and Features

1 Power

Press and hold this button to turn the device on or off.
Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

2 Volume


Press the left of volume toggle to increase speaker volume or press the right of the toggle to decrease volume.

3 Home


Tap to return to the home screen. If your wego A model does not have a Home button, use 1 Power.

4 Charging the wego A

Charge device and speaker at night or when not in use.

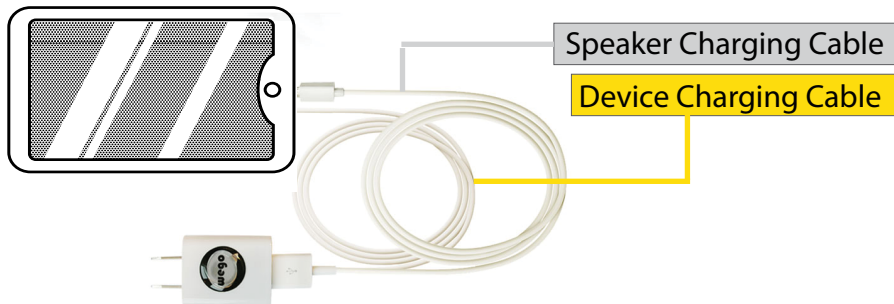

wego 5A

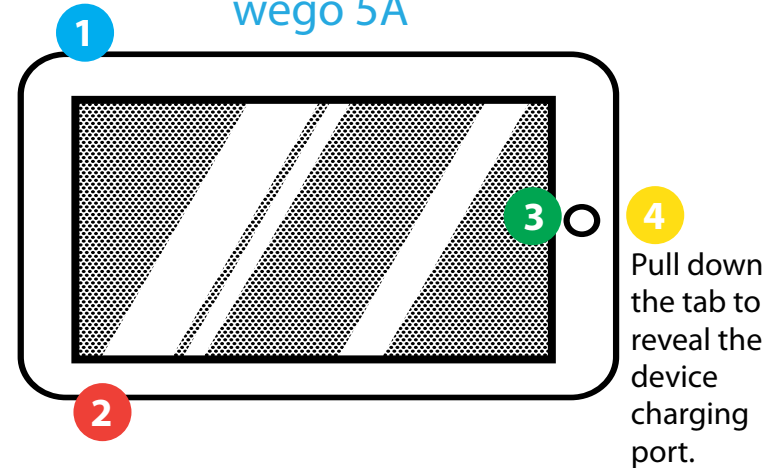

wego 7A, 10A & 13A

