




We bring speech to life™

# wego™ A Quick Reference TouchChat



## Edit a Button Message and Image

1. **Menu > Edit Page > select button to edit > Edit This Button.**
2. *Button Label:* In text box below *Button Label*, type text to display on button > **Done** (keyboard).
3. *Button Message:* In text below *Button Message*, type message. Button should speak when pressed > **Done** (keyboard).
4. Select, import, or take a new button image:  
*Choose a symbol:* **Select Image > tap magnifying glass**  > type word to search > **Search > select symbol > Save > Done.**  
*Import photo:* **Import Image > Image Library > select picture from photo library > Use > Save > Done.**  
*Take a picture:* **Import Image > Camera > press white circle on screen to take a picture > Use Photo > Save > Done.**

## Add a Button Action > NAVIGATE

1. **Menu > Edit Page > tap button to edit > Edit This Button.**
2. Scroll down to *Button Actions* > **Add > Navigate.**
3. Select page to navigate to > select **No Animation** or animation type > **Save > Done.**

## Create a New Page from a Template

1. **Menu > Edit Page .**
2. **Menu > Load Another Page > choose “.template” > Done.**
3. **Menu > Import/Copy Pages > Duplicate this Page.**
4. **Menu > Load Another Page > choose “.template (Copy)” > Done.**
5. **Menu > Edit Page Layout > under Page Name, type new page name > Save > Done.**
6. Connect the new page to a button by following *Add a Button Action > NAVIGATE* Steps 1–3.

## Copy and Paste Buttons

1. **Menu > Edit Page > select button to copy > Copy Button.**
2. Tap location to paste button > **Paste Button.**
3. To copy button style only, follow above instructions, but select **Paste Button Style** on Step 2.

## Change the Pronunciation of a Word

1. **Menu > Edit Pronunciations.**
2. Tap the **+** icon.
3. In the *Word* text box, type word you would like to edit (e.g. fajita).
4. In the *Pronounce As* text box, type the word’s proper pronunciation (e.g. fuhheeta).
5. Select **Pronounce** to hear pronunciation.
6. **Save > Done.**

## Hide Mode

1. **Menu > Edit Page > Menu > Hide Mode.** The task bar will turn green.
2. Select buttons to hide > **Done.**
3. To unhide buttons: **Menu > Edit Page > Menu > Hide Mode > select buttons to unhide > Done.**

## Guided Access

All dedicated devices come with *Guided Access* enabled.

1. Open *Guided Access* menu by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111).**
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. *Lock Menu* (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume.**

*Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com



We bring speech to life™

# Hardware, Buttons & Features



## 1 Power

Press and hold this button (plus a volume button on some models) to turn the device on or off.

*Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.*

## 2 Volume

Use these buttons to increase or decrease volume level.


## 3 Home

Tap to return to the home screen. If your wego A model does not have a Home button, swipe up from the bottom of the screen.

## 4 Charging the wego A

Charge device and TTMT Speaker (SPKR) at night or when not in use. Please only use a TTMT-approved charger with your device.


wego 5A-D

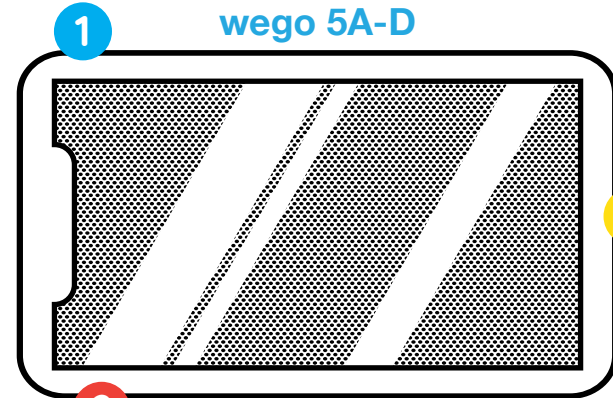

4

*Pull down the tab to reveal the device charging port.*

wego 7A-D, 10A-D & 13A-D

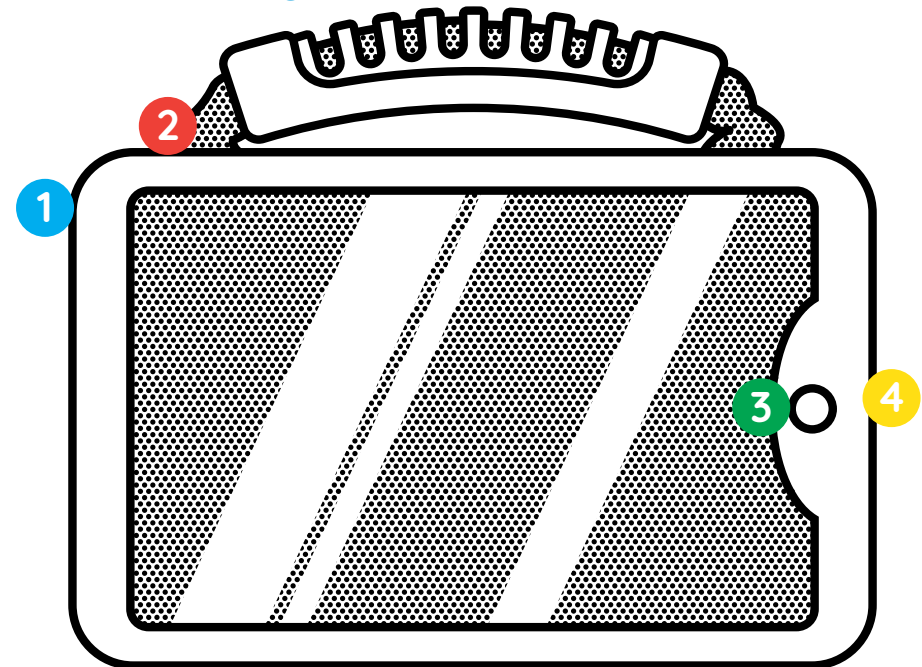

*If your wego A model does not have a Home button, swipe up from the bottom of the screen.*

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com