


Edit Mode and the editing screen overview

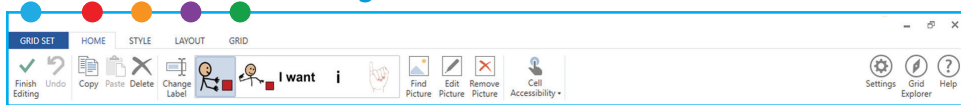

To enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid**.

The editing toolbar will appear at the top of your screen.

- Grid Set:** Navigate grids/vocabulary sets while in edit mode, and create new grid/vocabulary sets from existing templates.
- Home:** Editing commands, including options to customize vocabulary cells.
- Style:** Options to edit cell style and formatting.
- Layout:** Insert, delete, and arrange rows and columns, and adjust cell spacing.
- Grid:** Choose the background color, add button commands, and modify scan blocks for access switches.

Change button label, image, or message

Enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid**.

Change Button Text

Select **Change Label** > in the button text space, type new label.

Change Button Message

In the *Write Text* box, type the message to speak when the button is pressed.

Change Button Image

Select **Find Picture** > select one of the following:

- **Symbols:** Type label name or word to search symbols > select symbol > **OK**.
- **Picture File:** Select folder containing image > tap photo to use.
- **Camera:** Select camera > press **Capture** to take a photo.

Finish Editing

When finished, select **Finish Editing** to save changes and exit Edit Mode.

Create a new page from an existing template

1. Enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid** > >
2. Click to select **.new page template** > **Copy** > **Rename** > **OK**.

Note: To add a page from a different Grid Set, refer to the extended User Guide.

Link (jump) to a new page

1. Enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid**.
2. Select button you want to link > **Add Command**.
3. > **Next** > select the new grid/page you created > **OK**.

Copy and paste a button

To the same page:

1. Enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid**.
2. Touch and drag the button to a new location on the same page. The buttons will swap locations.

To another page:

1. Enter Edit Mode: Select (**Menu**) from the top toolbar > **Edit grid**.
2. Select button > **Copy**.
3. Navigate to the page you want to paste the button.
4. Select the location to paste the button > **Paste**.

Change the pronunciation of a word

1. (**Menu**) > **Settings** > **Speech** > **Pronunciation**.
2. In the *Word* box: type the word with the correct spelling (i.e. fajita).
3. In the *Pronunciation* box: type the word as you want it pronounced (i.e. faheeta).
4. Select **Test** to hear the word pronounced.
5. When finished, select **Add Pronunciation** > **OK**.

Edit/use a word list

1. (**Menu**) > **Edit Word List** > **Add** > type your new word into the text box.
2. Choose a picture or add a photo > **OK**.

Tip: Add multiple words at one time by pressing Enter on the keyboard after each word.

Note: If your grid/page does not utilize a word list, the "Add" button will not be visible. Create a page for word lists by following the instructions in the extended User Guide for Create a new page from an existing template.



Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.