



We bring speech to life™

# zuvo™ Quick Reference

## Edit Mode and the editing screen overview

To enter Edit Mode: Select (Menu) from the top toolbar > Edit grid.  
The editing toolbar will appear at the top of your screen.

- Grid Set:** Navigate grids/vocabulary sets while in edit mode, and create new grid/vocabulary sets from existing templates.
- Home:** Editing commands, including options to customize vocabulary cells.
- Style:** Options to edit cell style and formatting.
- Layout:** Insert, delete, and arrange rows and columns, and adjust cell spacing.
- Grid:** Choose the background color, add button commands, and modify scan blocks for access switches.



## Change button label, image, or message

Enter Edit Mode: Select (Menu) from the top toolbar > Edit grid.

### Change Button Text

Select **Change Label** > in the button text space, type new label.

### Change Button Message

In the *Write Text* box, type the message to speak when the button is pressed.

### Change Button Image

Select **Find Picture** > select one of the following:

**Symbols:** Type label name or word to search symbols > select symbol > **OK**.

**Picture File:** Select folder containing image > tap photo to use.

**Camera:** Select rear or front camera > press **Capture** to take a photo.

### Finish Editing

When finished, select **Finish Editing** to save changes and exit Edit Mode.

## Create a new page from an existing template

1. Enter Edit Mode: Select (Menu) from the top toolbar >

**Edit grid** > >

2. Click to select **.new page template** > **Copy** > **Rename** > **OK**.

*Note: To add a page from a different Grid Set, refer to the extended User Guide.*

## Link (jump) to a new page

1. Enter Edit Mode: Select (Menu) from the top toolbar > Edit grid.
2. Select button you want to link > **Add Command**.
3. > **Next** > select the new grid/page you created > **OK**.

## Copy and paste a button

### To the same page:

1. Enter Edit Mode: Select (Menu) from the top toolbar > Edit grid.
2. Touch and drag the button to a new location on the same page. The buttons will swap locations.

### To another page:

1. Enter Edit Mode: Select (Menu) from the top toolbar > Edit grid.
2. Select button > **Copy**.
3. Navigate to the page you want to paste the button.
4. Select the location to paste the button > **Paste**.

## Change the pronunciation of a word

1. (Menu) > **Settings** > **Speech** > **Pronunciation**.
2. In the *Word* box: type the word with the correct spelling (i.e. fajita).
3. In the *Pronunciation* box: type the word as you want it pronounced (i.e. faheeta).
4. Select **Test** to hear the word pronounced.
5. When finished, select **Add Pronunciation** > **OK**.

## Edit/use a word list

1. (Menu) > **Edit Word List** > **Add** > type your new word into the text box.
2. Choose a picture or add a photo > **OK**.

*Tip: Add multiple words at one time by pressing Enter on the keyboard after each word.*

*Note: If your grid/page does not utilize a word list, the "Add" button will not be visible. Create a page for word lists by following the instructions in the extended User Guide for Create a new page from an existing template.*

*Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com



We bring speech to life™

# Hardware, Buttons & Features

## Power

Press this button to turn on and off the device. Tap button once to place in sleep mode and conserve battery life when the zuvo is not in use.

## Volume

Press toggle to increase and decrease volume.

## Charging the zuvo


Charge the device at night or when not in use. TTMT Speaker (SPKR) should also be charged nightly with the separate cord provided. Only use TTMT-approved chargers.

## USB Ports


Connect access options, back-up vocabulary files and transfer photos and other files.

## Headphone/Microphone


Audio input/output for sound feedback, voice and message banking and more.

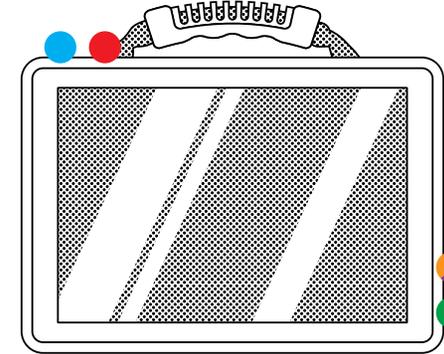

zuvo 10-D

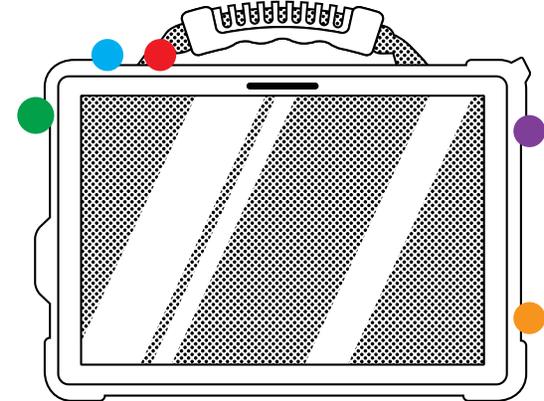

zuvo 12HD-D

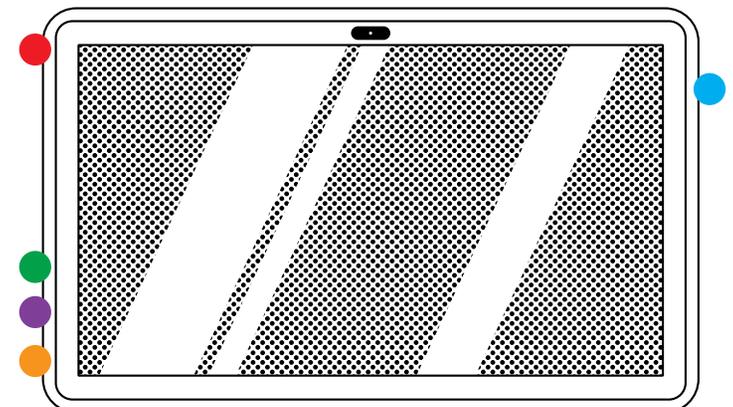

zuvo 18HD-D

Need help?

 877.392.2299 Option 3

 [support@talktometechnologies.com](mailto:support@talktometechnologies.com)

 [talktometechnologies.com](http://talktometechnologies.com)