

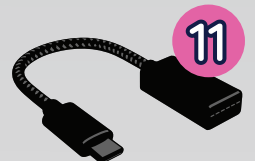
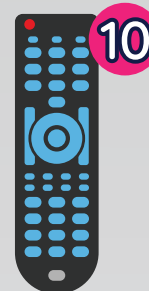
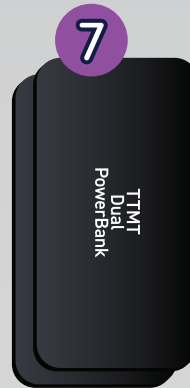


Included with your Zuvo™ 12 with eye gaze

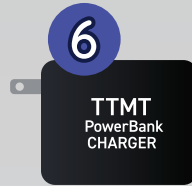


- 1 Speaker charger
- 2 Speaker
- 3 Device charger
- 4 Stylus (x2)
- 5 Keyboard
- 6 Dual battery bank charger
- 7 Dual battery banks
- 8 Welcome folder
- 9 Shoulder strap
- 10 Universal remote
- 11 USB-C to USB-A adaptor

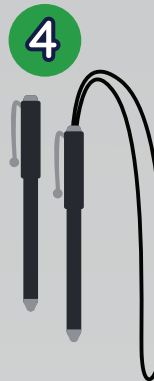

Smartbox family

 talktometechnologies.com

 877.392.2299

Hardware, Buttons, & Features



Power

Press this button to turn on and off the device. Tap button once to place in sleep mode and conserve battery life when the device is not in use.



USB-C Ports

Use port for connecting accessibility options such as a mouse, keyboard, or switch interface.



Headphone/Microphone

Audio input/output for sound feedback, voice and message banking and more.



Volume

Press the toggle to adjust volume levels.




Charging the zuvo


An external battery, TTMT Dual PowerBanks or Charger Extension Cable may be included with your device. Refer to the documentation included with your device.



Reserved for zuvo Control ECU - DO NOT remove.

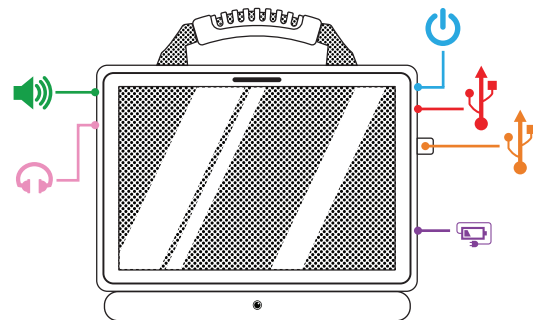

Zuvo 12 with Vida Go™

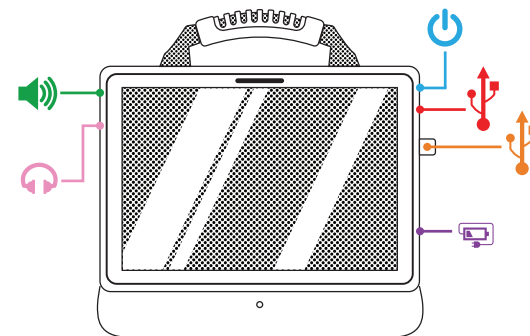

Zuvo 12 with Eyespeak Inspo™

Proper Positioning

The top of the device should be even with the top of the user's head. The device may be placed to the side, or tilted down if placed on a high table.

Vida Go:

approx. 19-33" (optimal distance 25")




Eyespeak Inspo:

approx. 14-31"



- 1 IR Ports*
- 2 Learning window for IR*
- 3 Switch ports
- 4 USB Port
- 5 Mount Plate

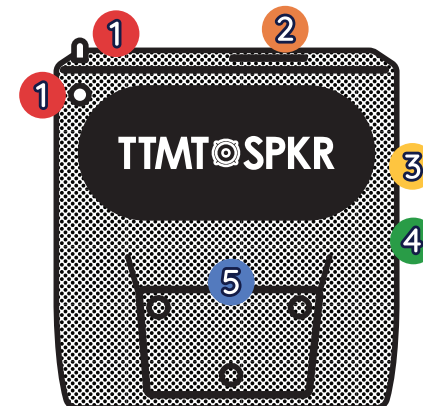

Note: It is important to shut down the device when you are no longer using it. This will ensure that the entire system continues to work well. Please make sure the system is shut down while charging overnight to allow it to fully charge.

*Contact TTMT for details

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com