

Included with your Zuvo™ 16 with eye gaze



1 Speaker charger




2 Speaker



3 Device charger



4 Stylus (x2)

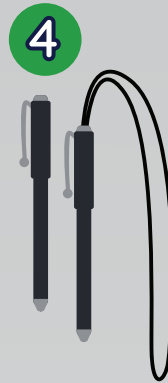

5 Keyboard



6 Dual battery bank charger




7 Dual battery banks




8 Welcome folder



9 Universal remote

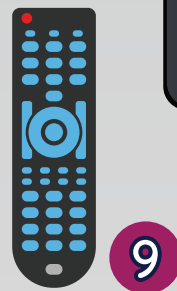


10 USB-C to USB-A adaptor

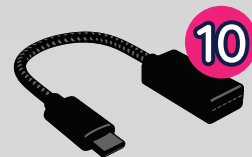

Smartbox family

 talktometechnologies.com

 877.392.2299

Hardware, Buttons, & Features



Power

Press this button to turn the device on and off.



USB-C Ports

Use ports for connecting accessibility options such as a mouse, keyboard, or switch interface.




Headphone/Microphone


Audio input/output for sound feedback, voice and message banking and more.



Reserved for Zuvo Control ECU – DO NOT remove.

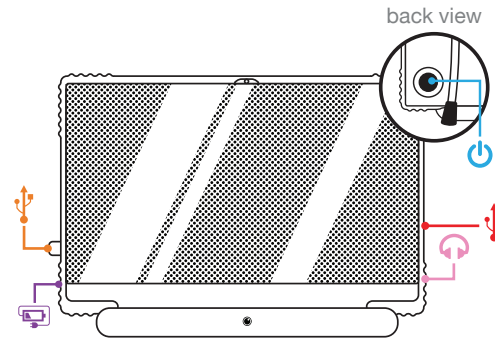

Zuvo 16 with Vida Go™

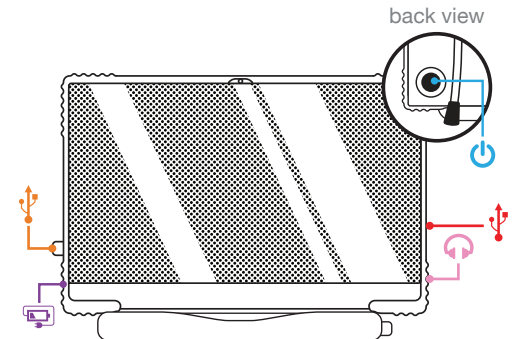

Zuvo 16 with Eyespeak Inspo™

Proper Positioning

The top of the device should be even with the top of the user's head. The device may be placed to the side, or tilted down if placed on a high table.


Vida Go:
approx. 19-33" (optimal distance 25")



Eyespeak Inspo:
approx. 14-31"



- 1 IR Ports*
- 2 Learning window for IR*
- 3 Switch ports
- 4 USB Port
- 5 Mount Plate

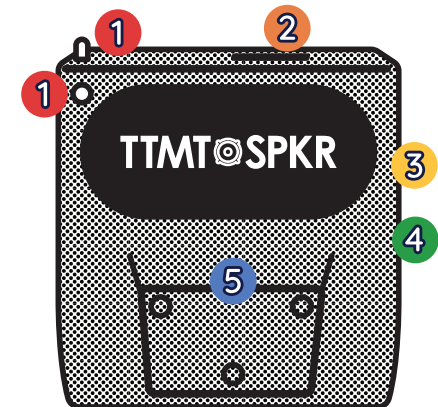

Note: It is important to shut down the device when you are no longer using it. This will ensure that the entire system continues to work well. Please make sure the system is shut down while charging overnight to allow it to fully charge.

*Contact TTMT for details

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com