

## Four steps to quick & easy keyguard care!



1

To remove the keyguard, **pinch** one corner of the speech display window on the keyguard while holding the device in place with your other hand.



2

Using force, **lift** the corner of the keyguard up and away from the device. Be sure to continue to hold the device in place as you pull.



3

Remove dust and dirt by **cleaning** the keyguard and mounting strips with a damp cloth. Use a dry cloth to wipe the screen of the device before reapplication.



4

**Reapply** the keyguard to the device by firmly, but gently pressing the corners with the mounting strips directly to the screen.

