



Customer Care
877-392-2299 #3
support@talktometechnologies.com

We bring speech to life™


4 steps to quick & easy keyguard care!



TTMT keyguards are held securely in place by pressure from the device case. To remove the keyguard, pinch one corner of the speech display window on the keyguard while holding the device in place with your other hand.



Using force, lift the corner of the keyguard up and away from the device. Hold the device in place as you pull.

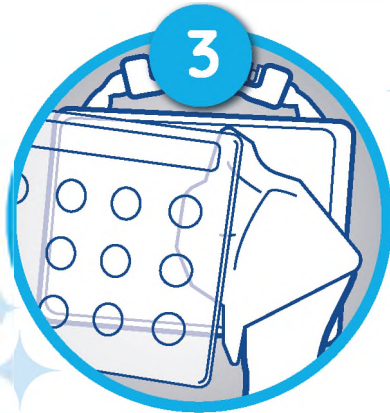

Remove dust and dirt by cleaning with a damp cloth. Use a dry cloth to wipe the screen of the device before reapplication.



To place the keyguard back on the device, line up one side of the keyguard directly onto the screen where it meets the case. Firmly but gently, press the opposite corners of the keyguard until it locks into place and sits flush with the case.



Find more resources online: talktometechnologies.com