



We bring speech to life™

wego™ A Quick Reference



Edit/change vocabulary categories

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select button to edit > **Edit Cell**.
3. To change the label of the button, select the label on the cell > type new label.
4. To change the picture of the button, select the picture on the cell > choose a different symbol or **Find Picture** for more options > select one of the following:
 - a. Symbols: Type label name or word to search symbols > select symbol.
 - b. Photo Library: Select folder containing image > select desired photo.
 - c. Web Search: Type label name or word to search web > select image.
 - d. Camera: Select front or back camera > press **Capture** to take photo.
5. Select **Done** > **Finish Editing** to save changes and exit *Edit Mode*.

Save phrases on category pages

Independently program the vocabulary without entering into the *Edit Mode*.

1. Select **Keyboard** > type the message you want to save to a button.
2. Select **myPhrases** > select the category you want to save you message under.
3. Select **Save** > touch inside the new button that appears.

*Note: To delete, select **Delete** > touch inside the button you want to delete.*

Edit/add words to word list within category

Add several words to a category at once.

1. Navigate to the desired grid/page.
2. Enter *Edit Mode* by selecting **Options** ••• > **Edit Word List** ☰
3. Select **+** to add new word.
4. Type the new word into the text box.
5. Select **Word List** to return to main page > **Done**.

Link/jump to grid/page from new button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select blank cell > **Jump to Another Grid** >
 - a. Select the existing grid/page you want to link/jump to **OR...**
 - b. Select **Copy Grid** > choose ".template" > rename the grid/page > **Done**.

Note: Not for Word List pages.

Link/jump to grid/page from existing button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select button you want to link/jump from > **Edit Cell** > **Add Command** > **Jumps** > **Jump To** >
 - a. Select the existing grid/page you want to link/jump to **OR...**
 - b. Select **Copy Grid** > choose ".template" > rename the new grid/page > **Done**.

Change pronunciation

1. Open *Settings* by selecting **Options** ••• > **Settings** ⚙️
2. **Speech** > **Pronunciation** > select **+** to add new word.
3. In the *Word* box: type the word with the correct spelling (e.g. fajita).
4. In the *Pronunciation* box: type the word as you want it pronounced (e.g. faheeta).
5. Select **Test** to hear how the word sounds.
6. When finished, select **Save** > **Done**.

Copy/paste button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select desired button > **Select** > **Copy** > select new location(s) > **Paste** > **Done**.

Note: Not for Word List pages.

Tip: You can also enter the button selection menu by pressing and holding the desired button until it is activated.

Hide/show cells

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids**.
2. Select desired button > **Select** > select more buttons if desired.
3. Open Style menu by selecting **Cell Accessibility** > **Hidden** > **Done**.
4. Select **Finish Editing** to save changes and exit *Edit Mode*.

Note: Not for Word List pages.

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



www.talktometechnologies.com



We bring speech to life™

Hardware Buttons and Features



1 Power

Press and hold this button to turn the device on or off.
Note: Press this button one time while the wego™ A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego™ A is not in use.

2 Volume


Press the left of volume toggle to increase speaker volume or press the right of the toggle to decrease volume.

3 Home

Tap to return to the home screen.

4 Charging the wego™ A

Pull down the tab to reveal the device charging port.
Charge device and speaker at night or when not in use.

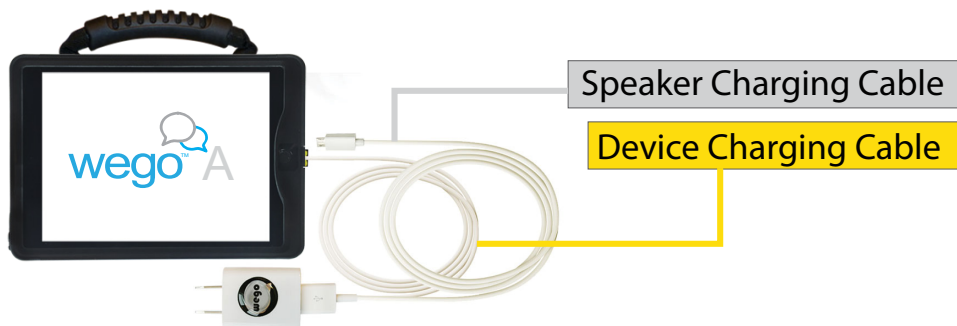

wego 7A, 10A & 13A

