



We bring speech to life™

# wego™ A Quick Reference



## Change button label, image and messages

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select button to edit > **Edit Cell**.
3. Make the button speak by adding **OR** editing the “Write” command:
  - a. Add: **Add Command** > **Chat and Writing** > **Write** > type spoken message > **Done**.
  - b. Edit: Select text area within the “Write” command > type the new spoken message.
4. Keep the message and label the same or make them different within the “Write” command:
  - a. Keep the same: Toggle on **Same as Cell Label OR...**
  - b. Make different: Toggle off **Same as Cell Label** > change the button label by selecting the current label on the cell > type new label > **Done**.
5. Change the button image by selecting the current image on the cell > choose a different symbol or **Find Picture** for more options > select one of the following:
  - a. Symbols: Type label name or word to search symbols > select desired symbol.
  - b. Photo Library: Select folder containing image > select desired photo.
  - c. Web Search: Type label name or word to search web > select desired image.
  - d. Camera: Select front or back camera > press **Capture** to take photo.
6. Select **Done** > **Finish Editing** to save changes and exit *Edit Mode*.

## Edit/add to word lists

1. Navigate to the desired grid/page.
2. Enter *Edit Mode* by selecting **Options** ••• > **Edit Word List** ≡
3. Select **+** to add new word.
4. Type the new word into the text box.
5. Select **Find Picture** to add a symbol or picture to the button.
6. Select **Word List** to return to main page > **Done**.

*Note: Word List buttons are generally a peach colored button on a category page.*

## Link/jump to grid/page from new button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select blank cell > **Jump to Another Grid** >
  - a. Select the existing grid/page you want to link/jump to **OR...**
  - b. Select **Copy Grid** > choose “.template” > rename the grid/page > **Done**.

*Note: Not for Word List pages.*

## Link/jump to grid/page from existing button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select button you want to link/jump from > **Edit Cell** > **Add Command** > **Jumps** > **Jump To** >
  - a. Select the existing grid/page you want to link/jump to **OR...**
  - b. Select **Copy Grid** > choose “.template” > rename the new grid/page > **Done**.

## Change pronunciation

1. Open *Settings* by selecting **Options** ••• > **Settings** ⚙️
2. **Speech** > **Pronunciation** > select **+** to add new word.
3. In the *Word* box: type the word with the correct spelling (e.g. fajita).
4. In the *Pronunciation* box: type the word as you want it pronounced (e.g. faheeta).
5. Select **Test** to hear how the word sounds.
6. When finished, select **Save** > **Done**.


## Copy/paste button

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select desired button > **Select** > **Copy** > select new location(s) > **Paste** > **Done**.

*Note: Not for Word List pages.*

*Tip: You can also enter the button selection menu by pressing and holding the desired button until it is activated.*

## Hide/show cells

1. Enter *Edit Mode* by selecting **Options** ••• > **Edit Grids** ✎
2. Select desired button > **Select** > select more buttons if desired.
3. Open Style menu by selecting  > **Cell Accessibility** > **Hidden** > **Done**.
4. Select **Finish Editing** to save changes and exit *Edit Mode*.

*Note: Not for Word List pages.*

*Important Note: Vocabulary sets are updated frequently. If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

Need help?



877.392.2299 Option 3



support@talktometechnologies.com



www.talktometechnologies.com



We bring speech to life™

# Hardware Buttons and Features

## 1 Power

Press and hold this button to turn the device on or off.  
Note: Press this button one time while the wego A is on to put it into "Sleep" mode. This feature will conserve battery life when the wego A is not in use.

## 2 Volume


Press the left of volume toggle to increase speaker volume or press the right of the toggle to decrease volume.

## 3 Home

Tap to return to the home screen. If your wego A model does not have a Home button, use 1 Power.

## 4 Charging the wego A

Charge device and speaker at night or when not in use.

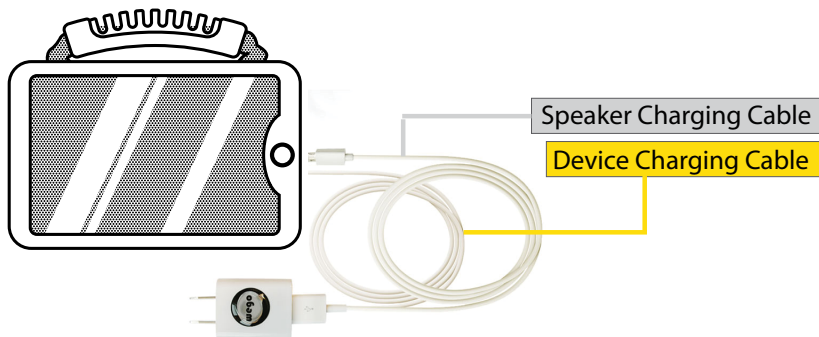

wego 7A, 10A & 13A

