



Wego 13A










Talk Pad Wego 10



Talk Pad Wego 8



Edit a Button Message and Image

- Edit  > tap button to edit > Change 
- The following are available on this page:
 - Edit the caption: Tap within the Caption text box and type the caption.
 - Select, import, or take a new button image:
 - Choose a symbol: Tap within the text box beside Find symbol, type the symbol you are looking for > **Search**. To see all options available, tap the **black arrow** next to the button image and scroll until you find the preferred image. Note: To search for web images, toggle to **Web**.
 - Import photo: Tap import icon  > All Photos > Select picture from photo library.
 - Take a picture: Tap camera icon  > press white circle on screen to take a picture > **Use Photo**.
 - Edit what the button will speak: Tap within the Speak as text box and type what the button will speak.
 - To test speech: Tap the test speech icon .
 - To record personal voice: Tap record button  > speak message > touch **stop** button.
 - To delete speech: Tap the delete icon .
 - Edit the word type: Select the drop down next to *Word type*.
- When finished editing, tap **Save**. To exit without saving changes, tap **Cancel**. Tap **Edit** to exit editing mode.

Adding a New Button or Category

- Edit > Add New.**
- Choose one of three options:
 - Add many items together**
 - To do this, enter all words (buttons) you want to add, separated by commas.
 - To make a button a category, add a "-" in front of it.
 - Select **Ok**. Tap **Edit** to exit editing mode.
 - Add a new message** - see above (*Edit a Button Message and Image*)
 - Add a new category** - see above (*Edit a Button Message and Image*)

Adjust Page Layout

- Settings > Pictures per screen**
- Select how many buttons per page > **Ok** > touch screen anywhere to return to vocabulary.

Searching for a Button

- Search** > type word to search > **Search**
- Tap **See Path** to see exactly where the button is located. If "No Matches Found," you will need to manually add that button into the vocabulary set (*See Adding a New Button or Category*).

Moving or Rearranging Buttons

- Edit** > press and hold on the button you want to move.
- While continuing to hold the button, move the button to the new desired location. If you move the button onto an existing button, it will swap the 2 buttons locations. Tap **Edit** to exit editing mode. Copy/Paste Buttons

Copy/Paste Buttons

- Edit** > select the button you want to copy > **Copy**.
- Go to the page that you want to paste the copied button > **Paste**.
- Tap **Edit** to exit editing mode.

Hide Mode

Tap **Hide**. Tap **Edit** to exit editing mode.

To unhide buttons: **Edit** > select the button(s) you would like to unhide > **Show** > Tap **Edit** to exit editing mode.

Guided Access

All dedicated devices come with Guided Access enabled.

- Open Guided Access menu by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
- Enter passcode: default password is all **1s (1111 or 111111)**.
- To continue *Guided Access*: press **Resume** (top right).
- To turn off *Guided Access*: press **End** (top left).
- Lock Menu (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume**.

Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

