



Wego 13A




Talk Pad Wego 10



Talk Pad Wego 8



Edit a Button Message and Image

1. Menu > **Edit Page** > select button to edit > **Edit This Button**.
2. Button Label: In text box below Button Label, type text to display on button > Done (keyboard).
3. *Button Message*: In text below Button Message, type message button should speak when pressed > **Done** (keyboard).
4. *Select, import, or take new button image*:
Choose a symbol: **Select Image** > tap magnifying glass  > type word to search > **Search** > select symbol > **Save** > **Done**.
Import photo: **Import Image** > **Image Library** > select picture from photo library > **Use** > **Save** > **Done**.
Take a picture: **Import Image** > **Camera** > press white circle on screen to take a picture > **Use Photo** > **Save** > **Done**.

Add a Button Action > NAVIGATE

1. **Menu** > **Edit Page** > select button to edit > **Edit This Button**.
2. Scroll down to *Button Actions* > **Add** > **Navigate**.
3. Select page to navigate to > select **No Animation** or animation type > **Save** > **Done**.

Create a New Page from a Template

1. **Menu** > **Edit Page**.
2. Menu > Add New Page > under *Page Name*, type new page name > **Save** > **Done**.
3. Connect the new page to a button by following *Add a Button Action* > *NAVIGATE* Steps 1–3.

Copy and Paste Buttons

1. **Menu** > **Edit Page** > select button to copy > **Copy Button**.
2. Tap location to paste button > **Paste Button**.
3. To copy button style only: Follow above instructions but select **Paste Button Style** on Step 2.

Change the Pronunciation of a Word

1. **Menu** > **Edit Pronunciations**.
2. Tap the + sign.
3. In the *Word* box, type the word you would like to edit (e.g. fajita).
4. In the *Pronounce As* box, type the word's proper pronunciation (e.g. fuhheeta).
5. Select **Pronounce** to hear pronunciation.
6. Save > **Done**.

Word Finder

1. **Menu** > Word Finder > Type word to search > Find.
2. The path to find the word will display.
3. Select the path and it will guide you through the motor plan to selected button.

Guided Access

All dedicated devices come with Guided Access enabled.

1. Open Guided Access menu by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111)**.
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. Lock Menu (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume**.



Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.