



Wego 13A



Talk Pad Wego 10



Talk Pad Wego 8



Wego 5A

Installing updates regularly will help ensure that your vocabulary/page set software runs properly. The following instructions will guide you through the software update process.

### Initial Setup: Turn On Access to the App Store

1. Turn off *Guided Access*: quickly triple-click **Home** button (or **Power** button if no **Home** button is present) to open *Guided Access* > enter passcode (default passcode is all **1s**) > press **End** (top left).
2. Press **Home** button (or swipe up from bottom of screen if no **Home** button is present) to view device home page.
3. Connect to Wi-Fi if not already: **Settings** > **Wi-Fi** > connect to your network.
4. **Settings** > **Screen Time** > **Content & Privacy Restrictions** > enter passcode: **9357**.
5. At the top of the screen, toggle **Off** Content & Privacy Restrictions.
6. Press **Home** button (or swipe up from bottom of screen) and *App Store* icon will appear on device home screen.

### Update Software in the App Store

1. Select the **App Store** icon.
2. Select the **Profile Icon** in the top right to display applications with available updates.
3. Select Update button next to application or select **Update All** to update all applications at once.
4. If prompted, enter your Apple user ID and password. This is printed in your *User Guide*. If you cannot locate your user ID or password, please contact our support department at 877.392.2299 option 3, or support@talktometechnologies.com.



### Update Complete: Turn Off Access to the App Store

1. After update is complete, press **Home** button (or swipe up from bottom of screen if no **Home** button is present) to return to home page.
2. **Settings** > **Screen Time** > **Content & Privacy Restrictions** > enter passcode: **9357**.
3. At the top of the screen, toggle **On** *Content & Privacy Restrictions*.
4. Press **Home** button (or swipe up from bottom of screen if no **Home** button is present) to return to home screen.
5. Open vocabulary set.
6. Quickly triple-click **Home** button (or **Power** button if no **Home** button is present) to return to *Guided Access*.

*Tip: Occasionally a pop-up notification may prompt you to update the IOS software. When time allows, follow the on-screen instructions to update the operating system.*

*Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

*LAMP Words for Life™ is a trademark of PRC-Salttillo. All rights reserved. The LAMP Treatment Approach is a trademark of the Center for AAC & Autism. All rights reserved. TouchChat® is a registered trademark of PRC-Salttillo. All rights reserved.*

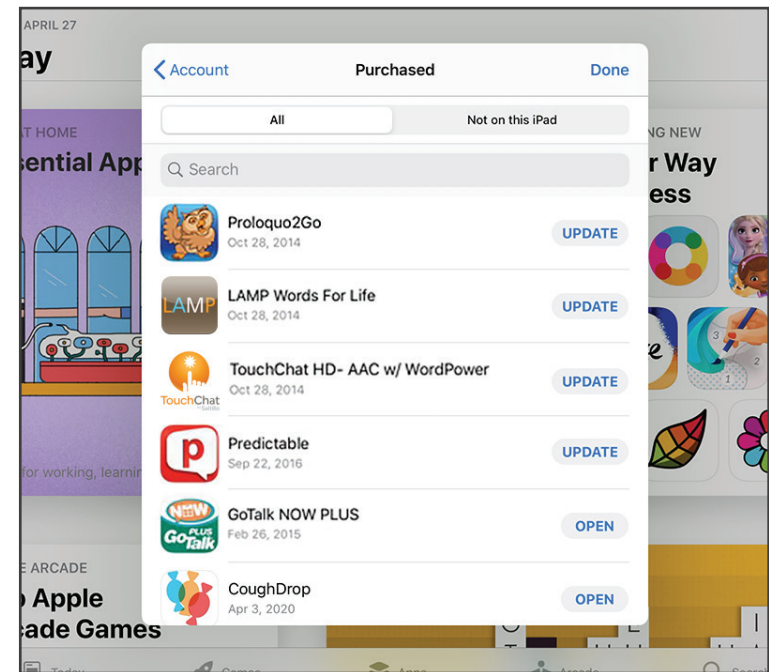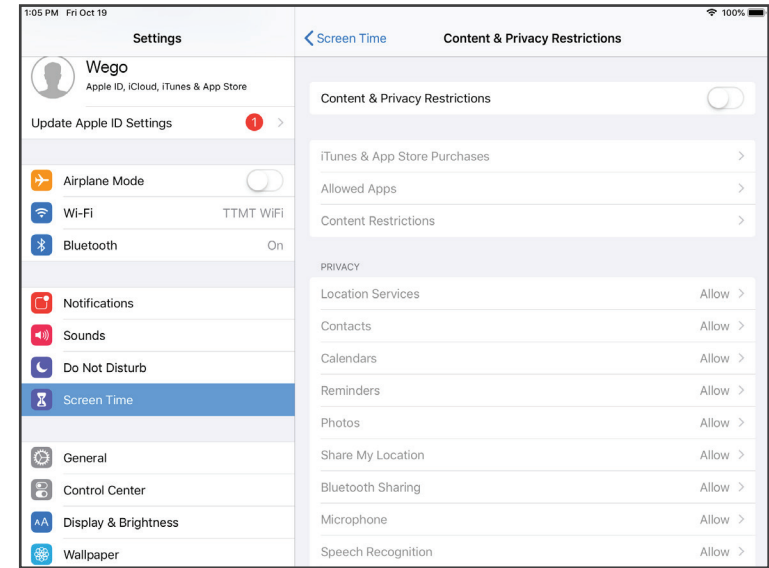

*Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.*

**Need Help?**

877.392.2299 Option 3

Support@TalkToMeTechnologies.com

TalkToMeTechnologies.com (click **Support**)

prc-salttillo  
Authorized App Reseller