



Wego 13A



Talk Pad Wego 10



Talk Pad Wego 8



Edit a Button Message and Image

Note: On the main screen the only editing functions available are the ability to change the image and change the button color. On a secondary page, you may change or add content.

1. Tap **Edit** in the top left corner > tap the button you wish to edit.
2. Select **Image Options** to change the button image.
 - a. Select *Take Photo* to open the camera and take a picture or select *Choose Photo* to select an saved image from the device > **Use Photo** OR
 - b. Tap the symbol on left side of editing menu to search for and select a different pre-loaded symbol.
3. Select *Cell Color* to change the button color > select **Done** in the top right corner to save changes > select **Done** in the top left corner to exit Edit Mode.

Tip: If "Word to Speak" appears red in color and will not save when adding a new button, this indicates that the button already exists in the vocabulary set. Speak For Yourself will not allow content to appear repetitively in the vocabulary set.

To add a new word

1. Navigate to the category/location where you plan to add the word > tap **Edit** > select a blank button.
2. Select **Image Options** to add a button image.
 - a. Select *Take Photo* to open the camera and take a picture or select *Choose Photo* to select an saved image from the device, then select **Use Photo** OR
 - b. Tap the symbol on left side of editing menu to search for and a different pre-loaded symbol.
3. Select *Cell Color* to change the button color.
4. Type the button message in *Word to Speak* > change the label if it is different from the message > select **Done** to save changes > select **Done** to exit Edit Mode.

Unlink a Primary button from the Secondary level

*Note: On the main screen the editing functions are limited. You can only change pronunciation on a secondary page.

1. Select **Edit** > tap the button you want to change.
2. Type the word's proper pronunciation in the **Pronounce Like** space. For example, "fuheetah" for the word "fajita."
3. Select *Speak Word* to test the pronunciation and modify as needed > select **Done** in the top right corner to save changes > select **Done** to exit Edit Mode.

Change Pronunciation of a word

1. Select *Open/Close* on top right side of toolbar along the top.
2. Select the buttons you would like to "close" and make inaccessible. They will turn gray. Press **Done**.
3. To hide a large number, you can also tap the *Close all* button on the lower right corner of the screen . All of the buttons will turn gray, indicating that they are the buttons that will be "closed" or hidden.
4. Select the buttons that you want to remain accessible. They will brighten as you touch them. Select **Done** in the top right corner.

Using the Babble Feature

1. When buttons are "closed" (or hidden), you can select *Babble* on the top right to make all of the buttons in the vocabulary set visible and active.
2. When the AAC user is done exploring all vocabulary options, you can again select *Babble* to return to the individual user's customized open/close settings.

Hold That Thought

1. Once a phrase or sentence has been generated, a user can store this phrase with *Hold That Thought*. To do so, generate the desired phrase/sentence.
2. Select *Hold That Thought* > tap the + sign in the upper left-hand corner. The phrase will appear as a saved phrase and can be accessed at a later time in the *Hold That Thought* menu.

Access full programming guide

To access the entire programming guide built into Speak For Yourself, click on the * icon in the top right corner > scroll down to *Programming Guide* under the *Resources* heading.

Guided Access

All dedicated devices come with Guided Access enabled.

1. Open Guided Access menu by quickly triple-clicking the Home button. If your device does not have a Home button, use the Power button.
2. Enter passcode: default password is all **1s (1111 or 111111)**.
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. Lock Menu (restrict use of specific buttons/categories): open *Guided Access* menu > use finger to draw a circle around buttons to restrict > **Resume**.



Important Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.