




We bring speech to life™

wego[™]write Quick Reference

Predictable



Quick keys and customization buttons

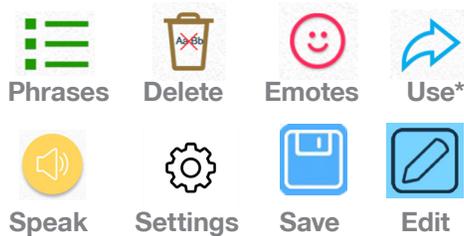

Add a phrase category

1. Select **Phrases** > select **Edit** at bottom of page > swipe left to the last category page > tap **Add category**.
2. Type name of new phrase category at the blue cursor.
3. Select **No Image**, or select one of the following to add an image:
 - Take from camera** > take photo using camera > **Use Photo**.
 - Choose from gallery** > tap to select image from gallery.
 - Choose from Predictable image library** > tap to select symbol.
4. When finished, tap **Save** at the top right of screen.

Save a phrase

1. **Phrases** > select desired category > select **Add a new phrase...** at the bottom of the list of saved phrases.
2. Select **Text** from left navigation menu.
3. On *Text to speak* line: type message to speak when phrase is selected.
4. On *Text to display* line: type text (label) to display in phrase bank.
5. When finished, tap **Save** at the top right of screen.

Quick-save a phrase from the homepage

1. Type phrase using keyboard on homepage > select **Use** > select **Add** from pop-up menu.
2. Select category to save new phrase > tap **Save** at the top right of screen.

Edit a saved phrase

1. **Phrases** > select desired category > select **Edit** at bottom of page > tap phrase to edit > choose **Edit** from pop-up menu.
2. Follow steps 2-5 from **Save a phrase** to finish editing.

Change pronunciation of a word

1. Select **Settings** > select **Speech** from left menu > select **Speech Settings** at the top right of the page > **Custom Words** > select **Custom Pronunciation**.
2. On *Display* text line: type word with correct spelling (i.e. fajita).
3. On *Pronunciation* text line: type word as you want pronounced (i.e. faheeta).
4. Press **Play** next to pronunciation to hear how word will sound.
5. When finished, tap **Save** at the top right of screen.

Guided Access

All dedicated devices come with *Guided Access* enabled.

1. Open *Guided Access* by quickly triple-clicking the Home button.
2. Enter passcode: default password is all **1s (1111 or 111111)**.
3. To continue *Guided Access*: press **Resume** (top right).
4. To turn off *Guided Access*: press **End** (top left).
5. *Lock Menu* (restrict use of specific buttons/categories): open *Guided Access* > use finger to draw a circle around buttons to restrict > **Resume**.

Note: If these instructions do not match your current vocabulary set, please contact Technical Support for assistance.

**Some Use key functions are enabled for -PLUS models only. Contact us for details.*

Need help?

877.392.2299 Option 3

support@talktometechnologies.com

talktometechnologies.com



We bring speech to life™

Hardware, Buttons & Features



Power

Press and hold this button to turn the device on or off.

Note: Press this button one time while thewego A is on to put it into "Sleep" mode. This feature will conserve battery life when thewego A is not in use.

Volume

Press the volume toggle up or down to adjust volume levels.

Home

Tap to return to the home screen.

Lock screen

You can turn on Lock Screen to lock the screen orientation. Switch button to left to unlock and to right to lock screen orientation.

Quick Keys

Thewegowrite touchscreen and keyboard features several quick keys: delete, speak, phrases, and emotes. The touchscreen also features the Use button containing options on how to share your message.

Charging thewegowrite



Charge device, TTMT Speaker (SPKR), and keyboard at night or when not in use. Please only use TTMT-approved charging cables to charge your device.

Keyboard power

Press the keyboard power button once to turn off and on. To conserve battery life, turn off the keyboard when not using with thewegowrite.

Detach the magnetic keyboard (if applicable)

If your model's keyboard is detachable, place one hand on the keyboard, then grasp the top of the device with your other hand and gently pull away.

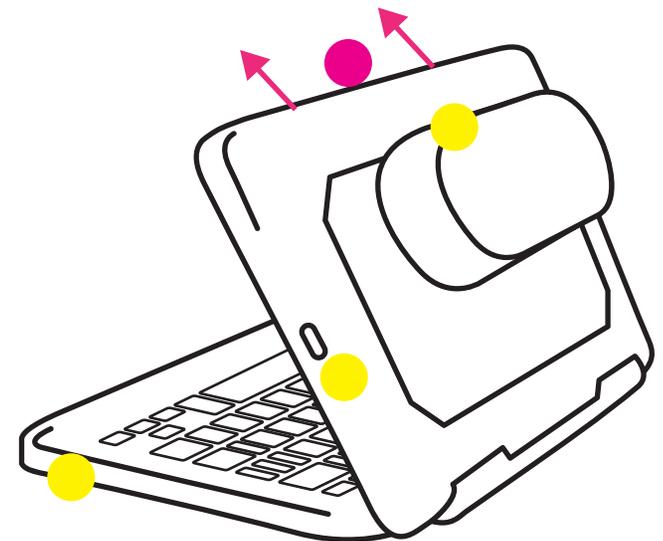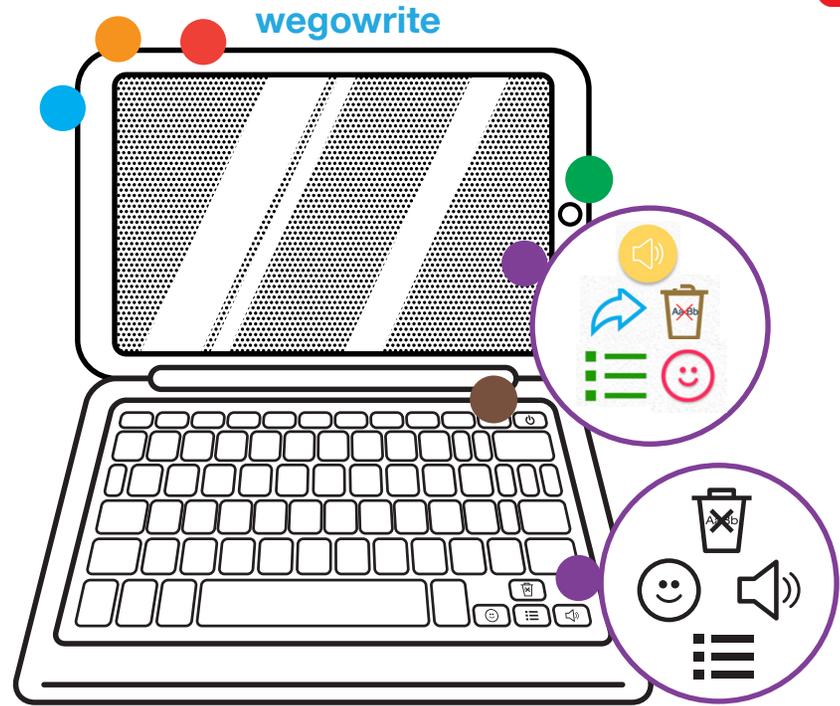

Need help?

 877.392.2299 Option 3

 support@talktometechnologies.com

 talktometechnologies.com