

Fygg™ – Feed Your Good Guys™

15.5% nano XIM at 57.34% concentration (8.6% nano-HAp)

Fygg

Feed Your Good Guys

Directions After Varnish Application

Your dental team is revolutionizing oral care by supporting your dental health and oral microbiome with Fygg's proprietary 8.6% nano-hydroxyapatite varnish which is currently the only form of nano-hydroxyapatite (nanoXIM.CarePaste manufactured by Fluidinova) approved by the SCCS in Europe for safety and efficacy.

This treatment can aid in remineralizing teeth, improving sensitivity, and whitening smiles. It also has added prebiotics and L-arginine to help support and nourish your oral microbiome. To obtain optimal results and efficacy, we suggest the following after your varnish application:

- Nothing to eat or drink for 30 minutes.

- Avoid hot beverages and alcohol, including antiseptic mouth rinses (which we suggest to avoid all of the time!) for the remainder of the day.

- Do not brush or floss your teeth for three hours.

You may feel a thin coating of varnish on your teeth immediately after. Still, you will not experience the unpleasant after-taste and texture of the sticky fluoride varnish you are likely used to. Our varnish is free of carrier agents like hexanes and the propylene glycol derivatives that many traditional varnish use, making it safe to swallow and without the need to brush it off later.

The power of our varnish lies in its ability to nourish your saliva, boosting its natural remineralization qualities and filling in microporosities and demineralization areas directly on your tooth surfaces.

Fygg™ - Feed Your Good Guys™ 5331 S MacAdam Ave Ste 258 PMB #723 Portland, OR 97239

Fygg™ - Feed Your Good Guys™ are trademarks of Oral Microbiome Solutions, LLC.

Used under license in the United States. © Oral Microbiome Solutions, LLC 2023. All Rights Reserved.

fygg.com

