


Femmely Ultra

Instruction manual

Package Contents

One massage warm belt,
One cable, One manual

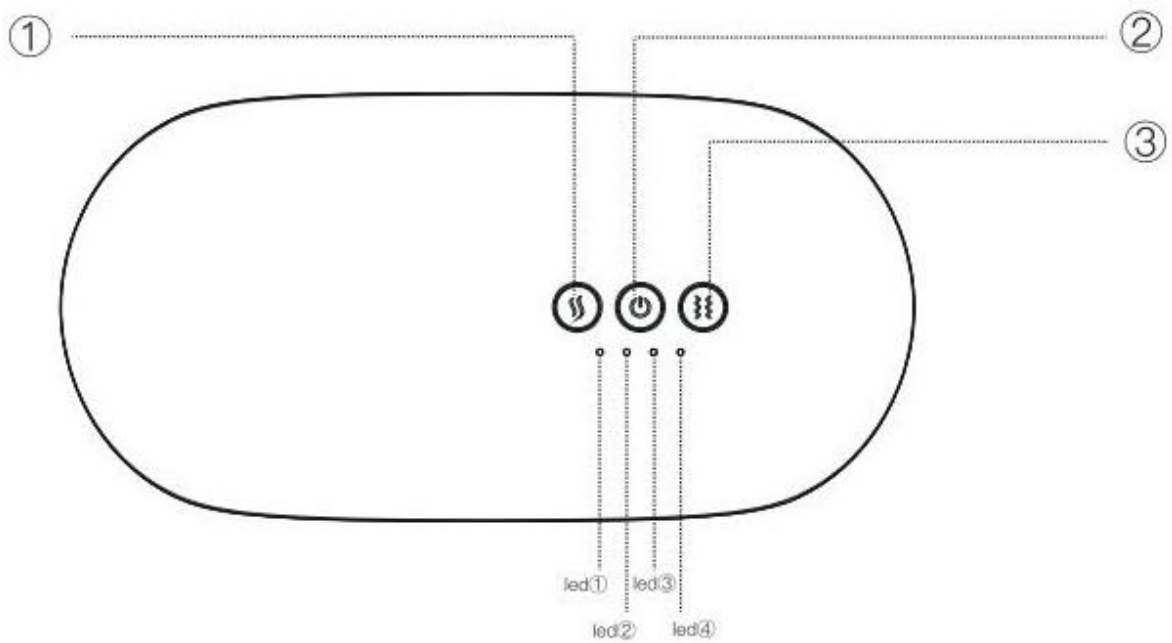


Component introduction

① Temperature control button

② Switch button

③ Vibration button

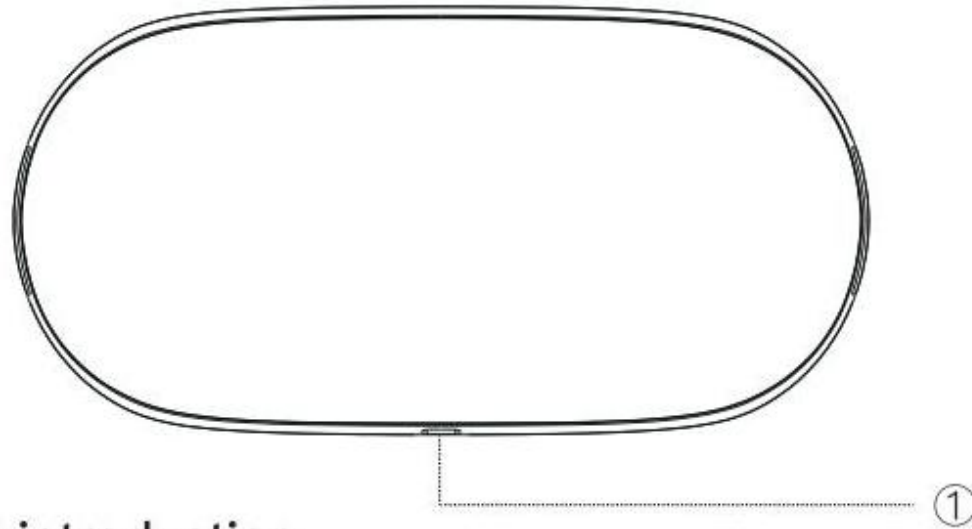

Component introduction

①Type – C charging interface

Wearing instructions

Unlock the elastic belt buckle, around the waist buckle, moderately regulate personal waistline.

Charging instructions

1. Plug the charging cable into the Type–c charging port of the warm belt to charge. When charging, the white indicator blinks successively, and when the charging is complete, the white indicator lights on.
- 2.This product does not support to use when charging, and the product will not be able to turn on when charging.

Instructions for use

- 1.Please charge the product fully before using it for the first time.
- 2.Long press the "On/Off button" for 2 seconds, then the white power indicator and the yellow atmosphere light in the window will turn on and enter the startup state.

3. In the startup state of heating mode:

Short press the "temperature control button" once, the power indicator led① lights up yellow for 3 seconds, and enters the first gear hot compress mode 37°C;

Short press again, the power indicator led①② light yellow for 3 seconds, into the second gear hot compress mode 45°C;

Press it again, and the power indicator led①②③ lights up the yellow light for 3 seconds and enters the third gear hot compress mode 55°C;

Press it again, and the power indicator led①②③④ lights up the yellow light for 3 seconds and enters the fourth gear hot compress mode 65°C;

Short press once again, power indicator led①②③④ off for 3 seconds, stop the heating function.

Cycle the operation in turn. Long press the temperature control button at any gear to stop the heating function.

4. Vibration mode under startup state:

Press "vibration button" once, keep the power indicator ① on for 3 seconds, and enter the first gear scraping mode;

Press it again, keep the power indicator ①② on for 3 seconds, and enter the second gear tap mode;

Press again, keep the power indicator ①②③ on for 3 seconds, enter the 3 gear press mode;

Press again, the power indicator led①②③④ blinks for 3 seconds, enter the 4 gear combination mode;

Press again, the power indicator led ①②③④ blinks for 3 seconds, enter the 5 gear kneading mode;

Short press once again, power indicator led①②③④ off for 3 seconds, stop the vibration function.

Cycle the operation in turn. Long press vibration button at any gear to stop the vibration function.

5.In the startup state:

- Long press the "On/Off" button for 2 seconds in any mode to enter the shutdown state.
- 6.When the heating belt is with low voltage, the white power indicator flashes for 10 seconds and then shuts down.
- 7.When the heating belt is powered on with low voltage, it cannot be powered on after the white power indicator blinks for 10s.

Matters needing attention

- 1.Please read carefully before using this product and understand all the contents of the manual before operation.
- 2.It is prohibited for pregnant women, children, patients with heart disease, abnormal blood pressure, malignant tumors, cerebrovascular diseases, patients with acute diseases or others under the treatment of doctors.
- 3.It is prohibited for those who do not have the ability to express consciousness.
- 4.Do not use it for sensitive skin.
- 5.Do not use it for patients with skin perception disorder or those who are not sensitive to heat.
- 6.It is forbidden to use vibration function during menstruation, which may cause increased menstrual volume. It is recommended to only use heating mode.
- 7.If you feel uncomfortable or abnormal skin during use, please stop using immediately and consult a doctor.
- 8.It is prohibited to use this product in places of high heat, inflammable, electromagnetic radiation and damp.
- 9.It is forbidden to disassemble and assemble this product without permission to avoid collision and water, so as to avoid failure.
- 10.Do not use water or other detergents to clean the products to avoid water failure.

Product specification

Product model	WL – 1009A
Input voltage	5V \equiv 2A
Input interface	Type-c
Temperature	37 ~ 65°C
Battery capacity	3.7 V \equiv 2000mAh (9.25Wh)
Charging time	\approx 2H
Product material	ABS+flannelette
Product weight	256g
Product size	190*95.2*52.8mm