


## **DSD 867/DSD 8107/DSD 8127 Construction - Saliva Drug Test Instructions**

Allow the test kit to reach room temperature (15-30°C) prior to testing. Instruct the donor to not place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.

1. Remove the collector from the sealed pouch and insert the sponge end of the collector into the mouth. Actively swab the inside of the mouth and tongue to collect oral fluid for a total of 5 minutes until the sponge becomes fully saturated. Gentle pressing the sponge between the tongue and teeth will assist saturation. No hard spots should be felt on the sponge when saturated. It is vital to fully saturate the sponge, if inadequate saliva is collected not all tests will run.
2. Remove the collector from the mouth. Place saturated oral fluid collector into test tube and screw the collector to press sponge fully to collect oral fluid.

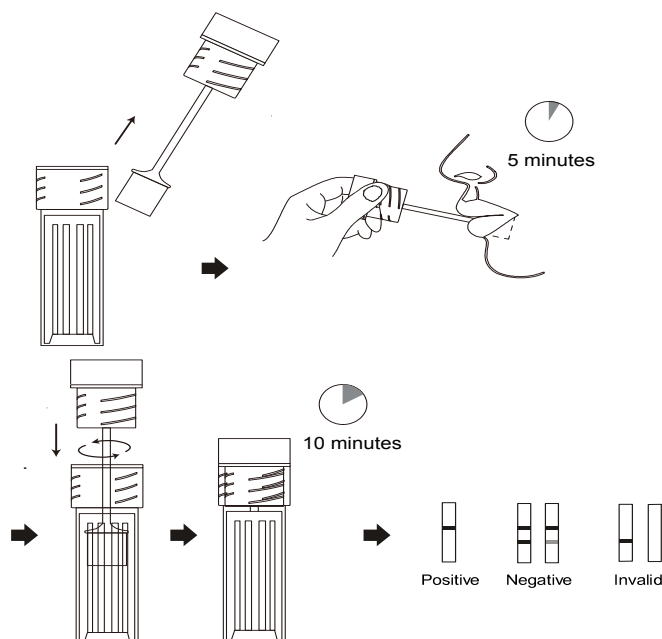

3. Place the test upright on a clean and level surface. Remove the peel off label, Wait for the coloured line(s) to appear. Read results at 10 minutes. Do not read results after 1 hour. See illustration below.

### **READING RESULTS** (Please refer to the above illustration)

**NEGATIVE:**\* Two lines appear. One coloured line should be in the control region (C), and another apparent coloured line adjacent should be in the test region (Drug/T). This negative result indicates that the drug concentration is below the detectable level.\*NOTE: The shade of colour in the test line region (Drug/T) will vary, but it should be considered negative whenever there is even a very faint line.

**POSITIVE:** One coloured line appears in the control region (C). No line appears in the test region (Drug/T). This positive result indicates that the drug concentration is above the detectable level.

**INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test panel. If the problem persists contact [customerservices@accessdiagnostics.co.uk](mailto:customerservices@accessdiagnostics.co.uk).

### **Drug test codes (please note not all may be included in the option purchased).**

Amphetamine (AMP 50ng) Methamphetamine (MET 50ng) Marijuana/Cannabis (THC 12ng) Phencyclidine (PCP 10ng) Cocaine (COC 20ng) Benzodiazepines (BZO 50ng) Opiates (OPI 40ng) Methadone (MTD 30ng) Methaqualone (MQL 300ng) Ketamine (KET 50ng) Barbituates (BAR 50ng) Propoxythene (PPX 50ng) Ecstasy (MDMA 50ng) Tramadol (TML 30ng) Buprenorphine (BUP 10ng) EDDP (50ng)

For further advice or guidance please see: [ukdrugtesting.co.uk](http://ukdrugtesting.co.uk) or Tel: 01263 731168