

ACCESS DIAGNOSTIC TESTS UK

www.ukdrugtesting.co.uk

Thank you for purchasing a rapid drug test kit. Please read the following advice before proceeding with your test.


Do not open the foil wrapped kit until the time you want to proceed with the test. To perform a test you will need a fresh urine sample in a container at least as wide as the test device, this should be cooled to room temperature, (approx 20 minutes is adequate)

MULTI - PANEL TEST DEVICE INSTRUCTIONS : CHECK BOTH SIDES

- Ensure that the urine sample and test devices are at room temperature.
- Remove the device from the foil.
- Remove the cover by grasping each end of the cartridge. Pull to reveal 3-10 test strips
- Insert the end of the test strips into the urine to just below half their length. Do not exceed the maximum urine level indicator (triangles). The sample should not touch the cartridge. approx 1cm depth is a good guide
- Dip for 10 to 15 seconds.
- Read results at five minutes (Do not read after more than 10 minutes.)

READ THE RESULTS

- Each test kit has a label for the Test line and a Control line. These are marked T, and C. before the test is performed no lines show in the result area. After the test is done the control line should show in all tests as a coloured line. The intensity of the test line may be less than the control line. Where the control line appears, any line in the test line area indicates a negative test. In positive tests the control line will show but no line will appear in the test zone. If no lines appear after 10 minutes the test is invalid. If only the test line appears with no control line the test is invalid (this is the in-built quality control)

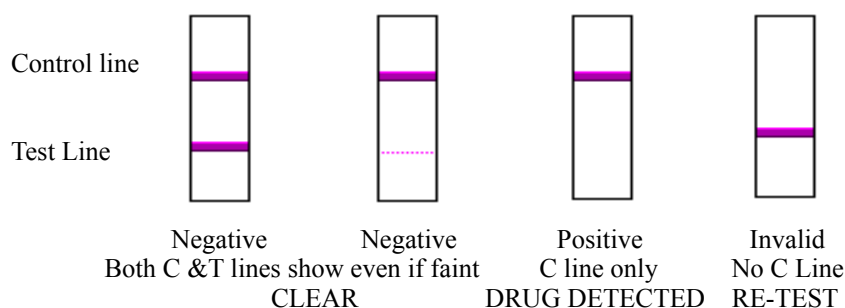

Negative tests can be read at as early as 1-2 minutes. Always allow a full five minutes development time for positive results. The line intensity may vary, Any line in the test zone should be read as a negative result, even if very faint. You should confirm all positive tests. Prescribed medication may give positive results (e.g. opiate/codeine pain killers) Do not read beyond 10 minutes.

ABP10/UR-144/JWH50 = Synthetic Cannabis AMP = Amphetamine BAR Barbiturates BUP = Buprenorphine BZO = Benzodiazepines COC = Cocaine (&Crack) COT = Cotinine(Nicotine) FYL = Fentanyl GAB = Gabapentin K2/K3 =synthetic cannabis KET = Ketamine MCAT = Methcathinine MDMA= Ecstasy MDPV = (Bath Salts) MEP = Mephedrone MET = Methamphetamine MOP/R = Heroin (& opiates) MTD = Methadone OXY =Oxycodone PCP =PCP PGB = Pregabalin PPX = Propoxyphene PVP = alpha PVP TCA = Tri-Cyclic Antidepressants THC = Cannabis (marijuana, THC) TML = Tramadol

FOR ADVICE ON RESULTS CALL 01263 731168 MON- FRI 8.30am – 5pm