

DEO
DENTAL

Multi-purpose heater for Composite Compules



(No products Included)



Product Code: DD129

**Composite Compules
to be used in the
Multi-purpose heater**

Product Code: DD030404



ISO 9001 Certified Organisation

Website: www.deo-dental.com | Email: hello@deo-dental.com

Follow us on Instagram [@deodental.uk](https://www.instagram.com/deodental.uk) | Youtube [@deodental](https://www.youtube.com/deodental)

Multi-purpose heater for Composite Compules

Instructions to use Composites, LA and Endodontic Solution Module

- Place the module in the drop in chamber and place in the heater
- Switch on the heater and select P4 - 100°C
- This programme will display 100 (max set temperature) and within a few seconds it will start flashing
- P4 and 100 (This is just a reminder of the programme selected)
- **Using the minus sign reduce the display temperature for the material of choice to be used**
- Place material to be used in the appropriate opening. Compules and local anaesthetics on the under side and syringes into the top openings slot for 10 -15 minutes before the start of the treatment
- P4 and the new selected temperature will continue to flash until the New temperature is reached
- After 10 minutes the display will show the New selected temperature

• **CLEANING**

Washer / Disinfector Machine:

90-95°C for a minimum of 1 minute with a PH of 105

Autoclaving: 134-137°C, 2.1-2.25 bar gauge for at least 3 minutes

The silicone ring for the aluminium chamber can be put through a Washer / Disinfector machine and an autoclave

Recommended Temperature:

- 40°C - LA
- 75°C - Composite
- 60°C - Endodontics Solutions
- The temperature can be changed by pressing the Plus or Minus on the control panel
- From time to time the adjusted set temperature will flash
- This indicates that the heater is maintaining the display temperature

Please visit the website for further information | Design Registration Number: 6352352

Terms & Conditions Apply

Website: www.deo-dental.com | Email: hello@deo-dental.com

Follow us on Instagram [@deodental.uk](https://www.instagram.com/deodental.uk) | Youtube [@deodental](https://www.youtube.com/deodental)



Scan to visit our website