REF G-36600, G-36601, G-36900, G-36901, G-38210, G-38215, G-38240, G-40006, G-40008

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HANDLES FOR SEPARABLE TUBE GUIDED INSTRUMENTS

PRODUCT RELATED PROCESSING INSTRUCTIONS

These product related processing instructions for semi-critical* medical devices (MD) are exclusively valid for the above listed items by GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

Fig. 1 example illustration, not to scale

ACCESSORIES

Product list together with accessories and tools		Accessories																							
		Forceps										Scissors													
		G-36680	G-36690	G-36695	G-36710	G-36755	G-36765	G-28220	G-38222	G-38242	G-38244	G-38247	Single-use forceps Uno Colorline	G-36902	G-36915	G-36925	G-36930	G-36955	G-36985	G-38232	G-38236	G-38238	G-38246	Single-use scissors Uno Colorline	
	G-36600, blue, 6 mm	x	x	x	x	x	x																		
	G-36601, blue, 8 mm	x	x	x	x	x	x																		
	G-36900, gold, 6 mm														x	x	x	x	x	x					
	G-36901, gold, 8mm														x	x	x	x	x	x					
	G-38210, blue							x	x																
	G-38215, gold																			x	x	x			
	G-38240									x	x	x												x	
	G-40006, 6 mm																								x
	G-40008, 8 mm																								x

Tab. 1: Accessories

OVERVIEW

Cycle limit	Assembly/Disassembly	Accessories
NO	YES	Separable tube guided instruments, see list of accessories (Tab. 1)
Pre-cleaning	Ultrasound	Automated processing
YES	YES	YES
Manual processing	Compressed air	Steam sterilization
YES	YES	YES

Tab. 2: Overview

AUXILIARY EQUIPMENT

First treatment	Soft tissue (e.g. compress), single-use syringe (5 times with 10 ml each)
Pre-cleaning	Soft plastic brush or steamer (steam jet, max. 2 bar)
Manual cleaning and disinfection	single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar) optionally: ultrasound bath 35-40 kHz
Drying	Medical compressed air (oil-free compressed air, max. 2 bar)
Maintenance	Instrument oil apt for sterilization, lint-free wipes

Tab. 3: Auxiliary Equipment

ASSEMBLY/DISASSEMBLY:

- Hold tip/UNO COLORLINE Single-Use Tip and handle as shown in fig. 2 without exerting any pressure on the mechanism. Align pin on handle with arrow on tip.
- Push handle all the way into tip, sliding pin into groove.

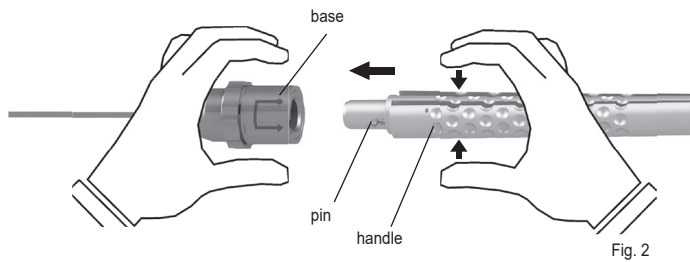
Fig. 2

Handles G-38600, G-38601, G-38610; 36900, G-36901, G-38215:

- Turn handle clockwise all the way to the end (fig. 3).

Fig. 3

Handle G-38240:

- Turn the handle clockwise until it engages perceptibly (fig. 4).

Fig. 4

Handle G-40006 und G-40008 (Single-use Scissors/Forceps Uno Colorline):

- Turn the handle clockwise until it engages perceptibly (Fig. 5).

Fig. 5

FIRST TREATMENT AT PLACE OF USE

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/immediately after application with a compress soaked in sterile water.
- Assemble the protecting cap according to fig. 6a und 6b.
- Place dry handle into a suitable storage or transport system and close it.

PREPARATION BEFORE CLEANING

Pre-cleaning

- Remove the tip / UNO COLORLINE single-use tip in reverse order as shown in fig. 2-5.
- Reusable tips must be reprocessed following their product-related processing instructions (1102017). Single-use tips must be disposed of.
- Rinse the handle under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.

OR

Ultrasound cleaning as supporting pre-cleaning (optional)

Basically, we recommend pre-cleaning with ultrasound all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the handle and make sure that it is completely moistened:
 - Insert joint handle in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the handle afterwards with demineralized water for neutralization:
 - Operate the handle when doing so.

* Classification recommended by GEUDER AG, the responsibility lies with the operator.

MANUAL CLEANING AND DISINFECTION IN ULTRASOUND BATH

Manual cleaning in ultrasound bath

- Introduce the handle and make sure that it is completely moistened.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the handle into demineralized water for at least 2 minutes for neutralization:
 - Operate handle when doing so (at least 10 times).
 - Remove residues of debris with a soft plastic brush (without pressure).
- Rinse handle under running demineralized water.

Manual disinfection in ultrasound bath

- Introduce the handle and make sure that it is completely moistened.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the handle into demineralized water for at least 2 minutes for neutralization:
 - Operate handle when doing so (at least 10 times).
- Rinse handle under running demineralized water.

AUTOMATIC CLEANING AND DISINFECTION

As additional load only ophthalmic products are permitted.

Thermic disinfection A_0 -value > 3000, 93 °C, 5 min; VARIO TD program.

The selection of suitable racks, charging trolleys for the optimal rinsing and flushing of the handles should be done after consulting with the cleaning device manufacturer.

Process

- Place the handle in the rack (e.g. sieve baskets, trays):
 - Avoid rinsing shadows.

DRYING

Dry the handles with medical compressed air, if necessary.

CONTROL AND MAINTENANCE

Visual control

- Check if the handle is clean:
 - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the handle is damaged:
 - Remove corroded or damaged handles.
- Check if the handle is complete.
- Check if the handle is dry.

Maintenance

- Care for the handle with a suitable instrument oil:
 - Let the handle cool down.
 - Apply the instrument oil selectively on the area that needs it (fig. 6).
 - Remove excess instrument oil, if any, with a lint-free wipe.
- Carry out a functional control:
 - Check if the handle is complete.
 - Operate movable parts, check if the moveable parts can be moved easily.

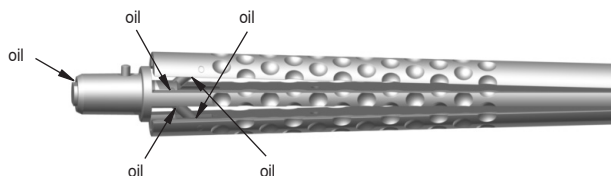
Fig. 6

PACKAGING

Use packaging materials and systems complying with the following standards:
DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

STERILIZATION

Steam sterilization in fractionated prevacuum method:
SAL 10⁻⁶, at least 3 min at 134 °C, 3 bar.

STORAGE

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

- Damaged or open sterile barrier systems (SBS) are deemed non-sterile.
- The storage time depends on the type of packaging and must be validated accordingly.

TRANSPORT

Transport must not have a negative effect on the characteristics of the sterile goods.

Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

DISPOSAL

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute (www.rki.de).

National standards, regulations and/or restrictions must also be considered!

GENERAL INFORMATION FOR PROCESSING

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

DETAILS OF VALIDATION

Summary of processes validated by GEUDER AG:

The selection of the parameters refers to worst case conditions.

Manual pre-cleaning, cleaning and disinfection

Pre-cleaning

Using tap water (according TrinkwV), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

Cleaning

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H, after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

Disinfection

- 3.0 %, 15 min/20 °C, immersion bath, lumen were filled with disinfectant agent. After disinfection time rinse for 1 min with demineralized water.

Process chemicals (manual):

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.
- Disinfection: Bode Chemie: Korsolex plus, demineralized water.

Automatic cleaning and disinfection

Cleaning

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Disinfection

- Thermal high level disinfection A_0 -value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection A_0 -value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Process chemicals (automatic)

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

Steam sterilization

- Program: half-cycle SAL10⁻⁶, fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature.
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective processing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Actual version available at GEUDER AG.

Subject to changes.