REF	G-15137	G-17501	G-17525	G-17575	G-18931	G-19545	G-19605	G-19695	G-19737	G-19765	G-19790	G-32107	G-32330
	G-17440	G-17505	G-17530	G-17580	G-18932	G-19550	G-19607	G-19710	G-19738	G-19770	G-31994	G-32250	G-32823
	G-17441	G-17506	G-17535	G-17585	G-18933	G-19570	G-19610	G-19715	G-19740	G-19775	G-31998	G-32260	G-32928
	G-17450	G-17510	G-17540	G-18720	G-18935	G-19575	G-19615	G-19718	G-19745	G-19776	G-32023	G-32261	G-32929
	G-17451	G-17511	G-17545	G-18725	G-18937	G-19580	G-19620	G-19720	G-19747	G-19777	G-32100	G-32262	G-32945
	G-17452	G-17515	G-17560	G-18730	G-18939	G-19585	G-19630	G-19725	G-19750	G-19778	G-32101	G-32320	G-32946
	G-17453	G-17516	G-17565	G-18849	G-18942	G-19590	G-19632	G-19727	G-19751	G-19780	G-32103	G-32321	G-19718S
	G-17500	G-17520	G-17570	G-18930	G-19540	G-19600	G-19690	G-19728	G-19760	G-19781	G-32105	G-32325	

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INSTRUMENTS WITH SPRING HANDLE
PRODUCT RELATED PROCESSING INSTRUCTIONS

These product related processing instructions for critical* medical devices (MD) are exclusively valid for the above listed item numbers of GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

Fig. 1 example illustrations, not to scale

OVERVIEW

Cycle limit	Assembly/Disassembly	Accessories
NO	YES	NO
Pre-cleaning	Ultrasound	Automated processing
YES	YES	YES
Manual processing	Compressed air	Steam sterilization
NO	YES	YES

Tab. 1: Overview

AUXILIARY EQUIPMENT

First treatment	Soft tissue (e.g. compress), single-use syringe (5 times with 10 ml each)
Pre-cleaning	Soft plastic brush or steamer (steam jet, max. 2 bar) single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar) optionally: ultrasound bath 35-40 kHz
Drying	Medical compressed air (oil-free compressed air, max. 2 bar)
Checking pH value	pH test strips
Maintenance	Instrument oil apt for sterilization, lint-free wipes

Tab. 1: Auxiliary equipment

ASSEMBLY/DISASSEMBLY:

Lock

- Press the handle parts until the lock engages perceptibly (see fig. 2).
- Press the handle parts further, until the lock opens.

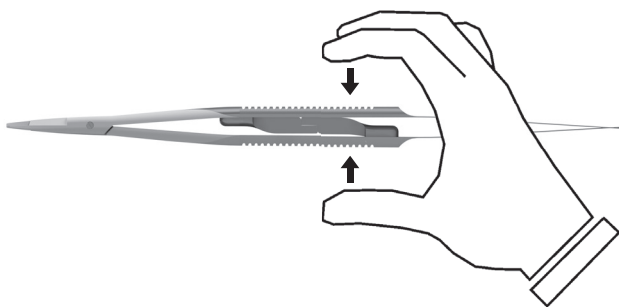
Fig. 2

Spring handle:

- Grasp the instrument with spring handle at both branches of the handle.
- Open the spring of the spring handle by slightly turning the branch and pulling it through the opening (see fig. 3).

Fig. 3

FIRST TREATMENT AT PLACE OF USE

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/ immediately after application with a compress soaked in sterile water.
- Place dry instrument with spring handle into a suitable storage or transport system and close it.

PREPARATION BEFORE CLEANING

Pre-cleaning

- Open the instrument with spring handle as shown in fig 2 and 3.
- Rinse the instrument with spring handle under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.

OR

Ultrasound cleaning as supporting pre-cleaning (optional)

Basically, we recommend pre-cleaning with ultrasound all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the instrument with spring handle and make sure that they are completely moistened:
 - Insert joint instrument in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the instrument with spring handle afterwards with demineralized water for neutralization:
 - Move the instrument with spring handle when doing so.

AUTOMATIC CLEANING AND DISINFECTION

As additional load only ophthalmic products are permitted.

Thermic disinfection A₀ value > 3000, 93 °C, 5 min; VARIO TD program.

The selection of suitable racks, charging trolleys for the optimal rinsing and flushing of the medical devices should be done after consulting with the cleaning device manufacturer.

Process

- Place the instrument with spring handle in the rack (e.g. sieve baskets, trays):
 - Open the joint instruments with spring handle (max. 90°).
 - Avoid rinsing shadows.

DRYING

Dry the instrument with spring handle with medical compressed air, if necessary.

Control of pH value

For ophthalmic instruments a batch-related routine control for alkaline residues is recommended.

- Apply the remaining humidity of the instrument with spring handle onto a pH test strip.
- Compare the pH test strip with the color scale:
 - pH must be neutral.

* Classification recommended by GEUDER AG, the responsibility lies with the operator.

CONTROL AND MAINTENANCE

Visual control

- Check if the instrument with spring handle is clean:
 - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the instrument with spring handle is damaged:
 - Remove corroded or damaged instruments with spring handle.
- Check if the instrument with spring handle is complete.
- Check if the instrument with spring handle is dry.

Maintenance

- Care for instrument with spring handle with a suitable instrument oil:
 - let the instrument with spring handle cool down.
 - Apply the instrument oil selectively on the area that needs it (see fig. 4).
 - Remove excess instrument oil, if any, with a lint-free wipe.
- Carry out a functional control:
 - Check if the instrument with spring handle is complete.
 - Assemble open instrument with spring handle (see fig. 3, in reverse order).
 - Check if the moveable parts can be moved easily.
 - Check if the lock mechanisms work (see fig. 2).

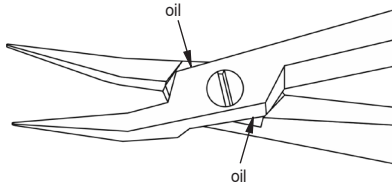
Fig. 4

PACKAGING

Use packaging materials and systems complying with the following standards:
DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

STERILIZATION

Steam sterilization in fractionated prevacuum method:
SAL 10⁻⁶, at least 3 min at 134 °C, 3 bar.

STORAGE

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

- Damaged or open sterile barrier systems (SBS) are deemed non-sterile.
- The storage time depends on the type of packaging and must be validated accordingly.

TRANSPORT

Transport must not have a negative effect on the characteristics of the sterile goods.

Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

DISPOSAL

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute (www.rki.de).

National standards, regulations and/or restrictions must also be considered!

GENERAL INFORMATION FOR PROCESSING

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

DETAILS OF VALIDATION

Summary of processes validated by GEUDER AG:

The selection of the parameters refers to worst case conditions.

Manual pre-cleaning

Pre-cleaning

Using tap water (according TrinkwV [German Drinking Water Ordinance]), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

Cleaning

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H, after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

Process chemicals (manual)

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.

Automatic cleaning and disinfection

Cleaning

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Disinfection

- Thermal high level disinfection A₀-value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection A₀-value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Process chemicals (automatic)

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

Steam sterilization

- Program: half-cycle SAL 10⁻⁶, fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: type Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature (max. 25 °C).
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective processing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Actual version available at GEUDER AG.

Subject to changes.