REF G-28764

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REF G-28764

ADAPTER, with protective cover for glass syringes
PRODUCT RELATED PROCESSING INSTRUCTIONS

These product related processing instructions for semi-critical* medical devices (MD) are exclusively valid for the above listed items by GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

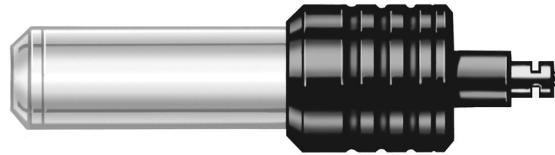
Fig. 1: example illustration, not to scale

OVERVIEW

Cycle limit	Assembly/Disassembly	Accessories
NO	YES	NO
Pre-cleaning	Ultrasound	Automated processing
YES	YES	YES
Manual processing	Compressed air	Steam sterilization
NO	YES	YES

Tab. 1: Overview

AUXILIARY EQUIPMENT

First treatment	Soft tissue (e.g. compress), single-use syringe (5 times with 10 ml each)
Pre-cleaning	Soft plastic brush or steamer (steam jet, max. 2 bar), single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar)
Manual cleaning and disinfection	optionally: ultrasound bath 35-40 kHz
Drying	Medical compressed air (oil-free compressed air, max. 2 bar)
Checking pH value	pH test strips

Tab. 2: Auxiliary Equipment

ASSEMBLY/DISASSEMBLY

- Screw the screw plug onto the protective cover until the yellow marking ring is no longer visible (fig. 2).
- The screw plug and the protective cover are provided with an endless thread.

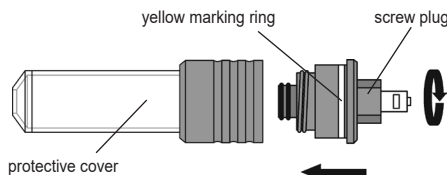
Fig. 2

FIRST TREATMENT AT PLACE OF USE

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/immediately after application with a compress soaked in sterile water.
- Place the dry adapter with protective cover for glass syringes into a suitable storage or transport system and close it.

PREPARATION BEFORE CLEANING

- If present, remove the glass syringe from the adapter with protective cover for glass syringes according to fig. 2 in reverse order:
 - The glass syringe is to be disposed of.
 - The adapter with protective cover for glass syringes, consisting of the adapter parts (screw plug and protective cover) is processed in disassembled state (cleaning, disinfection and sterilization).

Precleaning

- Rinse the adapter parts under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.
- Rinse the adapter parts with a cleaning gun with a suitable flushing adapter or a single-use syringe.

OR

Ultrasound cleaning as supporting pre-cleaning (optional)

Basically, we recommend pre-cleaning with ultrasound for all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the adapter parts and make sure that they are completely moistened:
 - Fill the adapter parts with cleaning agent.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the adapter parts afterwards with demineralized water for neutralization.
- Check the adapter parts for unobstructed flow:
 - Rinse the adapter parts with a cleaning gun, a steamer or a single-use syringe.

MANUAL CLEANING AND DISINFECTION IN ULTRASOUND BATH

Manual cleaning in ultrasound bath

- Introduce the adapter parts and make sure that they are completely moistened:
 - Fill the adapter parts with cleaning agent.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the adapter parts into demineralized water for at least 2 minutes for neutralization:
 - Remove residues of debris with a soft plastic brush (without pressure).
 - Rinse the adapter parts 5 times with at least 10 ml of demineralized water.
- Rinse the adapter parts under running demineralized water.
- Check the adapter parts for unobstructed flow and rinse them with a cleaning gun or a single-use syringe.

Manual disinfection in ultrasound bath

- Introduce the adapter parts and make sure that they are completely moistened:
 - Fill the adapter parts with disinfection agent.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the adapter parts into demineralized water for at least 2 minutes for neutralization:
 - Rinse the adapter parts 5 times with at least 10 ml of demineralized water.
- Rinse the adapter parts under running demineralized water..
- Check the adapter parts for unobstructed flow and rinse them with a cleaning gun or a single-use syringe.

AUTOMATIC CLEANING AND DISINFECTION

As additional load only ophthalmic products are permitted. Thermic disinfection A₀ value > 3000, 93 °C, 5 min; VARIO TD program. The selection of suitable racks, charging trolleys for optimal rinsing and flushing of the medical devices should be done after consulting with the cleaning device manufacturer.

Process

- Connect the Luer connector of the screw plug to the charging trolley:
 - Avoid rinsing shadows.
- Attach the open end of the protective cover to a suitable injector nozzle (fig. 3).

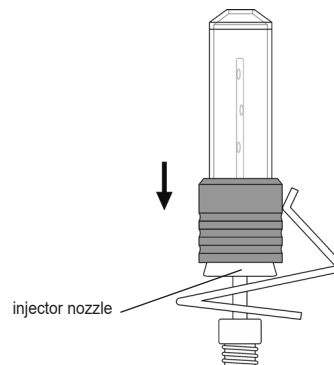
Fig. 3

DRYING

Dry the adapter parts with medical compressed air, if necessary.

Control of pH value

- For ophthalmic instruments a batch-related routine control for alkaline residues is recommended.
 - Apply the remaining humidity of the adapter parts onto a pH test strip.
- Compare the pH test strip with the color scale:
 - pH must be neutral.

CONTROL AND MAINTENANCE

Visual control

- Check if the adapter parts are clean:
 - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the adapter parts are damaged:
 - Remove corroded or damaged adapter parts.
- Check if the adapter parts are complete.
- Check if the adapter parts are dry.

Maintenance

- Carry out a functional control:
 - Check if the adapter parts are complete.

* Classification recommended by GEUDER AG, the responsibility lies with the operator.

PACKAGING

Use packaging materials and systems complying with the following standards:
DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

STERILIZATION

Steam sterilization in fractionated prevacuum method:
SAL 10⁻⁶, at least 3 min at 134 °C, 3 bar.

STORAGE

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

- Damaged or open sterile barrier systems (SBS) are deemed non-sterile.
- The storage time depends on the type of packaging and must be validated accordingly.

TRANSPORT

Transport must not have a negative effect on the characteristics of the sterile goods.
Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

DISPOSAL

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute (www.rki.de).

National standards, regulations and/or restrictions must also be considered!

GENERAL INFORMATION FOR PROCESSING

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

DETAILS OF VALIDATION

Summary of processes validated by GEUDER AG:

The selection of the parameters refers to worst case conditions.

Manual pre-cleaning, cleaning and disinfection

Pre-cleaning

Using tap water (according TrinkwV [German Drinking Water Ordinance]), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

Cleaning

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H, after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

Disinfection

- 3.0 %, 15 min/20 °C, immersion bath, lumen were filled with disinfectant agent. After disinfection time rinse for 1 min with demineralized water.

Process chemicals (manual)

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.
- Disinfection: Bode Chemie: Korsolex plus, demineralized water.

Automatic cleaning and disinfection

Cleaning

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836 CD, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Disinfection

- Thermal high level disinfection A₀-value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836 CD, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection A₀-value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836 CD, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

Process chemicals (automatic)

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

Steam sterilization

- Program: half-cycle SAL 10⁻⁶, fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: type Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature (max. 25 °C).
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective reprocessing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Actual version available at GEUDER AG.

Subject to changes.