**REF** G-15961, G-15962, G-15968, G-15969, G-17005, G-17019, G-17021, G-17024, G-17029, G-17031, G-33914, G-33920, G-33922, G-33925

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**EYE SPECULUMS with aspiration**

**PRODUCT RELATED PROCESSING INSTRUCTIONS**

These product related processing instructions for critical\* medical devices (MD) are exclusively valid for the above listed items by GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

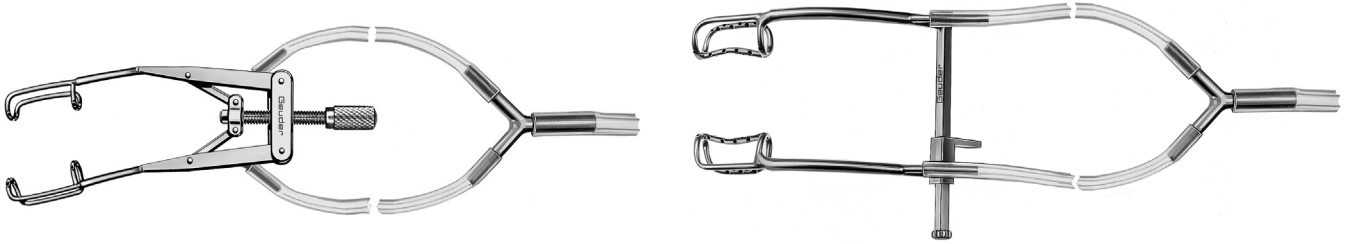
Fig. 1 example illustrations, not to scale

**ACCESSORIES**

G-31024	CLEANING ADAPTER of the Eye Speculum with aspiration	
G-33919	SPARE TUBING SET for eye speculum with aspiration	

Tab. 1: Accessories

**OVERVIEW**

<b>Cycle limit</b>	<b>Assembly/Disassembly</b>	<b>Accessories</b>
NO	YES	YES
<b>Pre-cleaning</b>	<b>Ultrasound</b>	<b>Automated processing</b>
YES	YES	YES
<b>Manual processing</b>	<b>Compressed air</b>	<b>Steam sterilization</b>
NO	YES	YES

Tab. 2: Overview

**AUXILIARY EQUIPMENT**

<b>First treatment</b>	Soft tissue (e.g. compress), single-use syringe (5 times with 10 ml each)
<b>Pre-cleaning</b>	Soft plastic brush or steamer (steam jet, max. 2 bar), single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar) optionally: ultrasound bath 35-40 kHz
<b>Drying</b>	Medical compressed air (oil-free compressed air, max. 2 bar)
<b>Checking pH value</b>	pH test strips
<b>Maintenance</b>	Instrument oil apt for sterilization, lint-free wipes

Tab. 3: Auxiliary Equipment

**ASSEMBLY/DISASSEMBLY**

**Tubing system**

- Remove the tubing system according to fig. 2 or fig. 3.

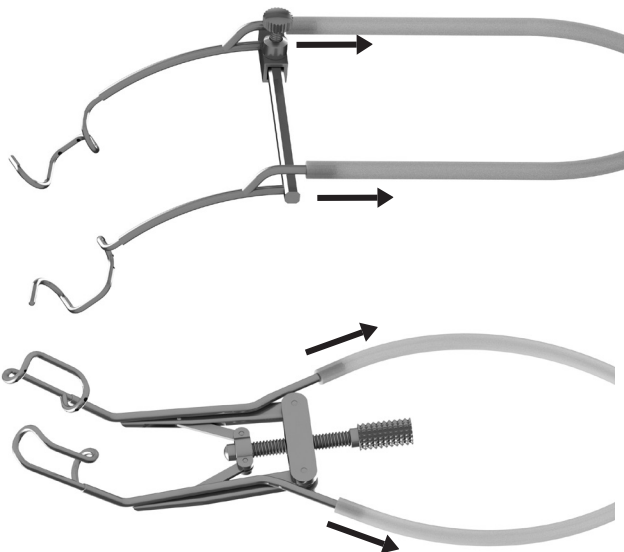
Fig. 2

Fig. 3

**PREPARATION BEFORE CLEANING**

- Remove the tubing system according to fig. 2 and reprocess it according to its product-related reprocessing instructions (1102227).

**Pre-cleaning**

- Rinse the Eye Speculum with aspiration under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.
- Assemble the cleaning adapter (G-31024) according to fig 4.
- Rinse the Eye Speculum with aspiration with a cleaning gun with a suitable flushing adaptor or a disposable syringe.
- Rinse the Eye Speculum with aspiration against the flow direction.

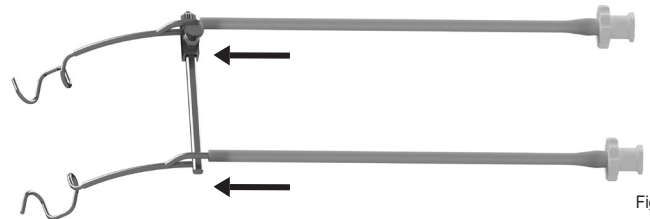
Fig. 4

**OR**

**Ultrasound cleaning as supporting pre-cleaning (optional)**

Basically, we recommend pre-cleaning with ultrasound all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the Eye Speculum with aspiration and make sure that it is completely moistened:
  - Fill the Eye Speculum with aspiration with cleaning agent.
  - Insert Eye Speculum with aspiration in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the Eye Speculum with aspiration afterwards with demineralized water for neutralization
- Check the Eye Speculum with aspiration for unobstructed flow:
  - Rinse the Eye Speculum with aspiration with a cleaning gun, a steamer or a single-use syringe.
  - Rinse the Eye Speculum with aspiration against the flow direction.

**FIRST TREATMENT AT PLACE OF USE**

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/ immediately after application with a compress soaked in sterile water.
- Place dry Eye Speculum with aspiration into a suitable storage or transport system and close it.

\* Classification recommended by GEUDER AG, the responsibility lies with the operator.

## AUTOMATIC CLEANING AND DISINFECTION

As additional load only ophthalmic products are permitted.

Thermic disinfection  $A_0$  value > 3000, 93 °C, 5 min; VARIO TD program.

The selection of suitable racks, charging trolleys for optimal rinsing and flushing of the medical devices should be done after consulting with the cleaning device manufacturer.

### Process

- Connect the Luer-Lock connector of the cleaning adapter to the mobile unit.
  - Avoid rinsing shadows.
- After cleaning and disinfection remove the cleaning adapter in reverse order (see fig. 4).

### DRYING

Dry the Eye Speculum with aspiration with medical compressed air, if necessary.

### Control of pH value

For ophthalmic instruments a batch-related routine control for alkaline residues is recommended.

- Apply the remaining humidity of the lumen instruments onto a pH test strip.
- Compare the pH test strip with the color scale:
  - pH must be neutral.

## CONTROL AND MAINTENANCE

### Visual control

- Check if the Eye Speculum with aspiration is clean:
  - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the Eye Speculum with aspiration is damaged:
  - Remove corroded or damaged Schott Eye Speculum with aspiration.
- Check if the Eye Speculum with aspiration is complete.
- Check if the Eye Speculum with aspiration is dry.

### Maintenance

- Care for the Eye Speculum with aspiration with a suitable instrument oil:
  - Let the Eye Speculum with aspiration cool down.
  - Apply the instrument oil selectively on the area that needs it (fig. 5).
  - Remove excess instrument oil, if any, with a lint-free wipe.
- Carry out a functional control:
  - Check if the Eye Speculum with aspiration is complete.
  - Check if the moveable parts can be moved easily.
  - Check if the lock mechanisms work.

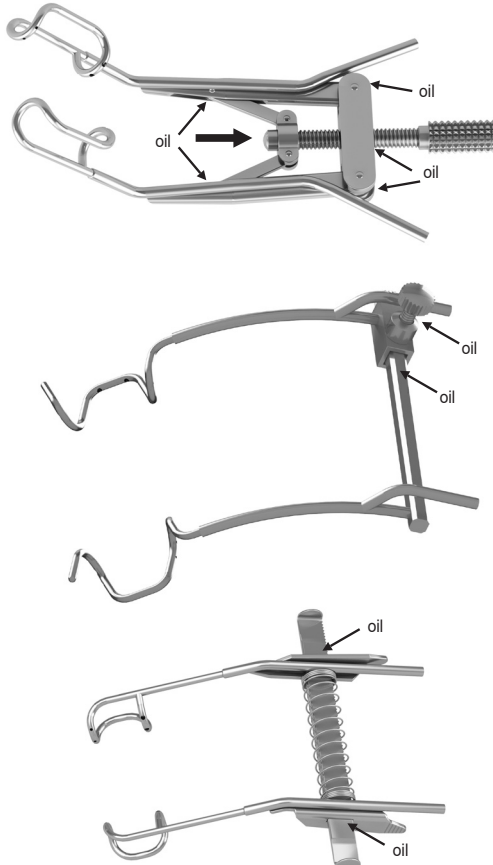
Fig. 5

## PACKAGING

Use packaging materials and systems complying with the following standards:  
DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

## STERILIZATION

Steam sterilization in fractionated prevacuum method:  
SAL 10<sup>-6</sup>, at least 3 min at 134 °C, 3 bar.

## STORAGE

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

- Damaged or open sterile barrier systems (SBS) are deemed non-sterile.
- The storage time depends on the type of packaging and must be validated accordingly.

## TRANSPORT

Transport must not have a negative effect on the characteristics of the sterile goods.

Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

## DISPOSAL

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute ([www.rki.de](http://www.rki.de)).

National standards, regulations and/or restrictions must also be considered!

## GENERAL INFORMATION FOR PROCESSING

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

## DETAILS OF VALIDATION

### Summary of processes validated by GEUDER AG:

The selection of the parameters refers to worst case conditions.

### Manual pre-cleaning

#### Pre-cleaning:

Using tap water (according TrinkwV [German Drinking Water Ordinance]), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

#### Cleaning:

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H; after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

#### Process chemicals (manual):

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.

### Automatic cleaning and disinfection

#### Cleaning

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

#### Disinfection

- Thermal high level disinfection  $A_0$ -value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection  $A_0$ -value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

#### Process chemicals (automatic)

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

### Steam sterilization

- Program: half-cycle SAL10<sup>-6</sup> fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: type Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature (max. 25 °C).
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective reprocessing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Actual version available at GEUDER AG.

Subject to changes.