caera®

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# Table of contents

- 20 Package contents
- 21 Overview of the bracelet functions
- 23 Emergency call procedure
- 24 Functions – brief overview
- 25 Charging the battery
- 26 Overview of the flashing sequences
- 31 Cleaning
- 32 Important information
- 35 Technical specifications – caera

# Package contents

1 caera emergency bracelet

4 5V mains plug

2 Quick-start guide

5 USB-C charging cable

3 Magnetic charging station

# Package contents

1 caera emergency bracelet

4 5V mains plug

2 Quick-start guide

5 USB-C charging cable

3 Magnetic charging station

## Overview of the bracelet features

For a detailed overview of the features and uses of this caera emergency bracelet, please visit the caera website or download the instruction manual.

# Overview of the bracelet features

- 1 Emergency button & switching on/off
- 2 LEDs
- 3 Speaker
- 4 Microphone
- 5 Stainless-steel folding clasp


## Other features

- Embedded SIM (permanently installed)
- Battery (lasts for up to 21 days)
- Modern, no-frills design
- Europe-wide location tracking
- Water- & dust-proof
- Automatic fall detection
- Medical & hypoallergenic silicone

# Emergency call procedure

## The emergency call procedure

The emergency call procedure can be triggered manually at the push of a button or is automatic if a fall is detected:

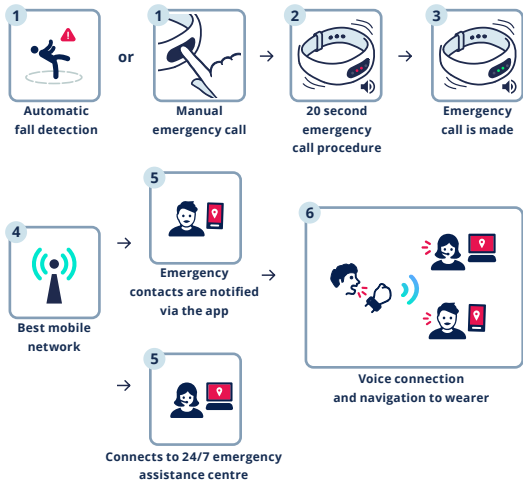# Brief overview of features

## Switching the caera emergency bracelet on and off

Press 4x

White light

Vibration

### To switch the device on, press the button 4 times in a row


- The device will switch on automatically as soon as it is placed in the charging station (**first-time use:** caera will not start until it is placed in the charging station).
- The LEDs flash white from the centre outwards
- A brief vibration can be felt
- The device is ready for operation once the lights have stopped flashing and the vibration has stopped
- To switch it off, press the button again 4 times in a row

### Triggering the emergency call manually/automatically

- **Manual trigger:** Pressing the button for 3 seconds makes an emergency call
- **Automatic trigger:** If a fall is detected, an emergency call is triggered
- The LEDs will flash red & switch to yellow once the emergency call has been made
- Emergency alarm and vibration via bracelet
- The button can be pressed again within 20 secs. to cancel the emergency call procedure
- The LEDs flash green to indicate that the emergency call has been made successfully
- The bracelet establishes a voice call between caera and the emergency contact

# Charging the battery

## Charging the battery/battery autonomy

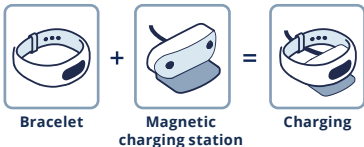Connect the USB charging cable to the magnetic charging station (included) and connect the cable to the power supply (included). Plug it into the power supply.

Connect the magnetic charging station to the caera alert bracelet. It will take up to 2 hours to fully charge the battery. The bracelet will operate for 21 days on one charge. The bracelet will emit an automatic (low) battery warning if the charge level goes below 30% and (again) at 10%. The bracelet will vibrate three times in quick succession and the LEDs will indicate the current battery level. Tap the emergency button once to manually check the battery level. While charging, the device will search for available updates and install them automatically. This means the device always has the most up-to-date software version.

# Overview of flashing sequences

## Feature

### Switching on the caera emergency bracelet

## Procedure

Tap the emergency button on the bracelet 4 times in a row.


**Note:** When using it for the first time, place the device in the charging station to switch it on.

### Switching off the caera emergency bracelet

Briefly press the emergency button on the emergency bracelet 4 times in a row.


### Triggering an emergency call manually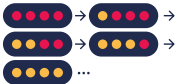

Press and hold the emergency button on the emergency bracelet for 3 seconds. The subsequent emergency call sequence comprises a 20 second emergency call timer and a connection process.

## Feature

### Emergency call - automatic fall detection

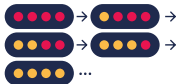## Procedure

The emergency bracelet automatically detects falls thanks to its intelligent fall detection system and automatically makes an emergency call. The subsequent emergency call sequence comprises a 20 second emergency call timer and a connection process.

### Emergency call timer/ establishing the server connection

Once the flashing light sequence has turned from red to yellow, the emergency call timer has finished. It is possible to cancel the emergency call during this sequence by pressing and holding the emergency button again.


### Voice connection

The connection to the server and the app is established. The emergency call can be aborted at any time while the connection is being established (green light sequence).

## Feature


### Battery indicator during charging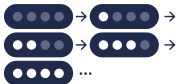

## Procedure

The charging process will start automatically when the emergency bracelet is placed in the magnetic charging station (included).

### Coupling process

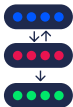During device set-up, you will be asked to put the caera emergency bracelet into the magnetic charging station to start pairing. Then the LEDs will continuously alternate between blue and red. To continue the pairing process, press the emergency button.

The LEDs will turn green to confirm pairing has been successful.

## Feature

## Procedure

### Low battery warning

A battery warning will be emitted if the battery level falls below 30%. Another battery warning is emitted once the critical level of 10% is reached.

### Cancelling the emergency call

During the 20-second emergency call sequence, the emergency call can be aborted by pressing the emergency button.

### Software update

Once the device has been placed in the magnetic charging station, it will automatically search for available updates.

### Checking battery status

37.5% battery =

A single quick tap on the emergency call button checks the battery status of the caera emergency bracelet. The number of white LEDs that are lit up indicates the status.

## Flashing white LED:


12.5 % battery

## White LED lit up:

25% battery

---

### Checking the signal strength

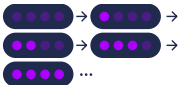37.5% signal strength =

Tap the emergency call button twice to check the signal strength of your caera emergency bracelet. The number of purple LEDs lit up indicates the signal strength. As with the battery status, each flashing LED represents 12.5% and each solid purple LED represents 25%.

#### Example:

2x solid lights = 50%

1x solid lights + 1x blinking LED = 25% + 12.5% = 37.5%

4x solid lights = 100%

# Cleaning

## **Cleaning the caera emergency bracelet and accessories**

Clean the caera alarm wristband and accessories regularly to maintain full functionality.

- Remove the bracelet from the magnetic charging station before cleaning and disconnect the power supply from the charging station.
- Clean the bracelet and accessories with a soft, dry or slightly moistened cloth. Never use chemical cleaning agents.
- If you use a damp cloth for cleaning, dry the bracelet and accessories afterwards, and check for any dirt or liquid that may have built up on the bracelet microphone or speakers.
- Check for any dirt or liquid that may have built up on the contacts of the magnetic charging station. Remove carefully.

# Important information

## Warnings & safety

Read the following warnings and information about handling your caera safely:

- The housing should only be opened by authorised specialists.
- In the event of a malfunction, please contact the caera customer service.
- Do not use with implants or other medical devices.
- Avoid areas where mobile phone use is blocked or dangerous.
- The caera emergency bracelet is dust- & waterproof.
- Operating temperature range of the caera: -10 °C to +50 °C.
- Set up the bracelet & subscription correctly to access the full range of functions.
- Charge the battery of the caera emergency bracelet regularly.
- Only use the original accessories supplied.
- Despite network coverage across Europe, it cannot be guaranteed that an emergency call can be placed from areas with dead zones or interference.
- Keep the packaging contents and operating instructions in a safe place so that they are to hand if needed.

# Important information

## Disposal and recycling

The caera battery should not be disposed of in the household waste. Please observe the local regulations on waste disposal.

Your caera should not be disposed of in the household waste. Take it to an electronic device recycling centre.

The compact design means that customer cannot remove the battery. Please use caera service for removing and disposing of the battery to ensure environmentally-friendly disposal.

# Important information

## Key to signs and symbols

The following symbols are used in this instruction manual and on the device:

Compliance with EU safety standards. This product complies with relevant European and national directives.

Disposal in compliance with the EU Waste Electrical and Electronic Equipment Directive (WEEE). This product should not be disposed of in the household waste.

This product contains an instruction manual with all the important information you need.

## Warranty & Service

A warranty (liability for defects) is a legal guarantee for customers. It applies to items that do not work correctly or do not match their description. These rights apply throughout the EU. For new goods, the guarantee is for a minimum of 2 years. This warranty does not apply in the following cases:

- **If the customer is at fault**
- **If the damage was caused by improper use**
- **If the device was opened improperly or by an unauthorised repair centre.**

# Technical specifications – caera

## Technical specifications

- Dimensions: Length 55 mm x width 29 mm x height 16 mm
- Bracelet length: 12.5 cm to 19.5 cm – adjustable clasp
- Weight: approx. 55 g (excl. cable & magnetic charging station)
- Four multi-coloured LEDs installed
- GSM: GSM/GPRS/EDGE DUAL band 900/1800 MHz
- GNSS: GPS, GLONASS, GALILEO, BEIDOU, AGPS
- Battery: 200 mAh Li-ion battery, 3.7 V
- Battery autonomy/charging time: up to 21 days/< 2 hours for 100%
- Microphone: Condenser microphone
- Speaker: 1.0 W
- Location assistance using WLAN access points and GSM triangulation
- Water resistance: tested to IP 67
- Water and dust-proof
- Permitted operating temperature: -10 °C to +50 °C (do not charge below 0 °C)

## CE Declaration of Conformity

microsynetics GmbH hereby declares that the following product **caera**, name: Emergency bracelet with automatic fall detection, based on the CE Declaration of Conformity, complies with the provisions of the Short Range Devices Directive 2014/53/EU. (<https://caera.de/de/declaration-of-conformity>)

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