



HOW TO CLEAN YOUR Versapak Pathology Transport Carrier


Versapak Medical Carriers are very easy to keep clean. Because of their antimicrobial properties, using a mild / household detergent is enough to eliminate bacteria. Please see below full instructions of how to clean your medical carrier.



Fill a bowl with warm
soapy water.



Wipe entire surface of
the carrier, rinsing the
cloth occasionally.

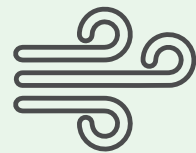

Leave the carrier
to air-dry.

VersaShield™ is an Antimicrobial, Silver-Ion based, technology that is embedded into the fabric of a Versapak carrier bearing this symbol (available only with bags/carriers fitted with the T2 Locking System). It is designed to eliminate bacteria that can build-up on the surfaces of bags and pouches, quickly and effectively.



Use free from alcohol cleaning products.
Use warm water. For heavily soiled,
dirty carriers, add further household
cleaning products.
Leave to air-dry after cleaning.



Do not use abrasive products to clean.
Do not use boiling water.
Do not use chemical agents or bleach.