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REF KED015/017

ENG THIGH SUPPORT

Please read the following instructions and keep for future reference.

For support and compression around the thigh. Intended uses can include but are not limited to mild pulled hamstring, quadriceps, groin strains, swelling/inflammation, bruising and sore muscles.

The **ACTIVE ELASTICATED** range is made from a high-quality fabric, with the combined benefits of nylon, polyester as well as latex for superior strength, flexibility and comfort, to ensure a secure fit. The support is firm and effective on weakened muscles and joints, and helps prevent injuries. Each product is designed to give maximum support whilst maintaining a comfortable fit. Using the sizing chart, select the size that best suits you, to get the maximum benefits from this product.

Note: The support should be sized correctly to ensure maximum benefit and to avoid over compression. If the support is being used following an injury, seek medical advice if no improvement in the condition is seen within 72 hours. If the injury is severe or if in any doubt seek medical advice immediately. If used by children, adult supervision is required. Keep out of reach of young children. Always use on clean intact skin. Do not use if liniments, ointments or creams have been applied to the affected area.

Warning: If soreness or a rash occurs discontinue use immediately and seek medical advice. This product contains latex which may cause allergic reactions, do not use if sensitive to latex. *If any serious incidents occur in relation to using this device it should be reported to the manufacturer and the competent authority of the member state in which the user is established.*

Washing & Storage Instructions: Hand wash in lukewarm soapy water, rinse and air dry. Store in a cool, dry place and away from direct heat and sunlight.

ACTIVE ELASTICATED RANGE THIGH SUPPORT

Website: www.kedley.co.uk


Impo International Ltd
Unit 3, 60 Lower Road,
Harrow, Greater London,
HA2 0DH, UK.


Eurolink Europe Compliance Ltd,
Dublin, D02 AY86, Ireland

REV: KED015FU-05/21v1


- 1) Select correct size from size chart on the back of the packaging.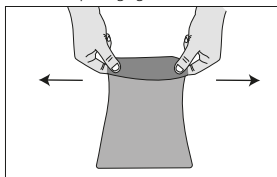


- 2) Fold over the top of the support and slide over appropriate foot.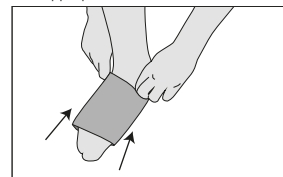

- 3) Slide up past the knee and over the thigh.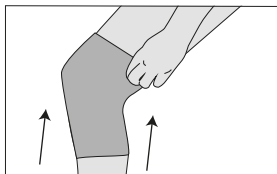

- 4) Support should fit tightly but it should not restrict circulation.

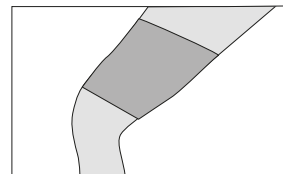KEDLEY

06052021

Video fitting instructions available @ www.kedley.co.uk

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