ENG COMBINED WRIST & THUMB SPLINT

The Kedley Combined Wrist & Thumb Splint has been manufactured using soft and comfortable material. The design allows for firm and effective support. The adjustable velcro strap enables you to adjust the size, while ensuring a comfortable fit.

The Kedley Orthopaedic range is made from only the highest quality materials. Each item is manufactured to give you steady support, to promote healing and comfort.

Fitting instructions

1. Check the size limit on packaging, and select the correct hand option. Slide your hand into the support with the palm splint on your palm.
2. Insert your thumb into the thumb hole and secure the middle strap, followed by the top strap.
3. Adjust the thumb strap, then readjust all straps for desired compression.
4. Check that the support fits tightly, but does not restrict circulation.

Washing instructions: Hand wash in lukewarm soapy water, rinse and air dry. Store in a cool, dry place and away from direct heat and sunlight.

Warnings: For serious/reoccurring injuries or if a rash/discomfort develops, discontinue use immediately and seek medical advice. This product contains latex, which may cause allergic reactions.

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

KEDLEY.co.uk

ORTHOPAEDIC SUPPORTS



COMBINED WRIST & THUMB SPLINT LEFT HAND

REF KED091

- 1) Slide your hand into the support with the palm splint on your palm.
- 2) Insert your thumb into the thumb hole and secure the middle strap, followed by the top strap.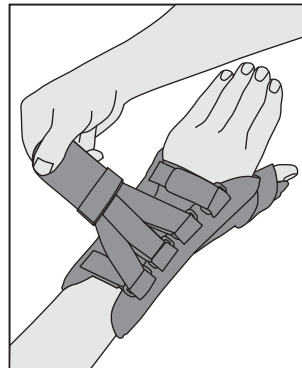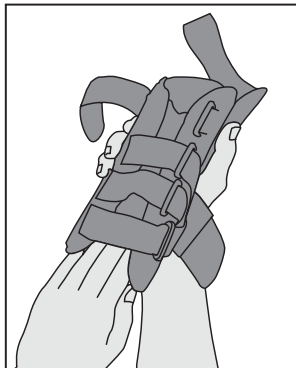

- 3) Adjust the thumb strap, then readjust all straps for desired compression.
- 4) Check that the support fits tightly, but does not restrict circulation.

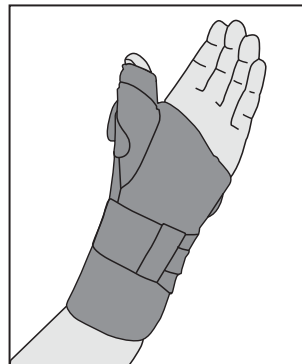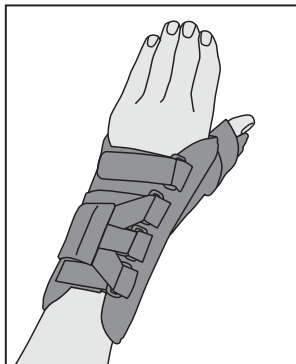ORTHOPAEDIC SUPPORTS Combined Wrist & Thumb Splint – Left

Website: www.kedley.co.uk

Impo International Ltd
Unit 3, 60 Lower Road,
Harrow, Greater London,
HA2 0DH, UK.

EC REP Eurolink Europe Compliance Ltd,
Dublin, D02 AY86, Ireland

REV: KED091-10/24v1

KEDLEY®

YouTube

Video fitting instructions and
more information available at
www.kedley.co.uk

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