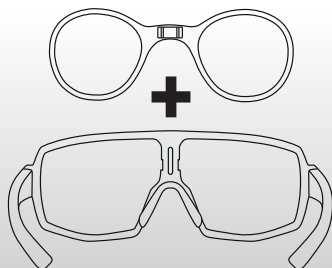
CLIPIN104

for model Performance

CLIP-IN + PERFORMANCE

STEP 1

Insert the "Clip-in" to the side, matching the hook with the hollow part of the mount. Press until you hear a click. Be sure to put a protective pad between the glasses and the "Clip-in" to protect the lenses.

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STEP 2

Press on the opposite side of the "Clip-in" to insert the other hinge on the frame. Remove the protective pad.

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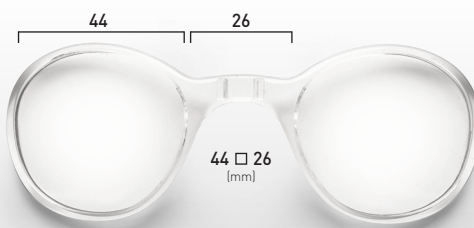
FRONT VIEW / VISTA FRONTALE

The "Clip-in" hooks are inserted in the recesses of the frame.

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CLIP-IN LENS SIZE / DIMENSIONE CLIP-IN

EN Use: class I medical device, as for Rule 1, Annex VII of EU Regulation EU 2017/745 on Medical Devices, whose intended use is the association with ophthalmic lenses for the correction of visual defects. To mount ophthalmic lenses, in the optical insert contact a specialized optician. Compliant with EU Regulation 2017/745. Warning: it cannot be excluded that sensitive or allergic subjects may experience adverse reactions in contact with the materials and treatments used. Care and cleaning: the clip-in must be stored at a temperature between -10 ° C and + 35 ° C. Avoid exposure to the sun in closed, unconditioned environments (e.g. car dashboard) because the temperatures reached in such conditions can damage its functionality. In case of contact with agents such as saltwater (seawater), clean immediately with fresh water and neutral soap. In case of wear, replacement is recommended. For cleaning, use a damp cloth and a normal neutral detergent, and dry with a clean soft cloth. Avoid the use of solvents (e.g. alcohol or acetone) and aggressive chemical detergents. Marking: model identification and batch date are printed on product or on the label. Complain: if it is not possible to contact the store where it was purchased, for complaints or anomalies, send the request to info@demonocchiali.it - the declaration of conformity is available at the address: www.bigline.it/eu-conformity-declaration

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