

Instructions for use: DBL2-0002

Iconic Pro Reservoir LC is a light-curing acrylate containing plastic for model preparation in dental technology. The plastic carving material is supplied in syringes.

Composition

Iconic Pro Reservoir LC Gel consists of multifunction methacrylates with micro and macro fillers.

Applications

Iconic Pro Reservoir LC is used for model preparation in dental technology. It is primarily used for creating precise bleaching reservoirs on dental models, but can also be used as a build-up material.

Processing

Press the gel out of the syringe and remove with customary laboratory instruments and apply onto the model. Iconic Pro Reservoir LC is modelled and smoothed with a brush or instrument. If Iconic Pro Reservoir LC is applied the plaster should not be moistened. A UV light source is preferred for polymerisation but a halogen light source can also be used. After the material has cured it can be processed with milling burs.

Storage

Iconic Pro Reservoir LC have a shelf life of at least three years if kept in original containers and stored at temperatures not exceeding 22°C. This also applies to syringes that have been opened and then re-closed. Avoid exposure to direct sunlight.

Application remarks

The application instructions and other precautionary measures described in the instructions for use of this product must be observed. In the unlikely event a patient has an undesirable reaction, please notify us in all cases giving a precise description of the accompanying circumstances.

Cautionary Note

Irritant.

