

MAGIC SMILE HOME
GEL + LED



MAGIC SMILE

TEETH WHITENING

18% PAP (NON-PEROXIDE)

WHITENING & REMINERALIZATION

MAGIC SMILE

Home Whitening

Making not just a smile, but a feeling of joy, well-being and safety.

Magic Smile provides best quality products and breakthrough technologies for quick and safe home teeth whitening, in-office bleaching and oral care.

Home GEL + LED Kit with PAP gel effectively whitens your teeth and strengthens teeth enamel.

The teeth-whitening procedure is safe and effective when used according to the provided instructions and contraindications.

Delicate (NON-PEROXIDE) Whitening

Our breakthrough PAP gel with its unique ingredients causes the least amount of sensitivity while ensuring perfect bleaching results.

Multiple Applications

The kit is designed for up to 10 bleachings and 5 remineralizations depending on the surface area and amount of gel used per application.

Full Complex Treatment

Use Magic Smile Home Kit on its own or use as a follow-up treatment to in-office bleaching. Add Magic Foam for the best results.

Teeth Strengthening

Magic Minerals Gel contains powerful ingredients - Ca, Mg, potassium citrate & hydroxyapatite that will strengthen your enamel and help alleviate teeth sensitivity.


Safe to Use

The Magic Smile system does not use nitrates, abrasives or ultraviolet radiation. The blue and red light used in the LED Mini Lamp is the safest way to activate the bleaching gel.


Long-Lasting results

Magic Smile Home Kit used according to the provided instructions ensures results of up to 8 shades that last for up to 12 months

Package Contents

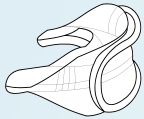


Silicone Teeth Whitening Tray

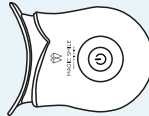

MagicLight Mini
Rechargeable Blue & Red LED Mini Lamp



Shade guide

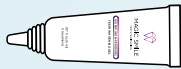

0.17 fl.oz (5ml) tube
of 18% PAP Gel (2pcs)



0.17 fl.oz. (5ml) tube
of Magic Minerals Gel (1pc)



Charging cable for
MagicLight Mini

Storage:


Avoid direct sunlight. Keep in a dark and cool place.
39°F - 50°F (4°C - 10°C)

Magic Smile technology is designed in the U.S.


BLEACHING PROCEDURE

IMPORTANT


Before use, carefully read the instructions, contraindications and post-treatment care

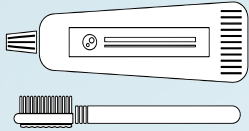


Brush your teeth
with toothpaste

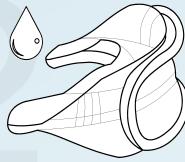


Before & after each procedure
- rinse the tray with warm
water and dry it

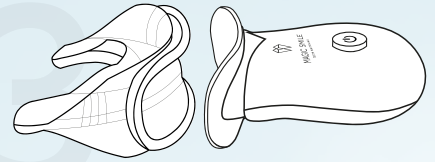


Insert MagicLight Mini into the tray

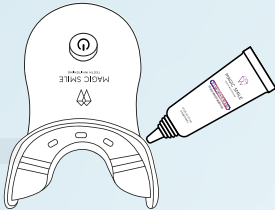

Apply one continuous layer of the gel
to the front side of the tray. Use
bleaching gel, approximately, 1 ml,
evenly dispensing a thin layer to the up
and to the bottom part of the tray

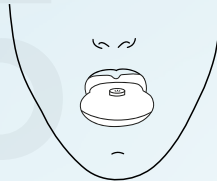

Place MagicLight Mini with filled
tray in your mouth and activate it.
Treatment time is 30 minute

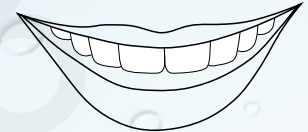

Repeat the procedure for
30 minutes a day for 5 -10
consecutive days

IMPORTANT:

You can whiten your teeth in just 15 minutes.

But for best results, we recommend 30 minutes treatment and you get red light benefits.

Blue light with non-ultraviolet spectrum is used for the safe gel activation.

Red light can help with a variety of tooth and mouth problems, can reduce oral bacteria counts.

Do not exceed the total time of gel applications over 60 minutes.


EATING/DRINKING AFTER PROCEDURE:

We recommend not to eat or drink for 4 hours after the bleaching procedure.

REMINERALIZING THERAPY


Apply Magic Minerals Gel the next day following a full course of home bleaching

1

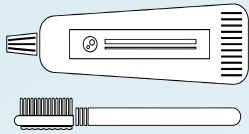


Brush your teeth with toothpaste

2

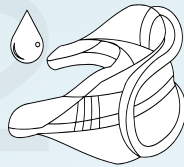


Before & after each procedure
- rinse the tray with warm water and dry it

3

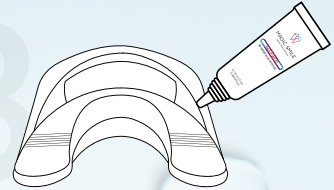


Apply one continuous layer of the blue gel to the front side of the tray. Use the blue gel approximately, 1 ml, evenly dispensing a thin layer to the up and to the bottom part of the tray

4

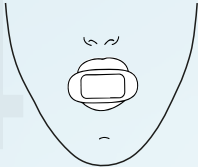

Place the tray with blue gel in your mouth for 30 minutes. Do not use the light activator

5

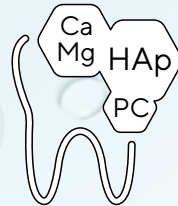

Magic Minerals Gel protects teeth enamel and reduces teeth sensitivity

6



Do not eat or drink for 2 hours after the procedure

To reduce the level of tooth sensitivity, immediately perform it after the whitening procedure

Phthalimidoperoxycaproic Acid (PAP) is Non-peroxide whitening agent that doesn't release free radicals when oxidizing pigment. Make your teeth safer, with less sensitivity and gum irritation.

Hydroxyapatite (HAp) is naturally restorative, replacing lost minerals in damaged parts of the enamel and rebuilding protective layers of the teeth.

Potassium Citrate (PC) is a powerful agent in treating tooth sensitivity. Potassium ions penetrate into the dentine and desensitize nerves in the tooth pulp.

Calcium(Ca) bonds to the teeth to fill in weakened areas of enamel. Calcium strengthens enamel to protect against erosion and cavities.

Magnesium(Mg) is fantastic mineral for overall health, playing a critical role in building strong teeth and bones. Magnesium and calcium work together, which is why we always use them both.

CONTRAINDICATIONS

Bleaching is not recommended for:

- those under 18 years old, since the pulp has not yet fully formed
- those with unrealistic expectations (for example, those with natural A2 teeth color expecting A1)
- those with tooth sensitivities, cracks, exposed tooth necks or erosion
- those with tooth decay, enamel or gum disease, and tooth or oral infections
- those who had any recent (2-3 weeks) dental work that involves curing or bonding (whitening decreases the effect of the bonding process)
- those with GERD or who have increased acidity, which leads to weak enamel or increased sensitivity
- those with a significant reduction in enamel as a result of tooth abrasion
- those who are pregnant or breastfeeding

KEEP IN MIND: Veneers, fillings and restorations have a different structure than enamel and are not cooperative with bleaching. In some cases, old restorations are suitable for bleaching when pigment has accumulated in the pores of the material; however, the material itself is unfit for bleaching.

POST-TREATMENT DIET

Avoid the following for 3 days post whitening procedure to achieve longer-lasting results:

- products containing colorings
- smoking
- overly hot or cold food and drinks
- lipstick or lip gloss
- any foods or drinks that would leave a stain on a white shirt
- colored toothpaste or mouthwash

We recommend following a “white diet” for at least two weeks after the teeth-whitening procedure. This will ensure the best and longest-lasting results, while eliminating the possibility of teeth staining.

THE “WHITE DIET”

Avoid the following food, drinks, and beauty products that can stain your teeth:



Coffe, tea and drinks based on them



Juice, wine, fruit drinks, lemonade



Chocolate, othe food and drinks containing cocoa



Vegetables, fruit, berries that cause staining: beets, blueberries etc.



Ketchup and other colored sauces, spices



Drinks containing a large amount of acids: smoothie, soda



Colored toothpaste or mouthwash

KEEP IN MIND: If you consume food or drink that stain the enamel, use MAGIC SMILE FOAM TOOTHPASTE to brush your teeth as soon as possible.

MAGICLIGHT MINI

LED Mini Lamp

Hold down the power button for a few seconds to turn on MagicLight Mini. During the treatment the lamp uses blue light for 25 minutes and red light for 5 minutes. MagicLight Mini will turn off automatically after 30 minutes.

- LED Bulb: 6 pcs
- Red color light: 2.0-2.2V/Wavelength: 621-625/Intensity:300-450
- Blue color light: 2.7-3.0V/Wavelength:466-470/Intensity:200-250
- Timer function: 30 minutes
- Battery type: Li-ion Rechargeable Battery
- USB-A to MicroUSB charging cable
- Capacity: 120mAh
- Voltage: 3.6V

MagicLight Mini charging steps:

1. Open the white silicone cover.
2. Connect the charging cable to MagicLight Mini and power source. The charging indicator on MagicLight Mini will turn red.
3. Once the battery is fully charged, the indicator will turn off automatically.
4. Disconnect MagicLight Mini from the charging cable and close white silicone cover.


INGREDIENTS

Teeth Whitening gel: Phthalimidoperoxycaproic Acid (PAP), Hydroxyapatite, Potassium Citrate, Glycerin, Deionized water, Sorbitol, Propylene Glycol, PEG-400, Cellulose Gum, Xanthan Gum, Hydroxyethylcellulose, Sodium Saccharin, Menthol, Aloe Barbadensis Leaf Juice, Anthemis Nobilis Flower Extract, Sodium Bicarbonate, Methylparaben, Propylparaben, Disodium EDTA, Sodium Citrate.

Magic Minerals Remineralizing Gel: Hydroxyapatite, Potassium Citrate, Magnesium Chloride, Calcium Glycerophosphate, Glycerin, Deionized Water, 1,2-propanediol, Carbomer, Polyvinylpyrrolidone, Poloxamer 407, Menthol, Edible Pigment (Calcium Carbonate, Silicon Dioxide, Yeast Protein).

DISCLAIMERS

Bleaching can be done only up to the natural color of your teeth and will not alter them any further. Bleaching results may vary from person to person depending on the type of teeth you have or the conditions of your teeth stains. It will, however, remove most of the stains and restore your teeth to their natural color. Keep out of the reach of children. Avoid contact of the gel with eyes or skin. If contact occurs, wipe off the gel and rinse with water thoroughly. MagicSmile bleaching gels are for teeth whitening use only, as according to the instructions. Magic Minerals gel is for teeth application only. May be harmful if gel is swallowed in large quantities. LED Mini Lamp is for teeth whitening use only as according to the instructions. Do not use the LED in any other way. All syringes and tubes are disposable and can't, nor should be reused. For 1 procedure, you can whiten teeth by up to 8 tones on the whitening scale. One shade can also be accomplished. The whitening effect lasts from 1 to 12 months. The product images and gel colors shown are for illustration purposes only and may not be an exact representation of the product. The color of the gel may be less saturated or a slightly different color or shade. Magic Smile Corp. reserves the right to change product images and specifications at any time without notice. Magic Smile teeth whitening system is provided as is without any guarantees or warranty. In association with the product, Magic Smile Corp. makes no warranties of any kind, either expressed or implied, including but not limited to warranties of merchantability, fitness for a particular purpose, of title, or of non infringement of third party rights. Use of the product by a user is at the user's risk. No representations or warranties, either express or implied, of merchantability, fitness for a specific purpose, the products to which the information mentions may be used without infringing the intellectual property rights of others, or of any other nature are made with respect to information or the product to which information mentions. In no case shall the information be considered a part of our terms and conditions of sale.

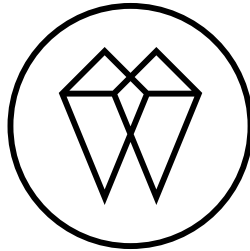

www.MagicSmileHome.com
info@MagicSmileHome.com