

## **SLENDER FENDER FIT KIT INSTALLATION GUIDE**

- Remove seat sling and install 3/8" spacers (provided) in between the top of the seat rail and fabric seat sling underside. \*
- For folding chairs with cross braces install the clamp onto the frame tubing, either below or beside the seat rail with the seat sling intact.
- Install the Slender Fender receiver clamp and post to your wheelchair seat rail, in line as close as possible to the wheel center.
- Make sure the widest slot in the post is facing the seat sling. Please note the clamp has two positions short and tall to accommodate 20" to 26" diameter wheels.
- Choose the position best suited for your install, by inverting the clamp.
- When tightening the four 10-32" deep socket cap screws, that, both above and below the seat rail are spaced evenly apart.
- Make certain that your fender post is parallel with your chair's back post.
- Position the Slender Fender onto the vertical post for a dry fit. Make sure the wheel spokes don't contact the fender surface. If there is contact reposition the wheel laterally on its axle and tighten securely.
- Position your fender on top of the tire to establish equal spacing.
- Secure the fender against the vertical post and drill a 1/4" (drill bit provided) diameter hole through the wide slot and through the plastic fender, somewhere in the middle of the slot.
- Install the 1/4"-20 lock nut (provided) into the wide slot of the vertical post.
- Put the 1/4"-20 screw (provided) through the plastic fender first and then mate with lock nut.
- Tighten the 1/4"-20 screw enough so that you can slide the fender vertically while you position it over the tire.
- Double check the clearance between the tire and the fender shell to be certain that's your preferred position for accessing the push rim.
- Holding the fender firmly, drill the second hole through the 1/4" clearance hole at the top of the vertical post. Install the next 1/4"-20 screw and lock nut (provided). Tighten both screws firmly with your 5/32" Allen wrench.

ANY QUESTIONS REGARDING PRODUCT INSTALLATION, PLEASE CONTACT ED, AT MY  
**EMAIL ADDRESS:** [info@seatshardware.com](mailto:info@seatshardware.com)

**\*This step only applies to fixed frame chairs, not folding frames.**