



Kingkraft

Instructions for use
The Kingkraft Bather Collection



PLEASE READ THESE INSTRUCTIONS CAREFULLY BEFORE USE

CONTENTS

- IMPORTANT SAFETY INSTRUCTIONS
 - THE BATHER RANGE
 - INSTRUCTIONS FOR USE
 - ACCESSORIES
 - CLEANING INSTRUCTIONS
 - WARRANTY INFORMATION

IMPORTANT SAFETY INSTRUCTIONS



WARNING!

Please read these instructions carefully before using your Kingkraft product, and keep them safe for future reference.

Please also make sure that each person and any carer who uses the Bather is also aware of their contents and follows the instructions each time they use the unit.

Please bear in mind, for special / bespoke Bathers the instructions may vary. Please contact Kingkraft if you are unsure about the safe use of the product.

Please ensure that all carers and users are aware of these instructions.

These instructions cover the general use of Kingkraft Bathers and should be read in conjunction with the specific instruction for the product in use.

- **Never leave anyone unattended in the Bather at any one time**
- When placing the Bather in the bath, care should be taken to ensure the surface is free from grease, oil or soap deposits. Ensure that the Bather is secured before use.
- Be aware that the cover/mesh may become detached from the frame in use through a variety of causes, such as interface by user or others, excessive movement, or turbulent water inflow into the bath. Stop using and re-attach if the mesh becomes loose.
- The Bather may move inadvertently if subjected to sufficient force by the user. Extreme care is needed to ensure that the product is correctly specified and used.
- Never try to lift the Bather with someone in
- Be careful that all limbs, fingers and toes are clear of any trapping between the frame and the bath surface and sides

WARNING!

The Bather mesh is made from Polyester. Please make sure the user will not have any allergic reactions to the material before use.

This product is designed to be used in a bath.

To reduce the risk of scalding, always check the temperature before a person is in the bath, run the cold tap first and top up with warm water or use a suitable Thermostatic Mixer tap. Care must be taken to ensure that if running hot water into the bath, this is mixed thoroughly with the cold in the area of the Bather as well as the rest of the bath.

The Bather Collection

The Kingkraft Bather collection consists of four standard ranges.

- **Komfort:** Can be used in most standard baths and some Kingkraft baths
- **Kozy:** Can be used in all Kingkraft baths and most standard baths and allows the use of a changing top
- **Slimline:** Slimmer design to suit a wider range of standard baths
- **Relaxa:** To allow a more relaxed posture whilst bathing

1.1 Intended Use Statement

The Kingkraft bathers are intended to be used in domestic properties and nursing homes by Occupational Therapists, parents, carers and general public.

They are intended to assist in the bathing of children and adults. The person sits in a supported position and allows the carer/parents to wash and interact with ease.

The person must never be left unattended whilst in the Bather at any one time.

Bather sizes


Width	480mm
Support Bars	130mm (not available on Kozy)

Weight limit 120kg (19 stone)

Kingkraft Komfort



Kozy Bather

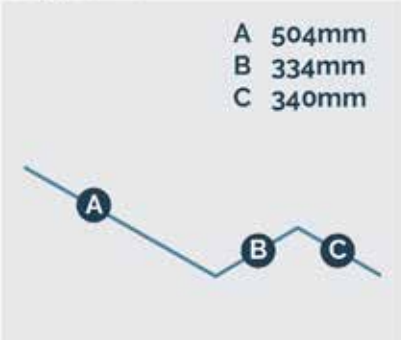


Width (Slimline)	457mm
Width (Relaxa)	480mm

Weight limit 120kg (19 stone)

Slimline



Relaxa

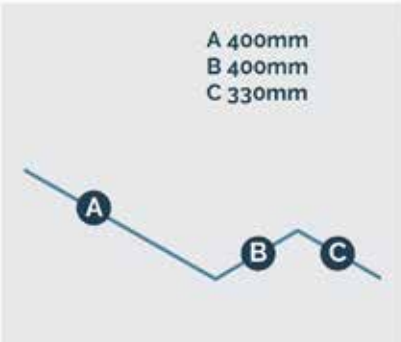

1.2 Intended use of accessories

Only specific Kingkraft accessories are intended to be used on the Bathers.

1.3 Classification

This product is a Class 1 Medical Device

Technical Information

Weight Load = 120kg

Weight of product = 5kg, please ensure you are able to lift the product with ease in and out of the bath

Instructions for use



WARNING!

The Bathers are NOT supplied in a sterile state.

The suitability of the person to use the Bathers should be assessed by a competent person.

Before use, the product should be visually checked for damage. If any damage found, stop using and contact Kingkraft immediately.

The mesh covers and wedges (if using) should be firmly held in place by the velcro. If the mesh is damaged in any way, do not use and contact Kingkraft.

The Bathers should be placed securely in the bath (*if you have a Komfort Bather or a Slimline Bather, ensure that the rubber arms are securely rested on the side of the bath. DO NOT USE IF THE BATH DOES NOT HAVE LEVEL SIDES.*

The sides of the bath should be also be checked to ensure the rubber feet can rest on a flat surface. Ensure that they can not move more than an inch (25.4mm) either side)

The Bather should be placed at the 'head' end of the bath away from the tap. The Bather should not be used with the tap behind the users head.

Any use of a lifting hoist to place the person in the Bather should be assessed by a competent person. This product is designed to be used under normal bathroom conditions. The Kozy Bather is designed to sit in the bath, ensure the seat is flat in the bath-if the seat rests on the curve of the

bath, this could damage the bath.

NEVER LEAVE A PERSON UNATTENDED IN THE BATHER

The Bather frames should be wiped clean with a soft cloth and soap based product after use, the mesh cover can be removed and machine washed. See cleaning instructions for further advice.

PLEASE DO NOT HAVE NAKED FLAMES NEAR THIS PRODUCT.

Accessories

WARNING!

THE HS25 HEAD SUPPORT CUSHION IS MADE FROM PVC!

PLEASE MAKE SURE THE USER WILL NOT HAVE ANY ALLERGIC REACTIONS TO THE MATERIAL BEFORE USE!

FITTING OF THE HS25 HEAD SUPPORT CUSHION

The HS25 vacuum support cushion is placed into the mesh cover through the velcro opening and the vacuum pump is threaded out through the hole. The support cushion is then positioned to suit the users head and then secured in place by the velcro strap around the back of the frame.

IMPORTANT

Water or moisture must NOT be allowed to enter the air extraction bulb/ valve, connecting pipe or the cushion itself. The ingress of moisture will seriously affect the performance of the cushion and will also invalidate the guarantee.

Instructions:

1. With the valve handle pointing downwards - turn the valve 1/4 anti-clock wise and leave to stand for two minutes to allow air pressure to balance.
2. Check that the surface to which the cushion is to be attached is clean and damp. Fix the cushion into position ensuring the velcro are firmly secured.
3. Place person on the cushion in the desired position
4. If necessary, further mould the cushion into the required shape

1. Turn the valve back to the down position
2. Press the bulb to extract the air until the degree of support is desired
3. Turn the valve to 1/4 clockwise to lock the vacuum
4. To release the vacuum, turn the handle back to the down position

Fitting of the wedges to the Bather

Place the wedges in the desired position and stick to the velcro strap. The wedge is intended as a support and isn't load bearing. It should not be treated as a side rail.

Cleaning instructions for the mesh cover

Please ensure that all users and carers are aware of the instructions in this manual.

Please note-when cleaning the mesh cover, non abrasive materials should be used. Under no circumstances should acid or bleach based disinfectant be used, such use will invalidate the guarantee.

Kingkraft recommend that the mesh cover should be cleaned regularly-this can be removed and washed at 40°C in a standard washing machine.

Warranty

The product comes with one year manufacturers warranty. Any faults occurred through misuse/user error will not be covered by the warranty.

The user must never be left unattended in the Bathtubs. If in any doubt about the safe use of this product, or the degree of secure support required, do not attempt to use. Please contact Kingkraft for further advice.

Always follow the instructions provided and in the case of any doubt or for further advice, please call 0114 2690697 or info@kingkraft.co.uk

BATHER - BIFU - ISSUE 3 - 09/19



Kingkraft Ltd 26D Orgreave Crescent, Dore House Industrial
Estate, Sheffield, S13 9NQ phone 0114 2690697
email info@kingkraft.co.uk web www.kingkraft.co.uk



WE PLAN + WE MANUFACTURE + WE INSTALL + WE MAINTAIN