R8571218
R8571418
R8581218
R8581418
R8591218
R8591418

SCT 2000, SCT 2100

Shower change table – Operation and maintenance manual en

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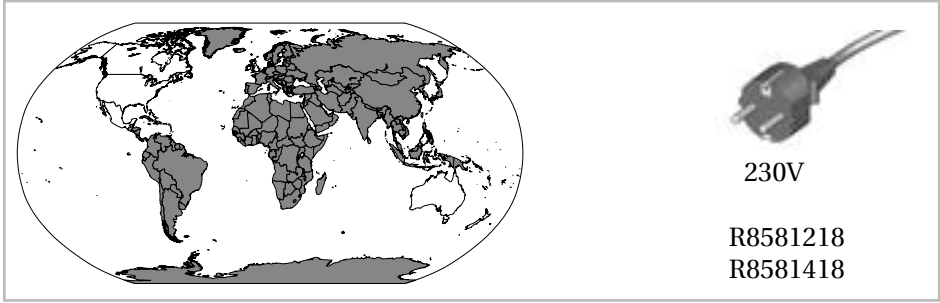
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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
A world map with shaded regions in South America, Africa, Europe, and Asia. To the right is a 230V power plug. Below the plug are the model numbers R8581218 and R8581418.

230V

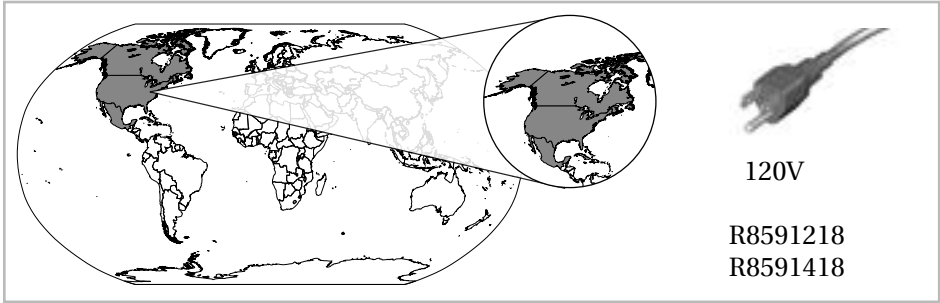
R8581218
R8581418

A world map with a callout circle highlighting North America. To the right is a 120V power plug. Below the plug are the model numbers R8591218 and R8591418.

120V

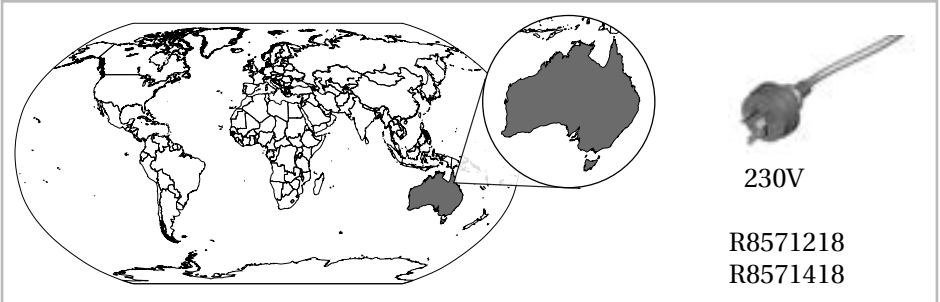
R8591218
R8591418

A world map with a callout circle highlighting Australia. To the right is a 230V power plug. Below the plug are the model numbers R8571218 and R8571418.

230V

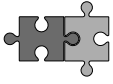
R8571218
R8571418

R8491

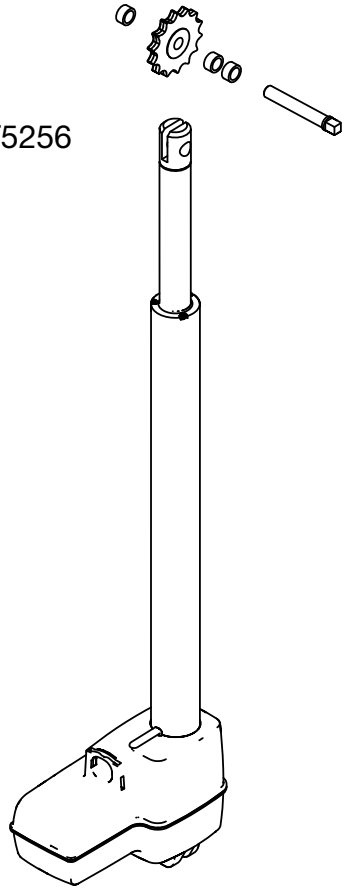T5256



| | | |
|--|---|---|
| |   |   |
| | 73 kg / 161 lbs | 200 kg / 440 lbs |

| | Max P | U | Max I |
|--|--------------|----------|--------------|
| R8581218 R8581418 R8571218 R8571418 | 70 W | 230V | 0,3A |
| R8591218 R8591418 | 36 W | 120V | 0.3A |

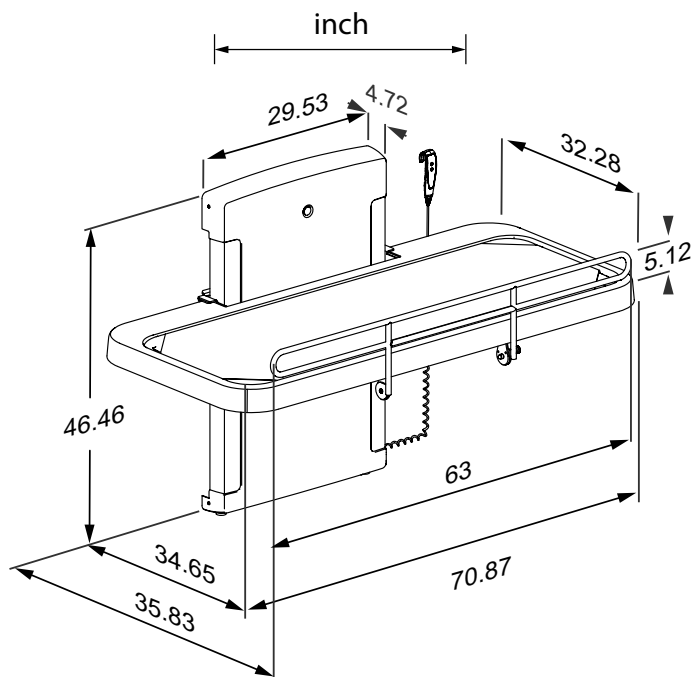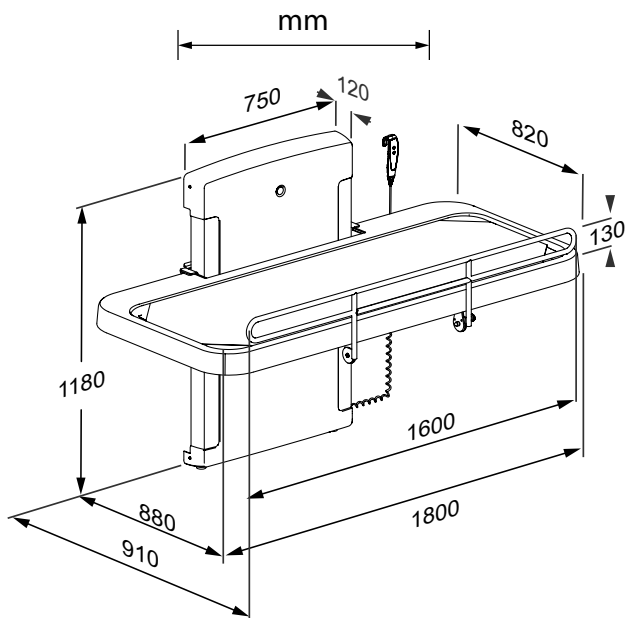Content

| | |
|--|----|
| Mounting instruction | 8 |
| Operational signage | 8 |
| Symbols used in this manual | 9 |
| General safety | 10 |
| Intended use | 11 |
| Intended operator profile | 11 |
| Intended user profile | 11 |
| Operating environment | 11 |
| Contraindications for use | 11 |
| Biocompatibility | 11 |
| Operating instructions | 12 |
| Safety rail | 13 |
| Height adjustment | 14 |
| Mesh/canvas cover | 15 |
| Zipping the cover onto the frame | 16 |
| Cleaning | 17 |
| Disinfection | 17 |
| Maintenance | 18 |
| Maintenance inspection | 18 |
| Resetting | 20 |
| Trouble-shooting | 20 |
| Electromagnetic compatibility | 22 |
| Electromagnetic immunity | 23 |
| Electromagnetic emission | 26 |
| Product label | 27 |
| Technical data | 29 |
| Disposal and recycling | 29 |

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

| | | | |
|---|--|--|--|
|  <p>Hand injury warning</p> |  <p>Head injury warning</p> |  <p>Electricity / voltage warning</p> |  <p>Do not leave patient unattended</p> |
|  <p>Power specifications</p> |  <p>Spare parts</p> |  <p>Article number / versions</p> | |
|  <p>Product weight</p> |  <p>Max. user weight</p> | | |

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the shower table.

Intended operator profile

The product is to be used by a person trained in health care and/or trained in washing a person in their care.

Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

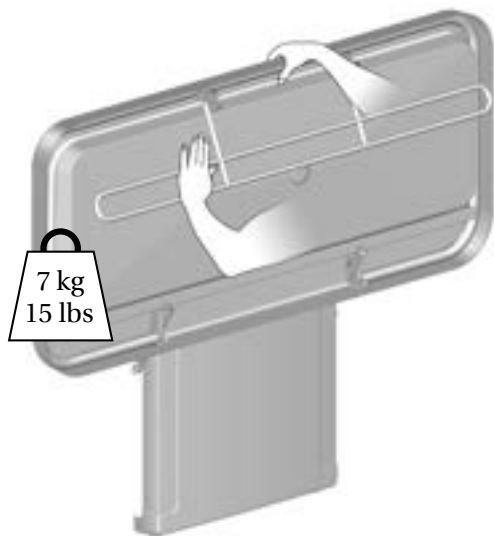
Use of the unit is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the unit only after consulting with the attending physician.

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

Operating instructions

Support the shower table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the shower table up against the wall after use.

Take care to avoid injury to hands when in the vicinity of closing mechanical parts.

CAUTION

Always be careful when moving a person to and from the shower table.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

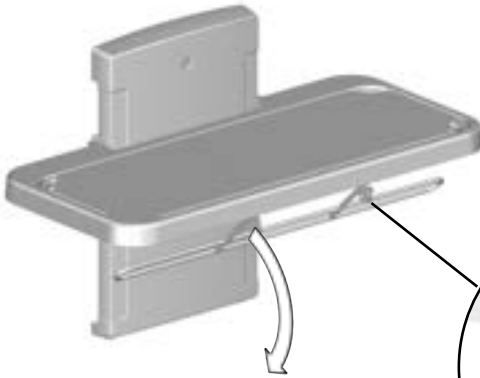
NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.


Safety rail

To pull the safety rail up: Swing the rail upwards from under the shower table. The safety rail will automatically lock into position.

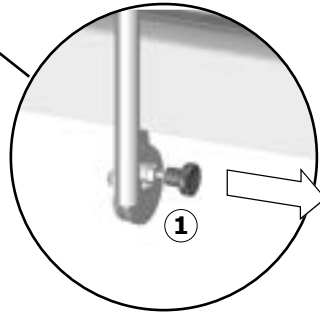To fold the safety rail away: Release the lock ① and swing the safety rail down under the shower table.

CAUTION

Never let a person lie on the nursing bench without the cover in place. The tray under the pads is for water collection only. It is not to be used as a bathtub. Risk of injury to the user and damage to the product.

Height adjustment

When the shower table is installed at the recommended height, it can be adjusted between 480 and 880 mm (18.9" and 34.6") above the floor.

The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the unit. When the button is released, the motion stops.

Press and hold the down button to lower the unit. When the button is released, the motion stops.

en

To avoid risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.

CAUTION

Never place objects underneath the shower table.

Do not leave the patient unattended.

Mesh/canvas cover

en

The frame with the cover can be removed for cleaning.
The cover can be zipped off and washed in regular washing machine at max 70° C.

Dry the water collection tray.

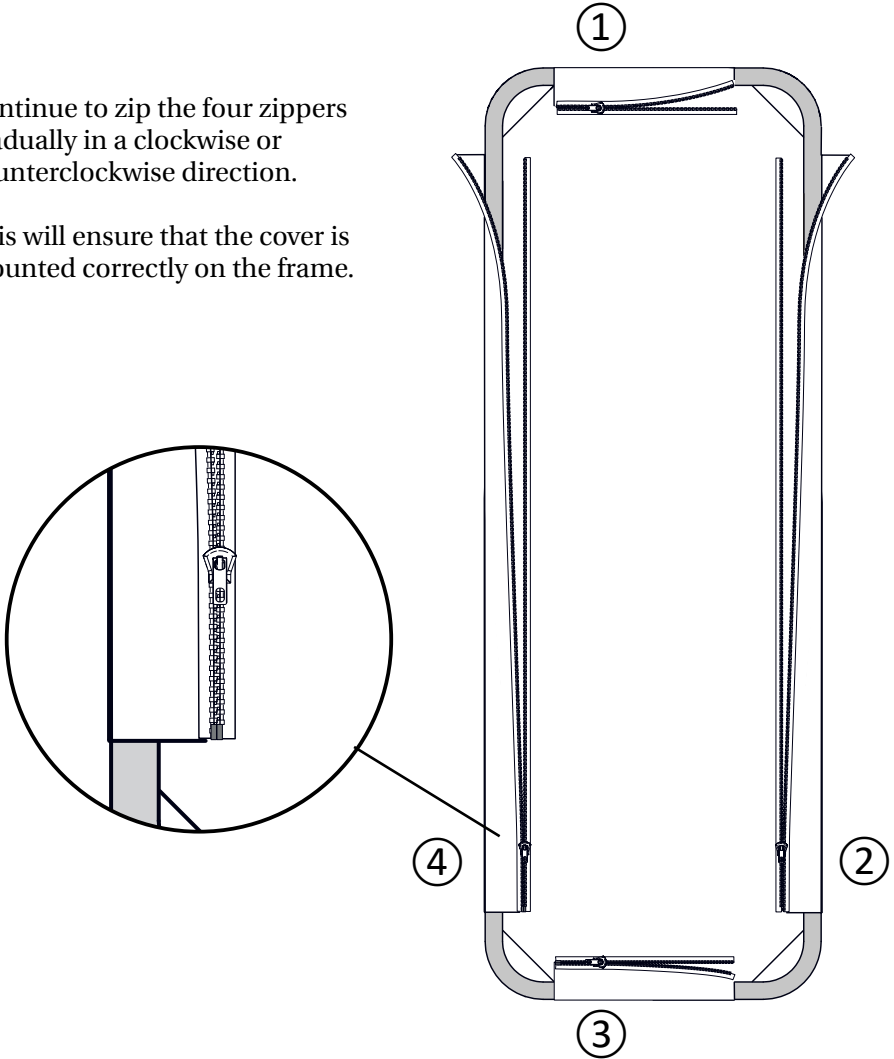
Zippering the cover onto the frame

When the cover has been removed for washing, it must be zipped back onto the frame again.

Zip all zippers approximately 5 cm (2 inches).

Continue to zip the four zippers gradually in a clockwise or counterclockwise direction.

This will ensure that the cover is mounted correctly on the frame.

en

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

Disinfection

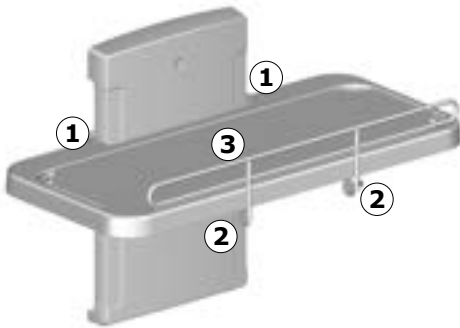
When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection. A list of suitable disinfectants can be found at pressalit.com.

Maintenance

Expected service life of the shower table is 10 years, provided that maintenance is carried out as described below:


It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the shower table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

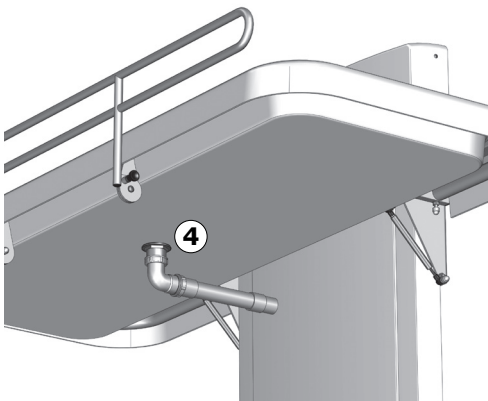
- Check the hinges ① on the shower table and the safety rail ② . Tighten if necessary.
- Check the cover ③ for damage and replace if necessary.


- Check the drain pipe connections ④, and check and clean the water trap (if fitted).

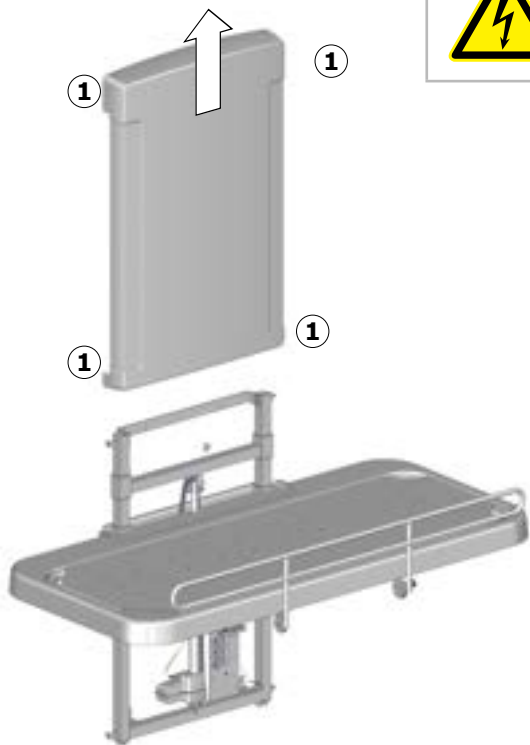Disconnect mains before removing the covers.

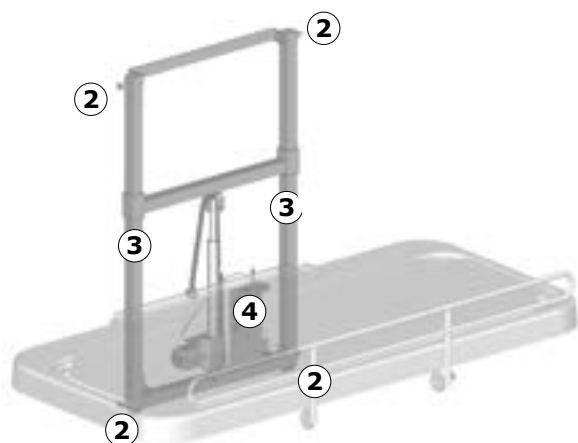- Remove the upper and lower retaining screws ① on the cover and remove it carefully.

- Check the wall brackets ② and tighten if necessary.
- Oil moving parts, especially the sliding brackets ③.
- Check all electrical connections to the control box ④.
- Put the unit cover back again and tighten the retaining screws.
- Reconnect mains.

en

Resetting

If the height adjustment does not work, and the control box beeps and flashes, the product must be reset.

Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.


Take the shower table to its lowest position. The shower table is now reset.

Trouble-shooting

| Symptom | Cause | Action |
|--|----------------------------------|--|
| The shower table does not stay in upright position. | The gas cylinders are defective. | Contact Pressalit or your local dealer. |
| The product does not work, even though the remote control is used correctly. | Fault in power supply. | Check that all wires and plugs are intact and correctly connected. Contact Pressalit or your local dealer. |
| The product does not work, and the control box beeps and flashes. | The product needs to be reset. | See "Resetting". |

| | | |
|--|---|--|
| The water does not run out of the water collection tray. | Adjustment screws too high or too low. | Adjust the adjustment screws ①. |
| The shower table runs up and down unevenly. | The sliding brackets are not properly lubricated. | Oil the sliding brackets as described under “Maintenance”. |

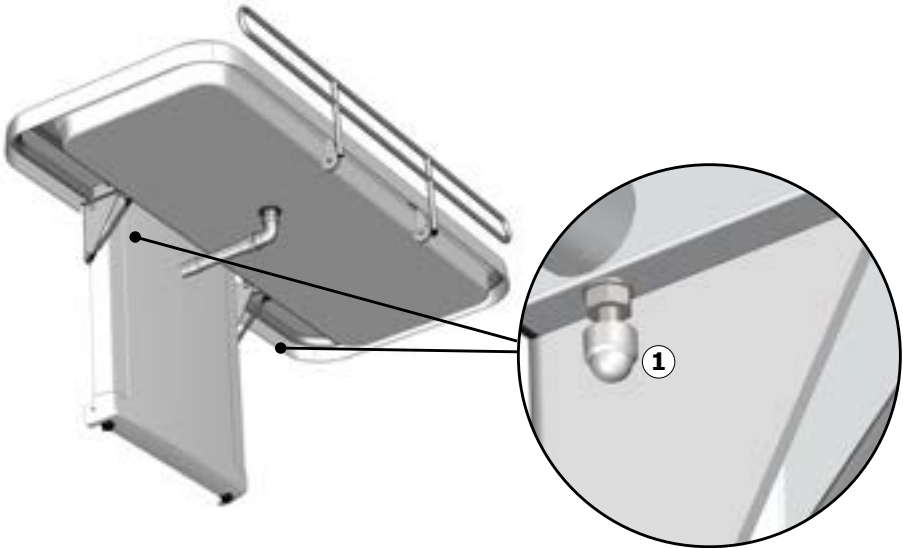en

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user can still get on and off the product.

| Cable | Length |
|---------------------------|-----------------------------|
| Power cable | 3,2 m (126") |
| Hand control spiral cable | Approx. 5 m (197") extended |

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

en

Electromagnetic immunity

| Electrostatic discharge | |
|---|--|
|  | The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective. |
| Test standard | Immunity test level |
| EN 61000-4-2 | +/- 8 kV contact +/- 2, 4, 8, 15 kV air |


| Radiated RF electromagnetic field | |
|---|--|
|  | The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective. |
| Test standard | Immunity test level |
| EN 61000-4-3 | 15 V/m 80 MHz to 1000 MHz (only on the front) 15 V/m 1,01 GHz to 3 GHz 80 % AM 1 kHz |

en

Immunity to proximity fields from RF wireless communication equipment

| | |
|---|--|
|  | <p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p> |
| <p>Test standard</p> | <p>Immunity test level</p> |
| <p>EN 61000-4-3*</p> | <p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave) 450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine 710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p> |

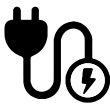Electrical fast transient/burst

| | |
|--|--|
|  | <p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p> |
| <p>Test standard</p> | <p>Immunity test level</p> |
| <p>EN 61000-4-4</p> | <p>+/- 1 kV , +/- 2 kV 100 kHz repetition frequency</p> |


Surge immunity

| | |
|---|--|
|  | The product must not be affected to initiate actions or to render components defective as a result of lightning. |
| Test standard | Immunity test level |
| EN 61000-4-5 | Line-to-line: +/- 0.5 kV, +/-1 kV Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV |

Immunity to conducted disturbances induced by RF fields

| | |
|---|---|
|  | The wires must not be affected by other RF fields so as to initiate actions or render components defective. |
| Test standard | Immunity test level |
| EN 61000-4-6 | 3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz |

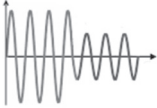Voltage dips

| | |
|---|---|
|  | The product must not initiate or limit actions as a result of a dip in the power supply. |
| Test standard | Immunity test level |
| EN 61000-4-11 | 0 % U_T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U_T ; 1 cycle at 0°, 70 % U_T ; 25 cycles at 0° Single phase at 0° |

Voltage interruptions

| | |
|---|---|
|  | The product must not initiate or limit actions as a result of a short-term power supply stop. |
| Test standard | Immunity test level |
| EN 61000-4-11 | 0 % U_T ; 300 cycle |

Electromagnetic emission


| Conducted RF emission | |
|---|--|
|  | The product must not send too much noise into the grid. This could lead to a disruption of other products. |
| Test standard | Emission test class and group |
| EN 55011 | Class B, group 1 |

| Radiated RF emission | |
|---|--|
|  | The product must not emit RF signals in the room. This could lead to a disruption of other products. |
| Test standard | Emission test class and group |
| EN 55011 | Class B, group 1 |

| Harmonic current emissions | |
|---|--|
|  | The product must not unintentionally draw power from the grid. |
| Test standard | Emission test class and group |
| IEC 61000-3-2 | Class A |

| Voltage changes, voltage fluctuations and flicker | |
|---|---|
|  | The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room. |
| Test standard | Emission test class and group |
| EN 61000-3-3 | Tested according to the standard. |

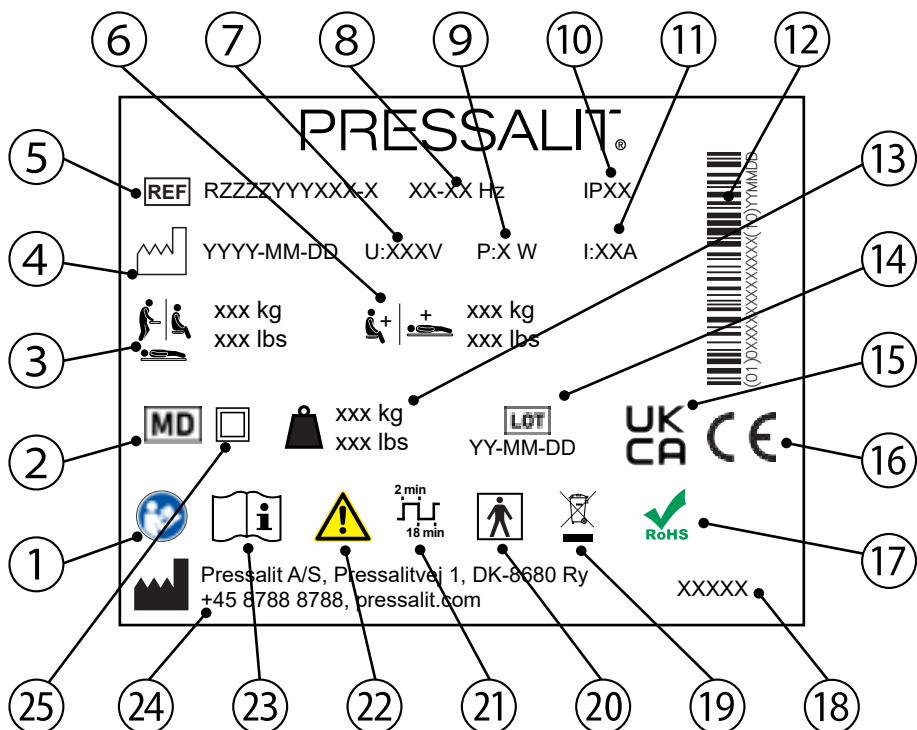Product label

en

- ① Consult manual for use

- ② Medical device

- ③ Maximum user weight

- ④ Production date

- ⑤ Article number

- ⑥ Max load

- ⑦ Mains voltage

- ⑧ Frequency

- ⑨ Maximum power consumption

⑩ IP class

⑪ Electric Current

⑫ GS1-128 barcode

⑬ Product weight

⑭ Batch code

⑮ UK Conformity Assessed

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at pressalit.com

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑱ Product name (if any)

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

㉑ Duty cycle

㉒ Consult the manual for important cautionary information such as warnings and precautions

㉓ Consult manual for use

㉔ Name and address of manufacturer

㉕ Double insulated electrical appliance

Technical data

| | |
|-----------|--|
| Materials | Powder laquered stainless steel Plastics: PVC (canvas) Polystyrene (cover) |
| IP class | The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction. |

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product. en

| Recyclable components | Material |
|------------------------------|---|
| Steel | Steel can be recycled by being re-melted and included as a secondary material in the production of new steel. |
| Plastics | Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production. |
| Electronics | The electronic components should be collected separately and returned to the designated recycling service. |

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