R9412717 - R9412721
R9432717 - R9432721
R9452717 - R9452721
R9462717 - R9462721

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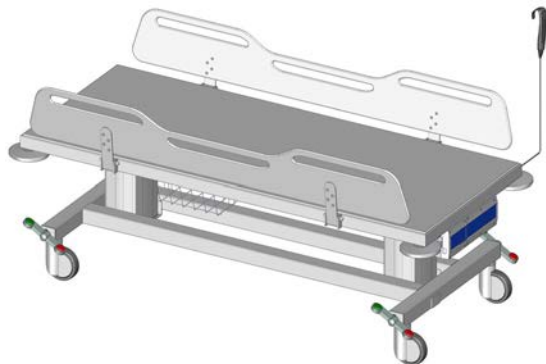
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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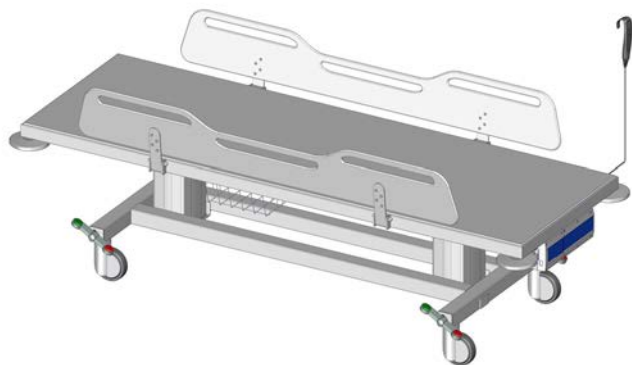
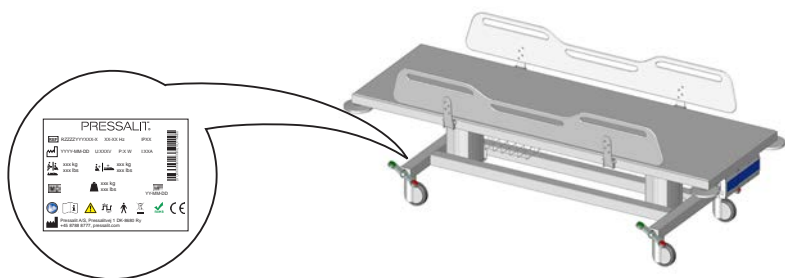
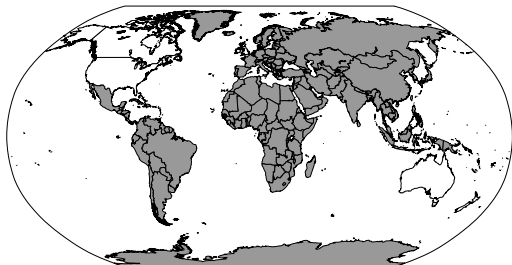
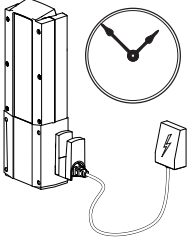
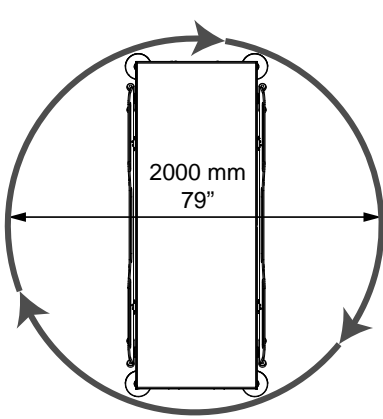
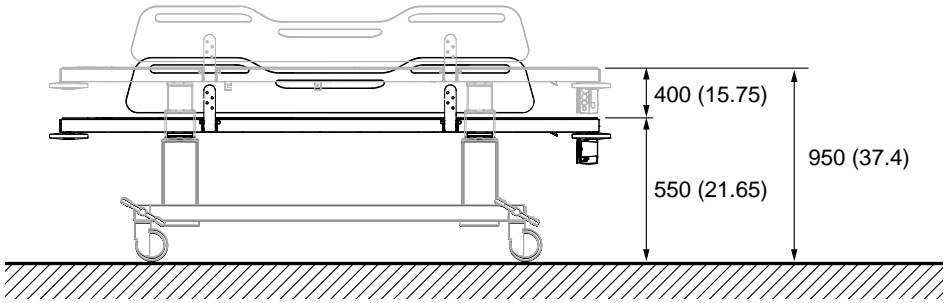
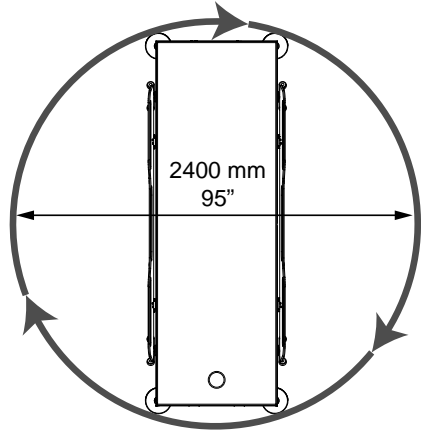
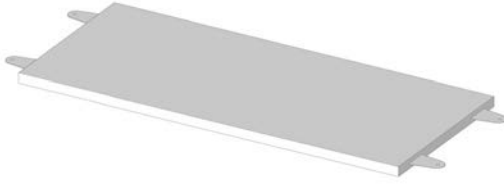
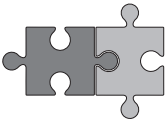
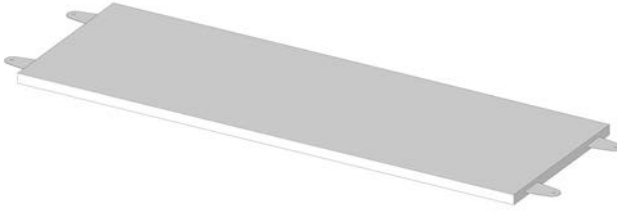
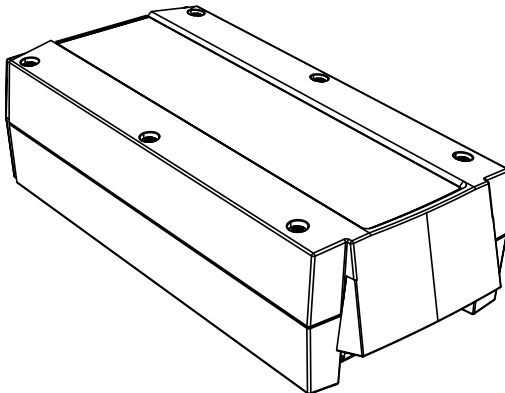
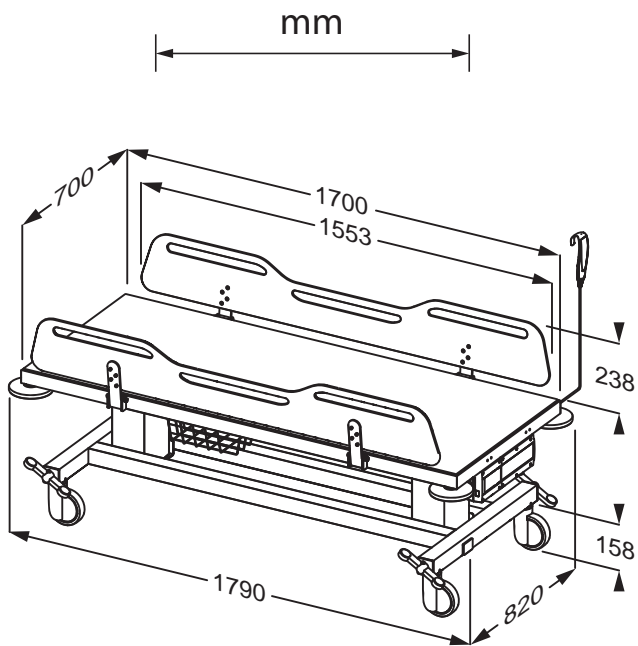
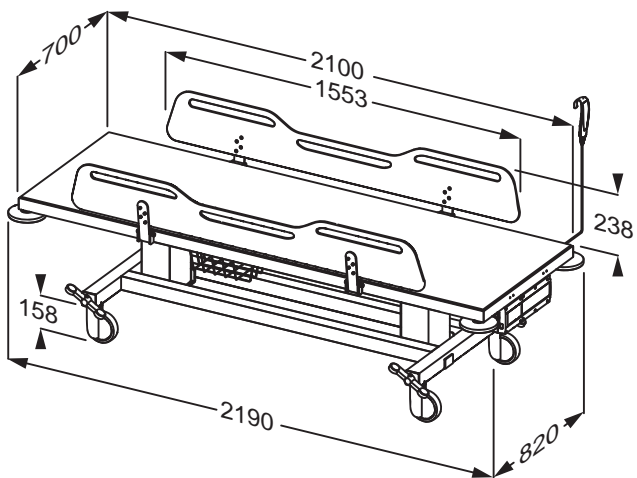
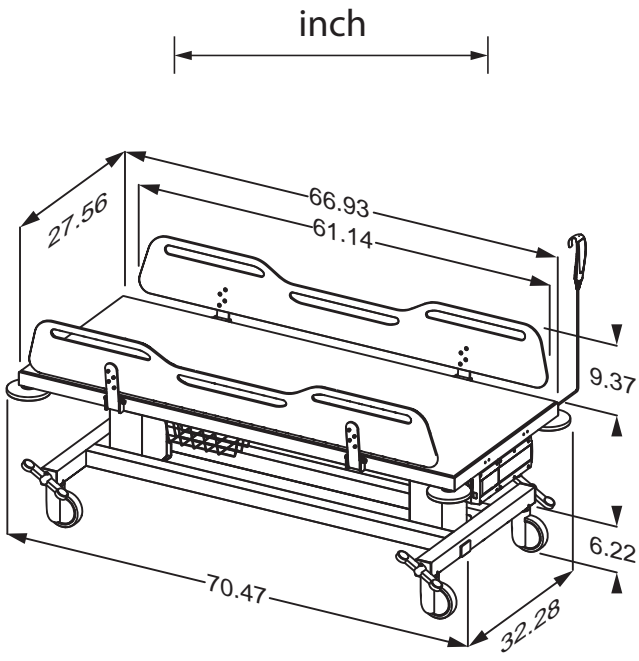
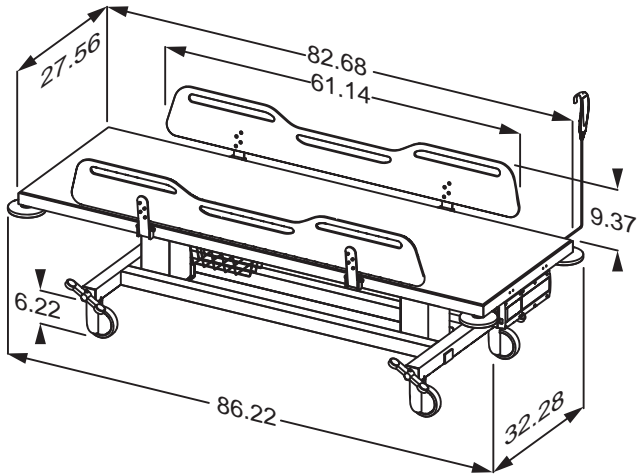
R9432721, R9452721, R9412721, R9462721

Content

Symbols used in this manual	11
General safety	13
Safety distances	14
Intended use	16
Intended operator profile	16
Intended user profile	16
Operating environment	16
Contraindications for use	17
Biocompatibility	17
Operating instructions	18
Safety brakes	18
Side supports	19
Height adjustment	20
Resetting	20
Charging the battery	21
Accessories	23
Mattress	23
Cleaning	24
Electromagnetic compatibility	25
Electromagnetic emissions	25
Electromagnetic immunity	26
Product label	27
Storage and transportation	29
Troubleshooting	29
Disposal and recycling	30

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Head injury warning

Hand injury warning

Genitalia injury warning

Feet injury warning

Do not leave the patient unattended.

Compatible products

Spare parts

Max. user weight

Article number/ versions

Product weight

Electrical data

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

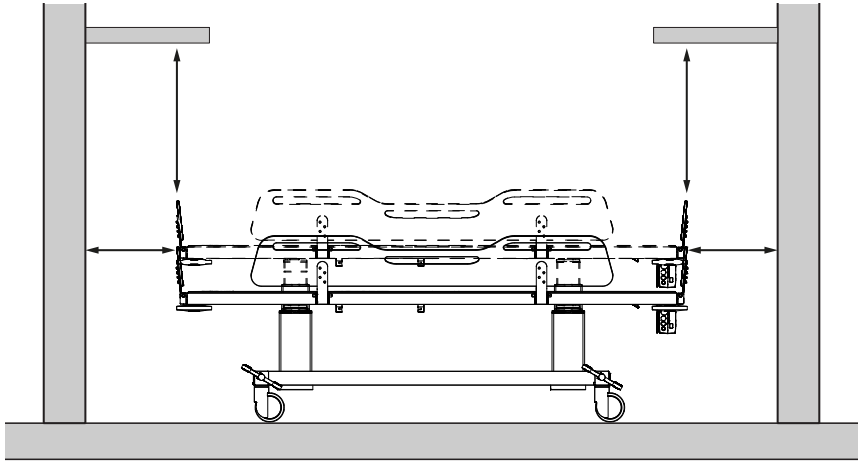
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.





Safety distances

Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

en

ISO 17966:2016		
	Max 120 mm (4.72") Min 300 mm (11.81")	Max 60 mm (2.36") Min 300 mm (11.81")
	Max 8 mm (0.31") Min 25 mm (0.98")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 8 mm (0.31") Min 75 mm (2.95")	Max 8 mm (0.31") Min 75 mm (2.95")
	Max 35 mm (1.38") Min 120 mm (4.72")	Max 25 mm (0.98") Min 120 mm (4.72")

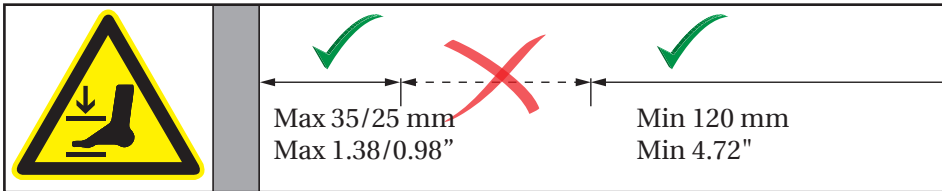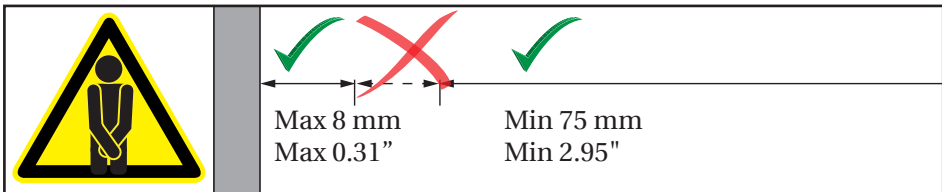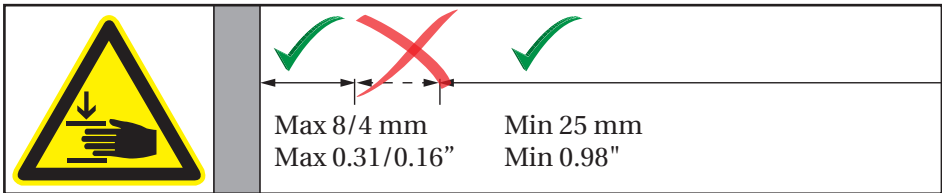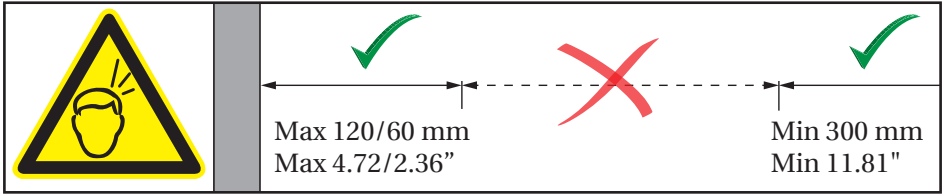 **WARNING**

If the unit moves within the safety distances, you must keep an especially close eye on surrounding people and objects to avoid crushing hazards.

en

Intended use

This product is intended for changing a person who is unable to stand or sit. The person is lying horizontally on the change trolley.

Intended operator profile

The product is to be used by personnel trained in health care and trained in changing a person in their care.

Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in public changing places, schools, private homes, nursing homes and hospitals under normal working conditions.

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Contraindications for use

Use of the change trolley is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the change trolley only after consulting with the attending physician.

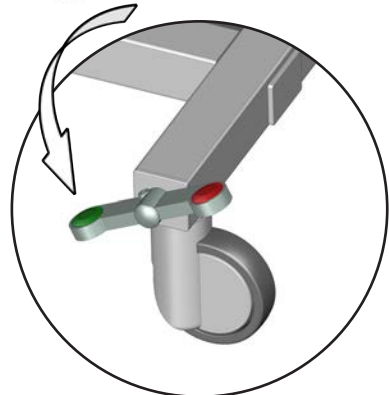
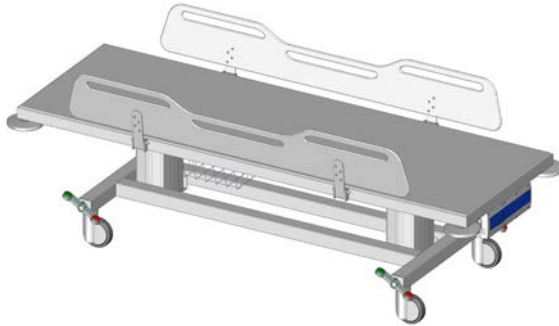
Biocompatibility

The materials in contact with the skin of the patient or the user during the changing procedure, have been tested for their content of harmful substances according to ISO 10993 Biocompatibility, and are therefore considered suitable for this purpose.

Operating instructions

Safety brakes

The change trolley is equipped with safety brakes on all four castor wheels to ensure that it will not move unintentionally.

To lock the brakes: Step on the red button on one of the four rocker pedals to lock the brakes on all four wheels.

To unlock the brakes: Step on the green button on one of the four rocker pedals to unlock the brakes on all four wheels.

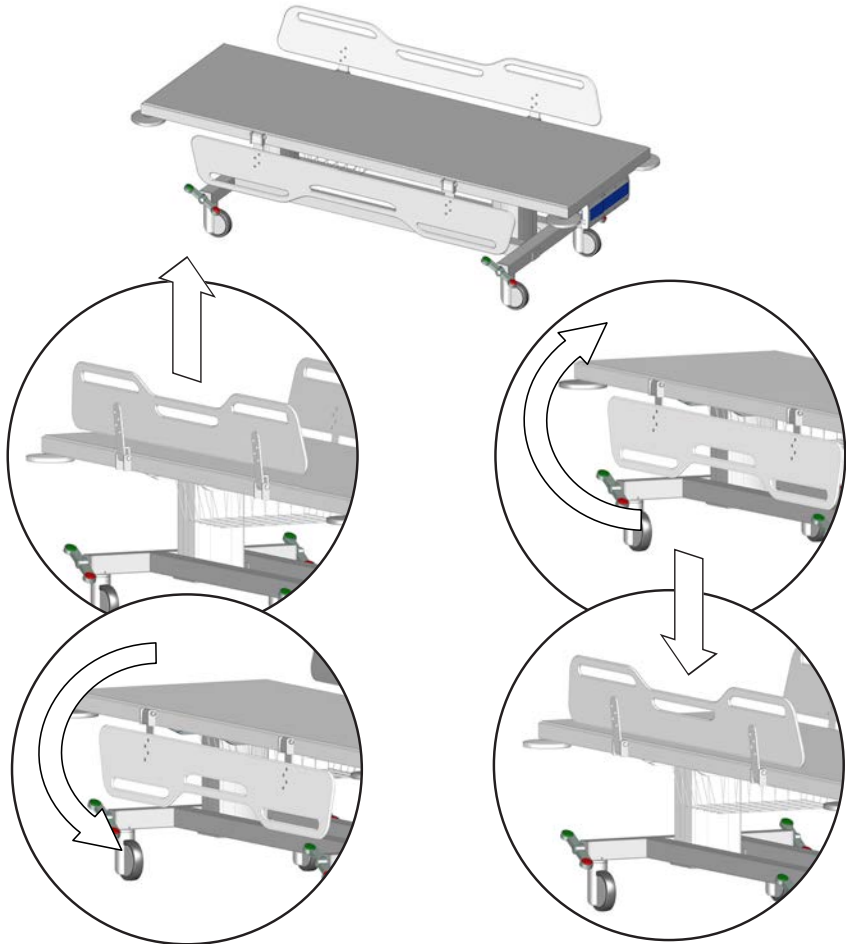
Always ensure that the brakes are locked when transferring a patient to and from the change trolley.

Do not leave the patient unattended.

Side supports

The side supports can be folded up and down individually for easy transfer of the patient.


To fold down: Lift the side support upwards to unlock it, and swing it down.

To fold up: Swing the side support up and push it down to lock it in place.

When the trolley is in use, make sure that all side supports are erect and in locked position to protect the patient from falling out of the unit.

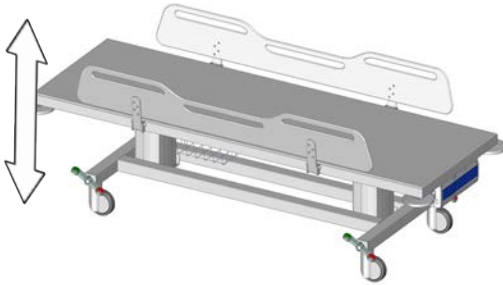Height adjustment

The change trolley can be adjusted 400 mm / 15¾" up and down.

**Max user weight:
200 kg / 440 lbs**

**Max load:
250 kg / 550 lbs**

The height adjustment is controlled by the wired remote control unit.

	Press and hold the Up button to raise the unit. When the button is released, the motion stops.
	Press and hold the Down button to lower the unit. When the button is released, the motion stops.

en

	To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.
--	---


			Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.
---	---	---	---

Resetting

If the lifting function does not work properly or if the two lifting columns are out of sync, try resetting the lifting system. Press and hold the up button and the down button simultaneously. The columns will move down after a few seconds while a beep tone is heard. Keep holding both buttons until the beep tone disappears. The lifting columns are now reset.


Charging the battery

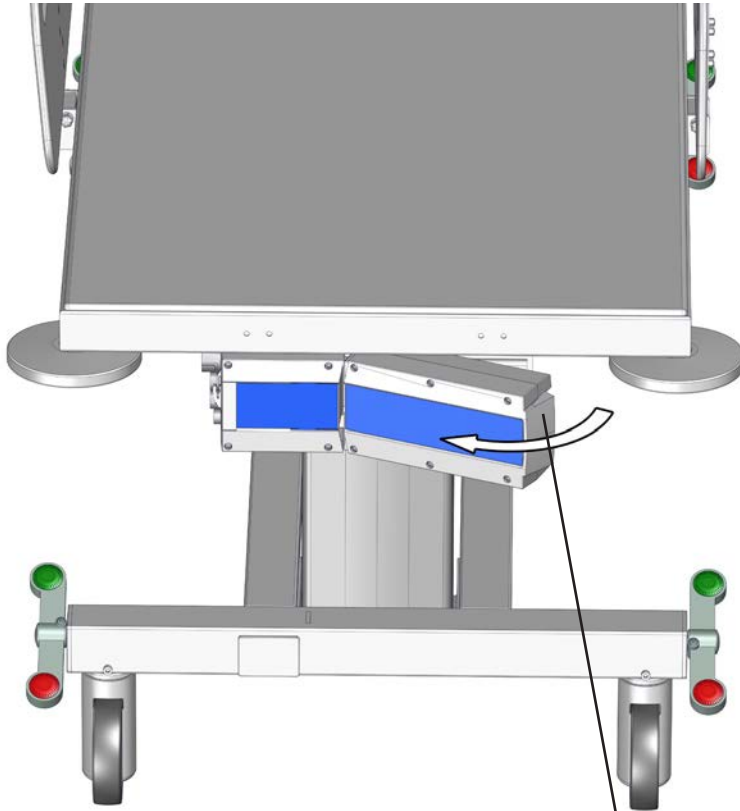
The battery can height adjust the change trolley up to 30 times when fully charged.

Release the lock to remove the battery for charging.

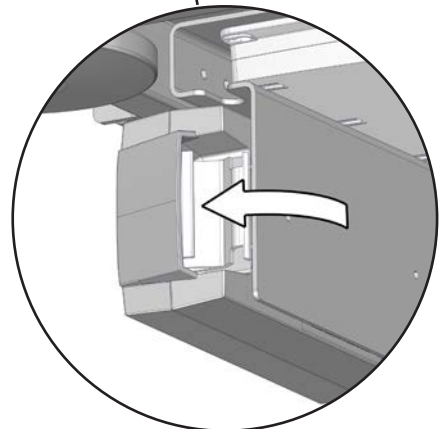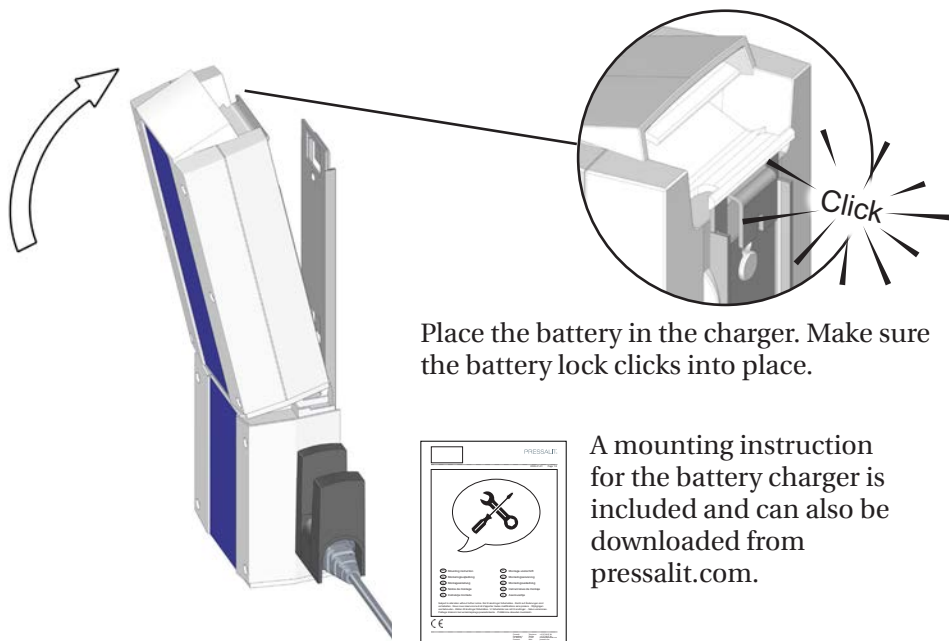Reattach the battery after charging before placing the patient on the change trolley.

Place the battery in the charger. Make sure the battery lock clicks into place.

A mounting instruction for the battery charger is included and can also be downloaded from pressalit.com.

CAUTION

When the battery is running low, a beep tone is heard when you operate the remote control. Although there is still enough power to raise or lower the unit, it is of utmost importance that you charge or replace the battery immediately. Failure to do so will soon cause the lifting function to stop working.

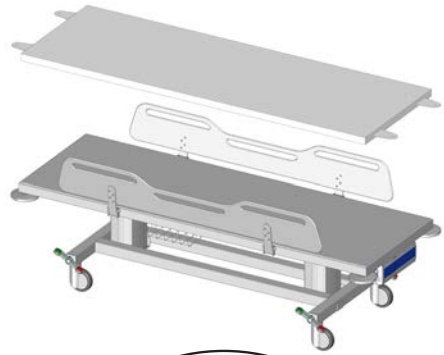
IMPORTANT

If the battery has not been used for more than a week, it enters into standby mode to save power and to protect the battery. Place the battery in the charger for approx. 15 seconds to exit the standby mode before use. Please note that there is no audio signal to indicate the standby mode or to indicate exit of standby mode. If there is still no power after exit of standby mode, the battery needs to be charged. The battery must be recharged every 6 months.


Accessories

Mattress

The optional mattress can be placed on the change trolley for optimal comfort. Both sides of the mattress can be used, as it is reversible.

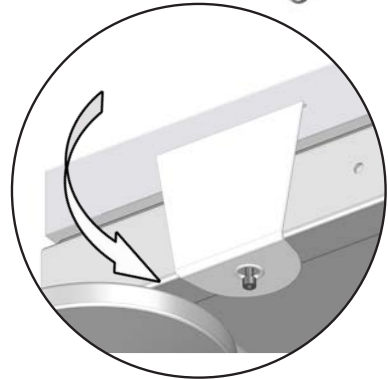
Unscrew the locking screw for the mattress ① approximately one centimeter.

Pull the mattress flap down over the screw. Do the same with all four flaps.

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/ disinfectants tested by Pressalit can be found on pressalit.com.

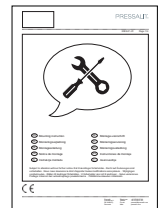A mounting instruction is included with the mattress, and can also be downloaded from pressalit.com.


IMPORTANT

Check for damages on the mattress on a regular basis. This will prevent infection of the mattress.

Cleaning

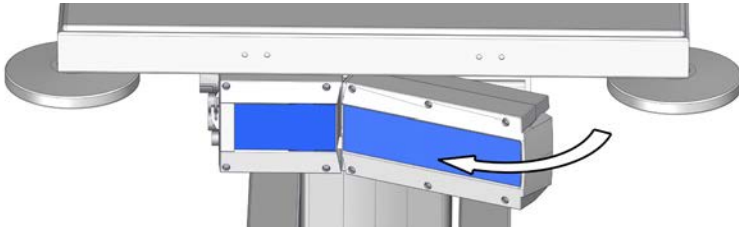Remove the battery before cleaning the change trolley.

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found on pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

CAUTION

Do not perform any type of cleaning or disinfection of the change trolley while the patient is lying on it.

Electromagnetic compatibility

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic emissions

en


Test standard	Test level / details	Test results
60601-1-2 ch. 7.1.7 - CISPR 14	30MHz – 1GHz	Passed

Electromagnetic immunity

Test standard	Compliance level	Test level / details	Test results
EN60601-1-2 and EN17966 (Home healthcare environment)	=> 10V/m 2,7GHz	15V/m Frequency range: 80MHz- 6GHz 1% step size	Passed
EN60601-1-2 (Proximity fields from RF wireless communications equipment)	=> 10V/m 2,7GHz	15V/m Frequency range: 385 - 5785 MHz	Passed
IEC 61000-4-4 (Electrical fast transients / bursts)		Quick pulse simulation	Passed
Conducted disturbances induced by RF	Pulse subjection, 6V	Pulse subjection, 10V	Passed
Electrostatic discharge		Statically charged person discharge on unit, 8kV and 15kV	Passed (*)

* If a statically charged person discharges on or near the shower trolley control box, a fault may occur in the lifting columns, and a reset must be performed. See "**Resetting**".

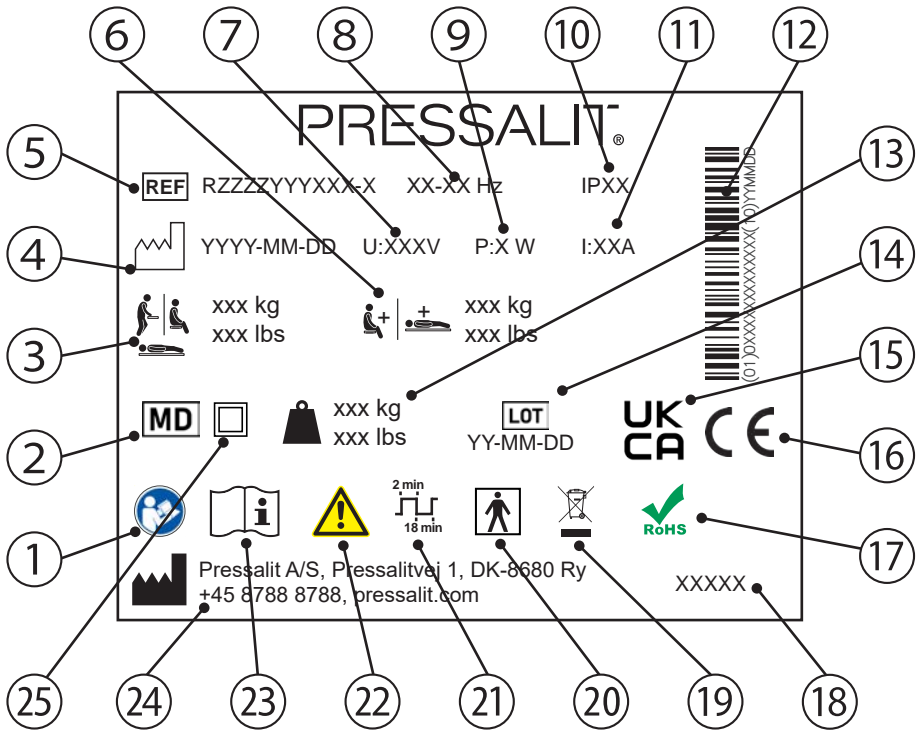Product label

- ① Consult manual for use

- ② Medical device

- ③ Maximum user weight

- ④ Production date

- ⑤ Article number

- ⑥ Max load

- ⑦ Mains voltage

- ⑧ Frequency

- ⑨ Maximum power consumption

en

⑩ IP class

⑪ Electric Current

⑫ GS1-128 barcode

⑬ Product weight

⑭ Batch code

⑮ UK Conformity Assessed

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at pressalit.com

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑱ Product name (if any)

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

㉑ Duty cycle

㉒ Consult the manual for important cautionary information such as warnings and precautions

㉓ Consult manual for use

㉔ Name and address of manufacturer

㉕ Double insulated electrical appliance

Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40 degrees Celsius.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Troubleshooting

Symptom	Cause	Action
No power.	Battery depleted.	Recharge the battery. See " Charging the battery ".
No power.	Battery in standby mode.	Place the battery in the charger for approx. 15 seconds. See " Charging the battery ".
The change trolley is skewed.	The lifting columns are out of sync.	Reset the lifting columns. See " Resetting ".
The trolley cannot be raised or lowered.	If a statically charged person discharges near the trolley control box, a fault may occur in the lifting columns.	Reset the lifting columns. See " Resetting ".

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Expected service life of the change trolley is 10 years.

en

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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